# USER MANUAL

**VICAIR®**

**MULTIFUNCTIONAL O2 BACK**

relief by air

<b>EN</b>	USER MANUAL	3
<b>NL</b>	GEbruikersHANDLEIDING	8
<b>DE</b>	GEBRAUCHSANWEISUNG	13
<b>FR</b>	MANUEL DE L'UTILISATEUR	19
<b>DA</b>	BRUGERVEJLEDNING	25
<b>SV</b>	BRUKSANVISNING	30
<b>NO</b>	BRUKERMANUAL	35
<b>FI</b>	KÄYTTÖOHJE	40
<b>ES</b>	MANUAL DEL USUARIO	45
<b>PT</b>	MANUAL DE UTILIZADOR	51
<b>IT</b>	MANUALE UTENTE	57
<b>GR</b>	ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ	63
<b>HR</b>	KORISNIČKI PRIRUČNIK	69
<b>SL</b>	UPORABNIŠKI PRIROČNIK	75
<b>PL</b>	INSTRUKCJA OBSŁUGI	80
<b>ZH</b>	用户手册	86
<b>JP</b>	ユーザーマニュアル	90

Dear customer,

Thank you for choosing a Vicair® product. We are sure you will appreciate the simplicity, reliability and fantastic effect. For quick installation you can use the quick installation guide provided with your cushion. In addition, please read and understand this user manual before use. Save both the Quick Installation Guide and this manual for future reference.

For more information or to (re)order manuals, contact your country's distributor.


Go to [vicair.com](http://vicair.com) for distributor information.

## INTRODUCTION

The Vicair Multifunctional O2 back is a wheelchair back support cushion that provides a very high level of skin protection for persons with a high risk of developing pressure ulcers. It can be used with hard shell back systems and tilt-in-space chairs to improve the body posture of asymmetric persons, to help prevent pressure ulcers and to provide comfort and stable body support.

It is also a high performance addition to a pressure ulcer treatment plan (stage I-IV). The Vicair Multifunctional O2 back is machine washable without removing the SmartCells!

## MAIN COMPONENTS

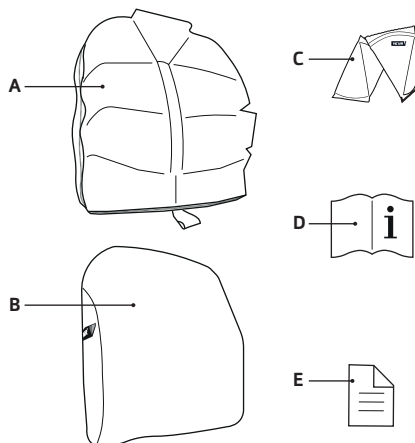- A. Cushion
- B. Cushion cover
- C. SmartCells™, placed in 10 compartments
- D. User manual
- E. Quick Installation Guide

## GENERAL PRODUCT SPECIFICATIONS

Product name:	Vicair Multifunctional O2 back
Manufactured by:	Vicair B.V., The Netherlands
Warranty:	24 months
Weight (excl. cover):	1,0 kg / 2.2 lbs
Available sizes:	See the grid in the back of the manual
Used materials:	All latex free
Cushion:	Top made of polyester fabric with polyurethane coating, bottom of polyester mesh fabric
Filling:	Air-filled SmartCells with low friction coating
Cover:	Top: Knitted Polyester fabric. Sides: polyester 3D mesh fabric

Machine washable. Cover complies with EN1021-1, 1021-2 and CAL117.

## PRECAUTIONS AND SAFETY INSTRUCTIONS

---

A Vicair wheelchair back support cushion may only be used as a wheelchair back support cushion. The use of this product in any other way, is considered to be misuse. Vicair B.V. cannot be held responsible for damage caused by misuse, improper use or negligence. Make sure that you read and understand this manual completely.

**Warning!** Vicair B.V. recommends to install the Vicair Multifunctional O2 back under direct supervision of a therapist or technical advisor.

**Warning!** Your back and spine should not touch the bottom/back part of the cushion or the sides of the shell. Checking for “bottoming out” should be done during first use and after adjustments to the cushion filling.

**Warning!** Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, consult your therapist or technical advisor.

**Warning!** Vicair B.V. recommends to adjust the filling grade under the direct supervision of your therapist or technical advisor.

**Warning!** No testing data is currently available about the safe use of Vicair back support cushions in motor vehicles. Therefore we do not encourage using the back support cushion in motor vehicles.

**Warning!** In case of soiling with body fluids, and when transferring the product to another user, the product must be disinfected to avoid cross contamination.

**Warning!** Every three months, check the product for wear and deflated cells. In case of serious damage or a large number of deflated cells, do not continue to use the product. Consult your dealer.

**Warning!** Using the Vicair Multifunctional O2 back may affect the static and dynamic behaviour of the wheelchair. Check and test the balance and stability of the wheelchair with the aid of your therapist or technical advisor.

**Caution!** Do not use or leave the product close to a naked flame or a high temperature heat source.

**Caution!** Do not steam clean or autoclave the product. The product cannot withstand excessive heat or pressure. When the given cleaning instructions are not followed, it is very likely that the product and/or the product's cover will age more rapid.

**Caution!** Do not use aggressive or abrasive cleaners, or sharp objects to clean the cushion.

**Caution!** Do not place the topside of the Vicair O2 cushion against the glass of the washer door.

## INSTALLATION

---

**Warning!** Vicair B.V. recommends to install the Vicair Multifunctional O2 back under direct supervision of a therapist or technical advisor.

**Warning!** Using the Vicair Multifunctional O2 back may affect the static and dynamic behaviour of the wheelchair. Check and test the balance and stability of the wheelchair with the aid of your therapist or technical advisor.

**Notice!** Always use cushion with cover.

**Notice!** Make sure that the wheelchair seat is clean and degreased.

**Notice!** After installation you can remove the back support cushion from the back shell by simply pulling it off the hook and loop strips.

**Notice!** The design of the cushion compartment layout is based on the natural curves of the spine. When installing the Vicair Multifunctional O2 back ensure that the bottom of the back support cushion is placed adjacent to the top of the wheelchair seat cushion.

## Perform the following steps when installing the Vicair Multifunctional O2 back

These steps correspond to the installation images 1 – 4 on the Quick Installation Guide:

- Check cushion size. Width (X) of the back support cushion, back shell and wheelchair seat should match. Height (Y) of the back support cushion should match the height of the back shell.
- Remove protective backing tape from selfadhesive hook strips attached to the cushion cover.
- Place the cushion in the back shell and push it firmly into position. Make sure the cushion label faces down.
- Check together with therapist or technical advisor if the back support cushion fits the dimensions of your body. Add or remove SmartCells if necessary.
- Sit in the wheelchair for a few minutes, to allow the back support cushion to adapt to your body shape.

### Checking for “bottoming out”

Image 5 - Quick Installation Guide

**Warning!** Your back and spine should not touch the bottom/back part of the cushion or the sides of the shell. Checking for “bottoming out” should be done during first use and after adjustments to the cushion filling.

**Warning!** Vicair B.V. recommends to adjust the filling grade under the direct supervision of your therapist or technical advisor.

- Check for bottoming out, after sitting with your back against the back support cushion for approximately 5 minutes.
- Lean slightly forward, let your therapist or technical advisor slide a hand between your back and the cushion, then lean back again.

- Ask your therapist or technical advisor to check if your back is supported by at least 2,5 cm / 1” of SmartCells.
- Do not use the cushion if bottoming out occurs. In most cases, adjusting the filling grade of the compartments will solve the problem.

### Stability

**Warning!** Using the Vicair Multifunctional O2 back may affect the static and dynamic behaviour of the wheelchair. Check and test the balance and stability of the wheelchair with the aid of your therapist or technical advisor.

### USE

---

#### Checking for redness of the skin

Image 6 - Quick Installation Guide

**Warning!** Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, consult your therapist or technical advisor.

#### Adjusting the filling grade

**Warning!** Vicair B.V. recommends to adjust the filling grade under the direct supervision of your therapist or technical advisor.

A Vicair wheelchair back support cushion has its own unique filling grade, depending on size and model and automatically manages small obliquities. However – if necessary to support more extreme asymmetry – the filling grade can be adjusted by removing or adding SmartCells from/to the various compartments. You can find the default filling grade on the label attached to the cushion.

## Conditions for use and storage

**Caution!** Do not use or leave the product close to a naked flame or a high temperature heat source.

- **User weight:**  
Minimum: 15 kg / 33 lbs,  
maximum: 150 kg / 331 lbs
- **Temperature:**  
Minimum: -20 °C / 4 °F,  
maximum: 55 °C / 131 °F
- **Use at high altitude**

**Notice!** Vicair B.V. recommends not to use the cushion above 2400 m (8000 ft) above sea level or below 50 m (164 ft) below sea level. Because each SmartCell is filled with air, the cushion may behave differently when used at high altitude or in an airplane. Under such circumstances, the cushion's firmness can be adjusted by removing a few cells from each compartment.

- **Air humidity:**  
Minimum: 0%, maximum 100%

## Use of Vicair wheelchair cushions in motor vehicles

**Warning!** No testing data is currently available about the safe use of Vicair back support cushions in motor vehicles. Therefore we do not encourage using the back support cushion in motor vehicles.

## Re-use - Transferring the product to another user

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. The product has to be thoroughly cleaned and disinfected, before another person is allowed to use the it. See chapter 'Maintenance and cleaning' for instructions.

**Notice!** Make sure the product is correctly adjusted to the other person.

## Replacing the cushion cover

Under normal use, Vicair B.V. recommends to replace the cover every year. Replacement covers can be purchased separately.

## Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulation. Ensure the product is cleaned prior to disposal to avoid any risk of contamination.

## MAINTENANCE AND CLEANING

---

**Warning!** Every three months, check the product for wear and deflated cells. In case of serious damage or a large number of deflated cells, do not continue to use the product. Consult your dealer.

**Caution!** Do not steam clean or autoclave the product. The product cannot withstand excessive heat or pressure. When the given cleaning instructions are not followed, it is very likely that the product and/or the product's cover will age more rapid.

**Caution!** Do not use aggressive or abrasive cleaners, or sharp objects to clean the cushion.

**Caution!** Do not place the topside of the Vicair O2 cushion against the glass of the washer door.

## Cleaning the Vicair Multifunctional O2 back

- **Cleaning the cushion cover**

The cushion cover can be machine washed. Follow the washing instructions on the label attached to the inside of the cover.

(60 °C= 140 °F)

- **Cleaning the cushion**

Your Vicair Multifunctional O2 back support cushion can be machine washed at 60°C / 140 °F using an anti-crease cycle and mild detergent. You do not need to remove the SmartCells before putting the cushion into the washing machine. Make sure the topside of the cushion is not placed against the glass of the washer door. Do not load the machine with more than half of its maximum load weight. Spin speed should be reduced. When using a machine with agitator, make sure that the cushion can move freely within the drum. Do not use any chlorine bleach and do not dry clean! Do not tumble dry your Vicair O2 cushion but let it air dry (5-6 hours).

## Disinfecting your Vicair Multifunctional O2 back

**Warning!** In case of soiling with body fluids, and when transferring the product to another user, the product must be disinfected to avoid cross contamination. Clean the cushion and cover as instructed above. Preferably use an antibacterial detergent. After washing spray your cushion and cover with a disinfectant. Bend compartments outwards to make sure the spray gets into the gaps.

## WARRANTY

---

Please see [vicair.com](http://vicair.com) for complete information on warranty processes, terms and conditions. Vicair B.V. offers a limited warranty against defects in workmanship and materials, provided the product has been used normally. Any damage arising from abnormal use, or caused by improper handling, cleaning, or negligence, is excluded from this warranty.

### Warranty period

From original date of purchase: 24 months.  
The SmartCells in this Vicair O2 cushion have a lifelong warranty period.

Should a defect in materials or workmanship occur within the above mentioned warranty period, Vicair B.V. will – at its own choice – repair or replace the product free of charge. The cushion cover is excluded from this warranty. Remedies for breach of express warranties herein are limited to repair or replacement of the product.

In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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