User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

9063

EN

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Verze: C, 2022-07

All rights reserved, including translation.

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1 Preface

First of all we wish to thank you for the trust you placed in us by selecting a Vermeiren product.

Before using the toilet chair, read the instruction manual carefully: it will familiarise you with this product.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

Information available

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

2 Before use

2.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- Toilet chair + cushioned seat
- Cushioned lid
- Toilet bucket with cover
- Instruction manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

2.2 Intended use

Indications and contraindications: Mobile, reusable toilet commode for indoor use, intended for users who suffer from incontinence or mobility problems. Should not be used for taking a shower, for storage, as a ladder, or over built-in toilets.

The toilet chair is designed and produced solely to be used as toilet aid for one person with a maximum weight of 120 kg.

2.3 Safety instructions

 CAUTION Risk of injuries

Please read and follow the instructions in this manual. Otherwise you may get injured, or your toilet chair may get damaged.

- When setting up the chair and when it is transported, carefully guard against bruising and shearing – danger of injury.
- Do not grasp further sideways than the length of your arm allows – danger of injury.
- Before use, check whether all four legs stand evenly on the floor.
- Do not lean too far forwards – danger of tipping over.
- Be aware that some parts of your toilet chair may get very hot or cold due to ambient temperature, solar radiation, heating devices, etc. Be careful when touching.
- Remove the bucket before transporting the chair.
- Empty the bucket every time after use.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

3 Assembly

The toilet chair is delivered fully assembled. The bucket can now be pushed onto the guide rails below the seat from the left. When not in use, the seat can be covered by the cushioned lid supplied.

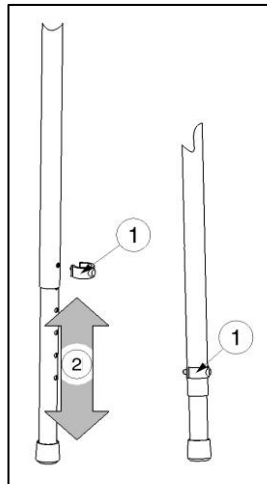
4 Usage

- Remove the bucket before transporting the chair.
- Remove and empty the bucket every time after use.
- Pour some water in the upper edge of the bucket. The bucket is then sealed airtight against bad odours when the lid is closed.

4.1 Height adjustment

1. Remove clip (1).
2. Adjust the leg to the desired position (2).
3. Mount clip (1).

Make sure that all four legs are well secured.

5 Maintenance

5.1 Cleaning

Use a soft, damp cloth and lukewarm water with a mild household soap to clean the toilet chair. Avoid soaking. Do not use abrasive cleaning agents to clean.

5.2 Disinfection

Use a soft, damp cloth and lukewarm water with a mild household disinfectant to clean the toilet chair.

In case of serious or permanent contamination, consider replacing the toilet bucket and cushioned seats.

5.3 Inspection / Service / Repairs

The expected service life of your toilet chair is 5 years. The lifespan of the toilet chair is influenced by its use, storage, maintenance, service and cleaning.

Before you resume use of the toilet chair, it should be inspected on the following points:

- Check the frame for plastic deformation and reduced functioning.

- Visual inspection of damage to the paintwork (danger of corrosion)
- Check the condition of the cushions (tears, brittleness, soaking)
- Bucket sealing (not in the case of re-using)
- Rubber tips on legs: no missing or damaged tips
- Completeness of consignment ?
- Instruction manual available?

Your toilet chair is constructed of steel and plastic components. We recommend to have your toilet chair inspected at least once a year, but the service frequency depends on the frequency and intensity of use. Consult your specialist dealer to make an agreement on service frequency.

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

5.4 Reuse

Before each reuse, have the toilet chair disinfected, inspected and serviced according to the instructions in this chapter.

5.5 Disposal at end of life

At end of life, you need to dispose your toilet chair according to the local environmental legislation. The best way to do so, is to disassemble the toilet chair to facilitate the transport of recyclable parts.

6 Technical details

Brand	Vermeiren
Type	Toilet chair
Model	9063

<i>Description</i>	<i>Dimensions</i>
Max. load	120 kg
Total height	755 - 885 mm
Seat height	470 - 560 mm (without cushioned lid) 490 - 580 mm (with cushioned lid)
Overall width	620 mm
Seat width	430 mm
Distance between armrests	470 mm
Backrest height	400 mm
Overall depth	650 mm
Seat depth	450 mm
Inner opening (LxW)	270 x 220 mm
Armrest to seat distance	215 mm (without cushioned lid) 195 mm (with cushioned lid)
Weight	ca. 9 kg
We reserve the right to introduce technical changes.	
Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.	

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Service registration form

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

Vermeiren GROUP
Vermeirenplein 1 / 15
2920 Kalmthout
BE

website: www.vermeiren.com