VICAIR®

the best in body support

LIBERTY TOILET CUSHION

GB

User manual

NL

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D

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N

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S

Bruksanvisning

DK

Brugermanual

SF

Käyttöohje

F

Mode d'emploi

E

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J

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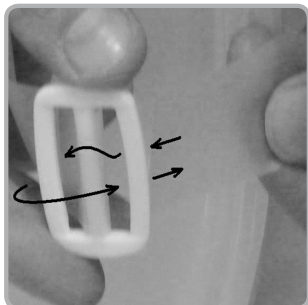
fig. B

INTRODUCTION

Your Vicair® Liberty TC is designed to be used as described in this manual. Please read the whole manual before use.

The cushion consists of the following parts:

Cushion

The inner cushion is made of a special made polyvinyl bag, filled with a specific number of triangular, air filled SmartCells™ and a small amount of lubricant. The Liberty TC has two compartments each with a valve on rear end.

Straps

With this cushion you will find four straps. You can use two straps to attach the cushion onto the toiletseat and two straps as reserve.

ASSEMBLY

Checking the cushion size

Together with your therapist or dealer, make sure the cushion fits the dimensions of your body and of your toiletseat.

Fitting the cushion on the toiletseat

- Remove the clear plastic wrapping.
- Place the cushion onto the toiletseat. The instruction pictograms on the cushion must face down. The valve must be on the rear end (fig. A).
- Install both straps on the left and the right of the toiletseat (fig. B, C).

USE

Use at high altitude

Because each SmartCell™ is filled with air, the cushion may become firmer and behave differently when used at high altitude or in an aeroplane. Vicair recommends not to use the cushion above 2400 m (8000 ft) ASL (above sea level).

Transferring the cushion to another user

Before allowing a second person to use the cushion, clean the cushion.

Flammability

Do not use or leave the cushion close to a naked flame or a high temperature heat source.

MAINTENANCE AND CLEANING

 **Caution:** Do not steam clean or autoclave the cushion. The cushion cannot withstand excessive heat or pressure.

Checking for wear and deflated cells

Every three months, check the cushion for wear and deflated cells. In case of serious damage, or a large number of deflated cells, do not continue to use the cushion. Consult your dealer.

Cleaning the cushion

- Wipe the cushion with a damp cloth using a mild detergent.
- Do not use aggressive or abrasive cleaners, or sharp objects to clean the cushion.
- Do not wash the cushion in a washing machine.

Disinfecting the cushion

In case of soiling with body fluids, and when transferring the cushion to another user, the cushion must be disinfected to avoid cross contamination.

- Clean the cushion as described above. Preferably use an anti-bacterial agent.
- Spray disinfectant on the cushion.

WARRANTY

Vicair offers a limited warranty against defects in workmanship and materials for a period of two (2) years after the original date of purchase, provided the product has been used normally. Any damage arising from abnormal use, or caused by improper handling, cleaning or negligence is excluded from this warranty. Should a defect in materials or workmanship occur within two (2) years from the original date of purchase, Vicair will, at its own choice, repair or replace the product free of charge. Remedies for breach of express warranties herein are limited to repair or replacement of the product. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

For additional product information, see our website: www.vicair.com.

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