VERMEIREN

D100 30°

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN

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D100 30° - Additions to manual D100

1 Product description (refer to manual D100 paragraph 1)

1.1 Technical specifications (add to manual D100 paragraph 1.2)


Technical terms below are valid for the wheelchair in standard settings. If other footrests / armrests or other accessories are used, the tabulated values will change.

Make	Vermeiren							
Address	Vermeirenplein 1/15, B-2920 Kalmthout							
Type	Manual wheelchair							
Model	D100 30°							
Maximum occupant mass	130 kg							
Description	Dimensions							
Effective seat width	380 mm	400 mm	420 mm	440 mm	460 mm	480 mm	500 mm	520 mm
Overall width (depends on the seat width)	580 mm	600 mm	620 mm	640 mm	660 mm	680 mm	700 mm	720 mm
Description	Dimensions Min.				Dimensions Max.			
Overall length with legrest	1010 mm							
Folded length	1010 mm							
Folded width	300 mm							
Folded height	880 mm				940 mm			
Total mass	18 kg							
Mass of heaviest part	10,5 kg							
Static stability downhill	16° (in standard configuration)							
Static stability uphill	9,5° (measured with anti-tipping, place axle block backwards)							
Static stability sideways	20° (in standard configuration)							
Maximum safe slope	12°							
Obstacle climbing	60 mm							
Seat plane angle	5°							
Effective seat depth	430 mm							
Seat surface height at front edge	470 mm				530 mm			
Backrest angle	5°				35°			
Backrest height	420 mm							
Distance between footrest and seat	385 mm				455 mm			
Angle between seat and footrest	8°							
Distance between armpad and seat	220 mm				240 mm			
Front location of armpad structure	410 mm							
Handrim diameter	535 mm							
Horizontal location of axle (deflection)	37 mm							
Minimum turning radius	1500 mm							
Diameter PU Rear wheels	22"				24"			
Diameter PU steering wheels	200 mm							
Storage and use temperature	+ 5 °C				+ 41 °C			
Storage and use humidity	30%				70%			

We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / °

Table 1: Technical specifications D100 30°

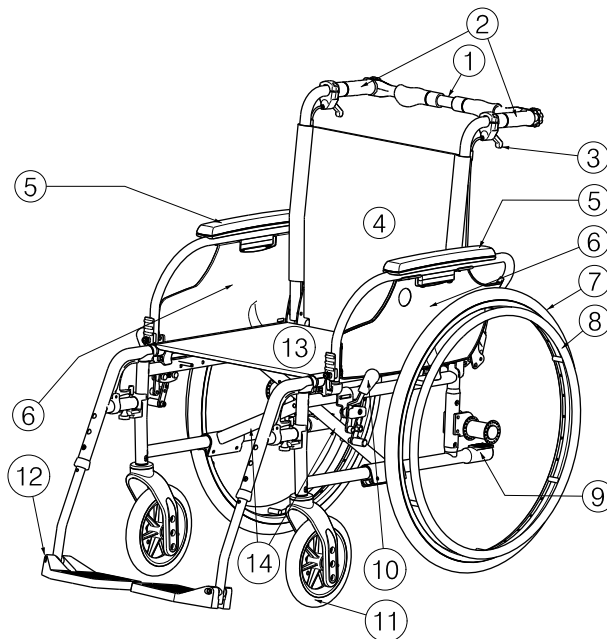1.2 Drawing (change paragraph 1.3 manual D100)

- 1 = Pushbar
- 2 = Handgrips
- 3 = Lever back inclination
- 4 = Backrest
- 5 = Armpads
- 6 = Armrests
- 7 = Driving wheels (rear wheels)
- 8 = Handrims
- 9 = Tip cap
- 10 = Brakes
- 11 = Steering wheels (front wheels)
- 12 = Footrests
- 13 = Seat
- 14 = Cross

1.3 Accessories (add to manual D100 paragraph 1.4)

The following accessories are available for the D100 30°:

- Drum brakes (B74)

1.4 Explanation of symbols (add to manual D100 paragraph 1.6)

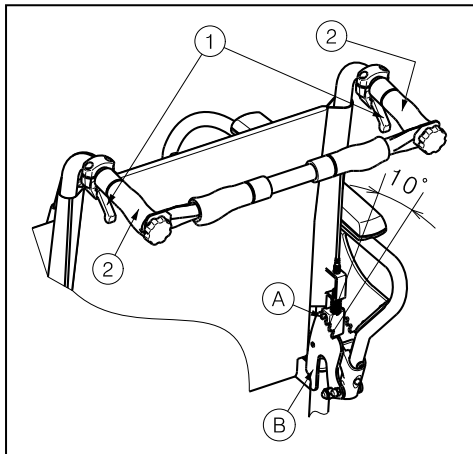
Not crash-tested

2 Use (refer to manual D100 paragraph 2)


2.1 Backrest inclination (add to manual D100)

- ⚠ **WARNING:** Risk of tipping over – Be aware that the stability decreases when the backrest is adjusted backwards.
- ⚠ **WARNING:** Risk of injury – Apply the parking brakes before adjusting the backrest.
- ⚠ **CAUTION:** Chance of pinching – Do not place your fingers, clothes between the adjustment mechanism.

With the model D100 30° it is possible to set the backrest backwards with a maximum inclination of 30° (4 different positions: steps 10°). Be sure that the patient sits in the chair when the attendant shall perform the back adjusting and that the wheelchair does not tip over.

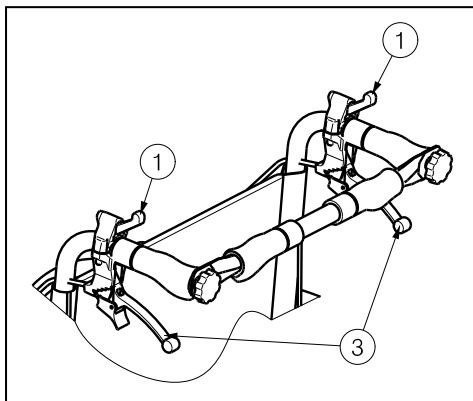
1. Pull lever ① on both sides towards the handgrip ② to adjust the backrest.
2. The pin ① comes out the groove of the toothed plate ②.
3. Pull both back tubes backwards continuously to the desired position (4 different positions: 5° - 15° - 25° - 35°).
4. Check that the pin ① is well fixed in the groove of the toothed plate ②.

If your wheelchair features drum brakes, the levers ③ are used as brakes.

2.2 Mounting or removing of pushbar (add to manual D100)

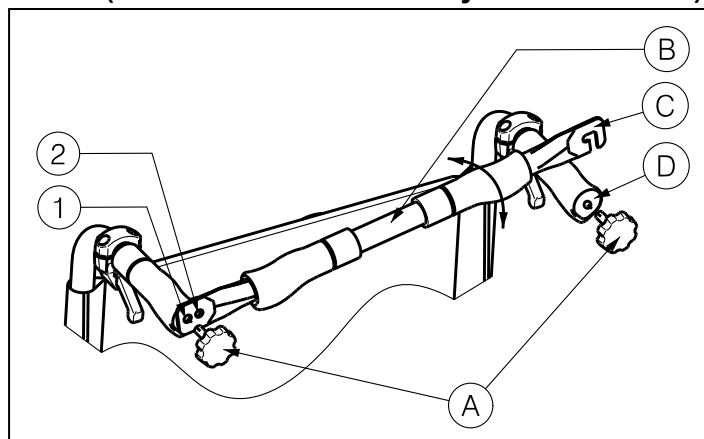- ⚠ **WARNING:** Risk of injury - Make sure that the starknobs ① are tightened.
- ⚠ **WARNING:** Risk of injury - When equipped with the pushbar, the pushbar shall always be mounted (→ to increase the stability of the backrest).

The mounting of the pushbar ② is done as follows:

1. Mount the pushbar ② with the starknob ① in the left handgrip.
2. Swing the pushbar ② to the right.
3. Mount hook ③ over the right handgrip ④ as shown on the drawing. Use holes ① or ② to adjust the tension of the backrest.

To take off the pushbar:

1. Loosen the starknob ① on the right handgrip.
2. Remove hook ③ from the right handgrip ④.

2.3 *Transport in the car* (add to manual D100 paragraph 2.14)

⚠ DANGER: Risk of injury – The wheelchair is not suited for use as a seat in a motor vehicle.

Never use your wheelchair as a seat in an automobile or other vehicle. The wheelchair shall be marked with following symbol.

2.4 *Use of the wheelchair as seat in a motor vehicle* (remove paragraph 2.15)

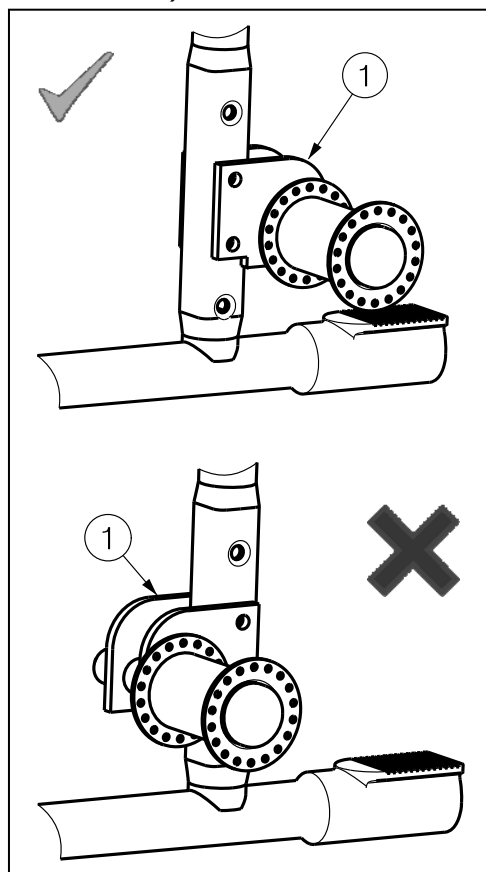
3 Installation and adjustment (refer to manual D100 paragraph 3)

3.1 *Manner of delivery* (add to manual D100 paragraph 3.2)

The Vermeiren D100 30° shall be delivered with:

- 1 frame with armrests, rear and front wheels (standard delivery seat height: 500 mm, seat angle: 5°)
- 1 pair footrests
- Back adjusting mechanism
- Tools
- Manual
- Accessories

3.2 *Adjusting the stability and the maneuverability* (add to manual D100)

Place axle block ① in most backwards position.

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All rights reserved, including translation.
Vermeiren Suisse S.A.