VERMEIREN

9042

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EN

Instructions for specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

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Version: B, 2016-06

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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INSTRUCTION MANUAL

Toilet chair Model 9042

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the toiletchair, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your toiletchair. Please remember that observance of our hints would keep your toiletchair in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.
To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility.

TECHNICAL DETAILS

Total height:	900 mm
Height of seat	510 mm
Width:	580 mm
Width of seat:	460 mm
Height of backrest:	445 mm
Depth:	560 mm
Weight:	ca. 15 kg
Max. load:	100 kg

We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg

EXPLANATION OF SYMBOLS

Maximum mass

Indoor use

CE conformity

Type designation

CONTENTS OF CONSIGNMENT

Unpack your product and verify if the delivery is complete. Following items should be included:

- Toilet chair + cushioned seat
- Cushioned lid

- Toilet bucket with cover
- Instruction manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

APPLICABILITY

Mobile, re-usable toilet commode. Should not be used for taking a shower, for storage, as a ladder, or over built-in toilets.

ASSEMBLY INSTRUCTIONS

The toilet chair is delivered fully assembled. Place the chair on level surfaces only and make sure that all four legs stand solidly on the floor. The bucket can now be placed onto the guide rails below the seat. When not in use the seat can be covered by the cushioned lid supplied.

- ➔ Remove the bucket before transporting the chair.
- ➔ Remove and empty the bucket every time after use.

HINT: Pour some water in the upper edge of the bucket. The bucket is then sealed airtight against bad odours when the lid is closed.

SAFETY

- ➔ When setting up the chair and when it is transported, carefully guard against bruising and shearing - danger of injury!
- ➔ Do not grasp sideways further than the length of your arm allows, otherwise the chair can tip over.
- ➔ Before using, check whether all four legs stand evenly on the floor – risk of tipping over!
- ➔ Do not lean too far forwards -- risk of tipping over!
- ➔ Remove the bucket before transporting the chair.
- ➔ Do not exceed the max load (see technical details)
- ➔ Repairs and disinfection should only be done by authorised persons!
- ➔ Empty the bucket every time after use.
- ➔ Pay attention to the appropriate state of the tips on the legs (damage or lack of them would cause "slip" when sitting down or standing up from a chair)
- ➔ Risk of burns – Be careful when using in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching.
- ➔ Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

CARE

Cleaning shall be according following instructions:

- **Upholstery / frame**
 - Clean upholstery / frame with a cloth moistened with hot water. Be aware that you do not soak the upholstery / frame.
 - Use a mild commercial detergent for removing stubborn dirt.
 - Stains can be removed by using a sponge or a soft brush.
 - Do not use strong cleaning liquids like solvents, nor use hard brushes.
 - Never clean with steam and/or pressure washers.
 - The manufacturer is not liable for damage caused by improper cleaning agents.
- **Plastic parts**

Clean plastic parts of your toilet chair with commercial plastic cleaners. Only use a soft brush or soft sponge. Examples of plastic parts are the bucket,
- **Surface layer**

The high quality of the surface layer guarantees optimal protection against corrosion. If the outer coat is damaged by scratching or in some other way, get your specialist dealer to repair the affected surface.

When cleaning, only use warm water and normal household detergents and soft brushes and cloths. Ensure that no wetness gets into the tubes. Initially the zinc parts only require rubbing with a dry cloth. Stubborn dirt can best be removed by using a suitable commercial zinc polish.

SERVICE

Lifetime of the toiletchair is influenced by its use, storage, regular maintenance, servicing and cleaning.

The toilet chair must be checked at least every second year by authorised persons/specialist dealers, and at least every time before resuming its use. The following parameters must be checked:

- Check the frame for plastic deformation and reduced functioning.
- Visual inspection of damage to the paintwork (danger of corrosion)
- Check the condition of the cushions (tears, brittleness, soaking)
- Bucket sealing (not in the case of re-using)

- Completeness of consignment?
- Instruction manual available?
- Cleaning

The service should only be confirmed in the "Service plan" if at least the above aspects have been checked. If your specialist dealer does not undertake the services which you require, kindly contact the manufacturer. We will gladly refer you to authorised specialist dealers in your vicinity.

DISINFECTION

All parts of the chair can be disinfected by scrubbing. In principle all surfaces of a system or a product have to be disinfected before handing it over to another user (re-use).

➔ Only authorised technicians trained in the functioning and application of disinfectants should apply disinfectants.

➔ Wear suitable protective clothing to prevent skin irritations which might be caused by the disinfectant. Also note the product information of the solutions used.

➔ The manufacturer does not accept liability for damage and injury caused by improper implementation of the disinfection.

Use only disinfectants included in the list of the Robert Koch Institute and the verified procedures of disinfection by scrubbing or wiping.

The current situation in regard to the disinfectants included in the RKI list, can be obtained from the Robert Koch Institute (RKI, home page: www.rki.de).

Toilet buckets and cushioned seats are not meant to be treated hygienically. **Replacement parts can be obtained from your specialist dealer.**

SHIPPING AND STORAGE

The shipping and storage of the toiletchair shall be according following instructions:

- Store in a dry place (between +5°C and +41°C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the toilet chair

from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

WARRANTY

Excerpt from the "General Business Conditions".

(...)

5. The guarantee period for warranty claims is 24 months.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts.

DISPOSAL

When disposing of the toiletchair, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

DECLARATION OF CONFORMITY

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V

Address :

Vermeirenplein 1/15

2920 Kalmthout

Belgium

declares under his sole responsibility that the CE marked devices :

Productgroup: Commode chairs

Productgroup (GMDN): Chair, toilet (GMDN 40539)

FAMHP (FAGG) registration

number: BE/CA01/11/2-02965-18-CLI

Brand: Vermeiren

Type: 139, 139B, 139E, 139SP, 139SPAV, 9139, 155, 175, 175BIS, 177, 177BIS, 180, 180BIS, 9042, 9052, 9060, 9062, 9062XXL, 9063, Liddy, Stacy, Tina, Gloria

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,

and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :

Medical devices directive MDD 93/42/EEC: 2007

and is in conformity with the relevant European harmonized standards:

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CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multipoosition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

- This warranty is not valid in case of:
- damage due to incorrect usage of the wheelchair,
 - damage during transport,
 - involvement in an accident,
 - a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
 - normal wear of the wheelchair,
 - non-return of the warranty card

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Notes

A series of horizontal dotted lines for taking notes.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Italy

Vermeiren Italia

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.vermeiren.it
e-mail: info@vermeiren.it

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Czech Republic

Vermeiren ČR S.R.O.

Nádražní 132
702 00 Ostrava 1
Tel: +420 596 133 923
Fax: +420 596 121 976
website: www.vermeiren.cz
e-mail: info@vermeiren.cz

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

Austria

Vermeiren Austria GmbH

Schärddinger Strasse 4
A-4061 Pasching
Tel: +43(0)7229 64900
Fax: +43(0)7229 64900-90
website: www.vermeiren.at
e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

Hühnerhubelstraße 59
CH-3123 Belp
Tel: +41(0)31 818 40 95
Fax: +41(0)31 818 40 98
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5
Sant Gregori Parc Industrial Edifici A
17150 Sant Gregori (Girona)
Tel: +34 972 42 84 33
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es