# VERMEIREN

## Cadeo

USER MANUAL  
MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

EN

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*Bedienungshandbuch*

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*Manual de instrucciones*

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This page is intentionally left blank

## Content

<b>Content</b> .....	<b>1</b>
<b>1 Preface</b> .....	<b>1</b>
<b>2 Your product</b> .....	<b>1</b>
<b>3 Before use</b> .....	<b>2</b>
<b>4 Assembly and adjustments</b> .....	<b>3</b>
<b>5 Using your rollator</b> .....	<b>4</b>
<b>6 Maintenance</b> .....	<b>5</b>
<b>7 Technical specifications</b> .....	<b>6</b>

## 1 Preface

Congratulations! You are now owner of a Vermeiren rollator!

Thank you for your trust in the products of Vermeiren. To support you on the use of this rollator and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your rollator.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

### **Important note**

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

### **Information available**

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

## 2 Your product

Figure 1 Rollator components

1. Push handles
2. Knob for height adjustment
3. Frame (aluminium)
4. Connecting plates
5. Brakes
6. Wheels
7. Safety bar
8. Seat
9. Storage bag
10. Location of the identification plate

## 3 Before use

### 3.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- Rollator frame with seat, 8 wheels
- Height adjustable pushing handles
- Safety bar
- Manual.

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

### 3.2 Intended use

CAUTION

#### Risk of falling

**The rollator is meant as support of the leg function during walking. Do not use the rollator if you have other physical limitations that may cause unsafe use of your rollator, like a weak arm, balance disorders, ...**

The rollator is designed and produced solely to be used as walking aid for one person with a maximum weight of 100 kg.

The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide better stability, support and safety during walking.

The rollator can be used indoors and outdoors.

It is not intended for transport of goods and/or more people than the user.

It should only be used on surfaces where all wheels and legs are touching the ground. Do not use the rollator on rocky, bumpy or sandy terrain, or places with a steep inclination. The rollator may become unstable. Look for flat, stable areas, and bevelled kerbs to walk with your rollator.

Please check all technical details and limits of your rollator in §7.

The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused

by improper use or lack of maintenance will cause the warranty to lapse.

### 3.3 Safety instructions

CAUTION

#### Risk of injuries

**Please read and follow the instructions in this manual. Otherwise you may get injured, or your rollator may get damaged.**

- Check that no stones or other objects block the wheels of the rollator.
- While walking, keep both hands on the handgrips.
- Be aware that some parts of your rollator may get very hot or cold due to ambient temperature, solar radiation, heating devices, etc. Be careful when touching.
- Before every use, check that the handgrips are firmly secured, see §4.1.
- Do not hold the handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- Do not proceed with your rollator into traffic. Always stay on the pavement.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

### 3.4 Symbols on the rollator

Identification plate is located on the inner side of the base frame

Maximum weight of user in kg

Type designation

### 3.5 Shipping and storage

The shipping and storage of the rollator should happen according to the technical specifications in §7. Make sure that the rollator is stored dry.

Provide sufficient covering or packaging to protect the rollator from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

Make sure no objects are stored on or against the rollator to prevent damage.

## 4 Assembly and adjustments

### 4.1 Height adjustments

CAUTION

**Risk of falling**

- **Firmly fix the handgrips before using the rollator.**
- **Do not exceed the minimum or maximum adjustment as indicated on the handgrips. Your rollator will not be stable anymore.**

The pushing handles can be adjusted in height according to the following steps:

1. Loosen knob (2) for height adjustment.
2. Remove the knob completely.
3. Remove the bolt (11) from the tube.
4. Adjust the pushing handles to the desired height (6 different positions). When standing straight, the handles should be as high as your wrists. Leave your arms hanging loose.
5. Place bolt (11) back in the hole of the tube.
6. Place knob (2) on bolt (11).
7. Tighten knob (2) firmly by hand.
8. Adjust the other pushing handle with the same instructions.
9. Check that the two pushing handles are adjusted with the same height.

### 4.2 Mounting the storage bag

CAUTION

**Risk of falling**

**Do not overload the storage bag.**

The storage bag is meant to be mounted below the seat.

Mount the storage bag with the snap fasteners (12) on the two horizontal tubes where the seat rests on.

The maximum carrying weight of the storage bag can be found in the table of §7.

## 5 Using your rollator

### 5.1 Unfolding / folding

**CAUTION**
**Risk of entrapment**

Keep fingers, buckles, jewellery and clothes away from the folding mechanism.

**CAUTION**
**Risk of falling**

Check that the rollator is completely open and that the frame is locked.

#### 5.1.1 Folding the rollator

1. Pull connection plates (4) a bit upwards [A].
2. Lift the rollator by its push handles to close the frame [B].
3. Pull safety bar (7) up to fold [C].

**A**

**B**

**C**

#### 5.1.2 Unfolding the rollator

1. Turn the safety bar (7) downwards to fold it open [D].
2. Push both handles down with the brakes (5) on the floor and push the rollator forward till it is completely open [E].
3. Turn the seat down to the sitting position.
4. Adjust the pushing handles to the desired height (see details in §4.1).

**D**

**E**

### 5.2 Brake system

**CAUTION**
**Risk of falling**

Check the working of the pression brakes and the wear of the end pieces to ensure your safety.

The rollator is equipped with pression brakes. To apply the brakes, just use your weight on the pushing handles of the rollator.

If the user sits on the seat of the rollator, the pression brakes are automatically applied.

### 5.3 Sitting down, standing up and walking

1. While leaning on the handgrips, turn around and sit down on the seat.
2. To stand up again, lean on the handgrips and turn around.
3. Step forward with the rollator close to your body.

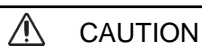
## 6 Maintenance

**i** Regular care ensures that your rollator is preserved in a perfectly functional condition. For the maintenance manual, refer to the Vermeiren website: [www.vermeiren.com](http://www.vermeiren.com).

### 6.1 Cleaning

Use a soft, damp cloth and lukewarm water with a mild soap to clean the rollator. Avoid soaking. Do not use abrasive cleaning agents to clean.

### 6.2 Disinfection

**Risk of damage**

**Disinfections may only be undertaken by trained persons. Consult your specialist dealer.**

### 6.3 Inspection / Service / Repairs

The lifespan of the rollator is influenced by its use, storage, maintenance, service and cleaning.

Before you resume use of the rollator, it should be inspected on the following points:

- Completeness.
- Folding mechanism: functioning well.
- Bolts and nuts: tightly fastened.
- Brakes: functioning well.
- Wheels (stability, tracking, fixing screws).  
In case of regular contact with wires, hair or other residues, the wheels may get blocked. Check regularly that the wheels rotate freely and that they are in good condition.
- Frame (deformation, stability, connections).
- Handgrips (safe locking of height adjustment system, good support of handgrips).
- Seat (wear or damage).

The service frequency depends on the frequency and intensity of use. Consult your specialist dealer to make an agreement on service frequency.

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

### 6.4 Reuse

Before each reuse, have the rollator disinfected, inspected and serviced according to the instructions in §6.2 and §6.3 .

### 6.5 Disposal at end of life

At end of life, you need to dispose your rollator according to the local environmental legislation. The best way to do so, is to disassemble the rollator to facilitate the transport of recyclable parts.

## 7 Technical specifications

The technical details below are only valid for this rollator, at standard settings and optimal ambient conditions. Take these details into account during use.

<b>Brand</b>	<b>Vermeiren</b>
<b>Type</b>	<b>Rollator</b>
<b>Model</b>	<b>Cadeo</b>

<b>Description</b>	<b>Dimensions</b>
Maximum user weight	100 kg
Overall length	640 mm
Folded length	300 mm
Overall width	625 mm
Folded width	625 mm
Overall height	810 – 895 mm
Folded height	942 – 1040 mm
Distance between handgrips	420 mm
Seat height	540 mm
Seat width	350 mm
Seat depth	300 mm
Total mass	4,60 kg
- without basket	4,40 kg
Wheel diameter	110 mm
Maximum load basket	5 kg
Storage temperature	+5°C - +41°C
Storage humidity	30% - 70%
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.	

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:
By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:
By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:
By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:
By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:

Vermeiren GROUP NV  
Vermeirenplein 1 / 15  
2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)

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### All rights reserved, including translation.