# VERMEIREN

## Nathalie

INSTRUCTION MANUAL  
MODE D'EMPLOI  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
ISTRUZIONI PER L'USO  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI

EN

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<b>EN Contents</b>	
1.	Technical details ..... 2
2.	Components ..... 2
3.	Check after delivery ..... 2
4.	Intended use ..... 2
5.	Assembly / Dismantling ..... 2
5.1.	Assembly pedal exerciser ..... 2
5.2.	Dismantling pedal exerciser ..... 2
6.	Exercises of the legs ..... 3
7.	Resistance adjustment ..... 3
8.	Safety instructions ..... 3
9.	Maintenance, Inspection, Cleaning ..... 3

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### Pedal Exerciser Nathalie

#### 1. Technical details

**⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.**

<b>Make</b>	<b>Vermeiren</b>
<b>Address</b>	<b>Vermeirenplein 1/15, B-2920 Kalmthout</b>
<b>Type</b>	<b>Pedal exerciser</b>
<b>Model</b>	<b>Nathalie</b>
Width	400 mm (15.75 in)
Length	530 mm (20.87 in)
Height	260 mm (10.24 in)
Dismantled height	240 mm (9.45 in)
Weight	1,95 kg (2.20 lbs)
Chargeable weight	max. <b>50 kg (110.23 lbs)</b>
Storage temperature	+5°C - +41°C (41°F - +105.8°F)
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg (0.59 in / 3.3 lbs)	

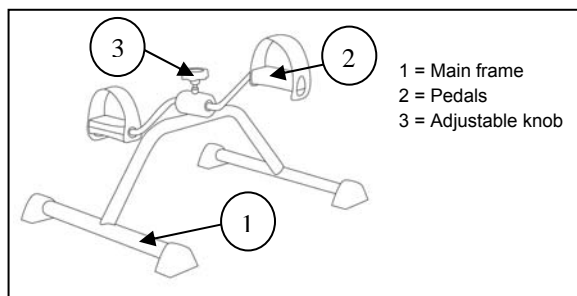
First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the pedal exerciser, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your pedal exerciser. Please remember that observance of our hints would keep your pedal exerciser in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer. To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

#### 2. Components

#### 3. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- pedal exerciser frame with 2 pedals
- manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

#### 4. Intended use

The pedal exerciser shall be used by 1 person.

The pedal exerciser shall be used indoor.

The construction of the pedal exerciser allows full use by elderly people or during revalidation in home environment. The pedal exerciser can increase the flexibility by exercising the range of motion of the joints of the lower body (exercise of the legs).

The pedal exerciser shall not be used as toy for children.

The pedal exerciser should only be used on horizontal, dry and non smooth surfaces.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

#### 5. Assembly/Dismantling

**⚠ CAUTION: Chance of pinching - Do not place fingers between the components of the pedal exerciser.**

**⚠ CAUTION: Risk of clamping - Keep fingers, cloths, jewelry or items away from moving parts of the pedal exerciser.**

##### 5.1. Assembly pedal exerciser

- Mount the adjustable knob on the pedal exerciser.

##### 5.2. Dismantling pedal exerciser

- Remove the adjustable knob from the pedal exerciser

## 6. Exercises of the legs

**⚠ CAUTION: Risk of injury - Sit always in a chair when performing the leg exercise. Do not stand on the pedal exerciser.**

**⚠ CAUTION: Risk of injury - Make sure the chair does not move or rotate. If the chair have wheels, be sure the brakes are engaged.**

**⚠ CAUTION: Risk of injury - Be sure you wear good shoes that not slip off the pedals. (Rubber soled shoes)**

**⚠ CAUTION: Risk of burns – Do not take the pedal exerciser to the pedals tube after training, this could be warm caused of the friction of moving parts.**

- Place pedal exerciser on a horizontal, non smooth surface directly in front of the chair.
- Sit comfortable in a non movable chair.
- Place your feet in the pedals.
- Now you can pedal with the pedal exerciser.

## 7. Resistance adjustment

Turn on the adjustable knob to adjust the resistance of pedaling.

Turn clockwise: Resistance increase -- heavier pedaling.

Turn counterclockwise: Resistance decrease - - looser pedaling.

## 8. Safety instructions

- Consult your specialist (doctor) before using he pedal exerciser.
- Stop the exercises and consult your specialist immediately if you feel

dizziness, severe muscle/joint pain or chest pain.

- Warm-up and stretch about 5 minutes before beginning your exercises.
- Exercises shall be in a controlled manner and begin slowly.
- Do not exhaust you.
- **Do not exceed the chargeable weight. The chargeable weight for your pedal exerciser is 50 kg (110.23 lbs).**
- Use the pedal exerciser on horizontal, dry, non-smooth and indoor surfaces.
- Keep a minimum safety area clearance of 1 meter of the pedal exerciser. Be sure these area remains clear during use of the pedal exerciser.
- Keep it away from children. The pedal exerciser shall not be used as a toy.
- When it is used by, on or near children, invalids or disabled persons close supervision is necessary.
- Risk of burns – Be careful when using in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching.
- Do not place fingers between components of the pedal exerciser.
- Keep fingers, cloths, jewelry or items away from moving parts of the pedal exerciser.
- Wear comfortable and suitable clothing when using the pedal exerciser.
- Be sure you wear good shoes that not slip off the pedals.
- Make sure the chair where you sits in does not move or rotate.
- Do not stand on the pedal exerciser.
- Do not use the pedal exerciser when it is not working properly.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

## 9. Maintenance, inspection, cleaning

For the maintenance manual of the pedal exerciser refer to the Vermeiren website: [www.vermeiren.com](http://www.vermeiren.com).

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## ENGLISH

### CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multipoosition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

### APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

### EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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---

## Notes

A series of horizontal dotted lines for taking notes.

---

## Notes

A series of horizontal dotted lines for taking notes.

## Belgium

### N.V. Vermeiren N.V.

Vermeirenplein 1 / 15  
B-2920 Kalmthout  
Fax: +32(0)3 666 48 94  
website: [www.vermeiren.be](http://www.vermeiren.be)  
e-mail: [info@vermeiren.be](mailto:info@vermeiren.be)

## France

### Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin  
F-59710 Avelin  
Tel: +33(0)3 28 55 07 98  
Fax: +33(0)3 20 90 28 89  
website: [www.vermeiren.fr](http://www.vermeiren.fr)  
e-mail: [info@vermeiren.fr](mailto:info@vermeiren.fr)

## Italy

### Vermeiren Italia

Viale delle Industrie 5  
I-20020 Arese MI  
Tel: +39 02 99 77 07  
Fax: +39 02 93 58 56 17  
website: [www.vermeiren.it](http://www.vermeiren.it)  
e-mail: [info@vermeiren.it](mailto:info@vermeiren.it)

## Poland

### Vermeiren Polska Sp. z o.o

ul. Łączna 1  
PL-55-100 Trzebnica  
Tel: +48(0)71 387 42 00  
Fax: +48(0)71 387 05 74  
website: [www.vermeiren.pl](http://www.vermeiren.pl)  
e-mail: [info@vermeiren.pl](mailto:info@vermeiren.pl)

## Czech Republic

### Vermeiren ČR S.R.O.

Nádražní 132  
702 00 Ostrava 1  
Tel: +420 596 133 923  
Fax: +420 596 121 976  
website: [www.vermeiren.cz](http://www.vermeiren.cz)  
e-mail: [info@vermeiren.cz](mailto:info@vermeiren.cz)

## Germany

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Wahlerstraße 12 a  
D-40472 Düsseldorf  
Tel: +49(0)211 94 27 90  
Fax: +49(0)211 65 36 00  
website: [www.vermeiren.de](http://www.vermeiren.de)  
e-mail: [info@vermeiren.de](mailto:info@vermeiren.de)

## Austria

### Vermeiren Austria GmbH

Schärddinger Strasse 4  
A-4061 Pasching  
Tel: +43(0)7229 64900  
Fax: +43(0)7229 64900-90  
website: [www.vermeiren.at](http://www.vermeiren.at)  
e-mail: [info@vermeiren.at](mailto:info@vermeiren.at)

## Switzerland

### Vermeiren Suisse S.A.

Hühnerhubelstraße 59  
CH-3123 Belp  
Tel: +41(0)31 818 40 95  
Fax: +41(0)31 818 40 98  
website: [www.vermeiren.ch](http://www.vermeiren.ch)  
e-mail: [info@vermeiren.ch](mailto:info@vermeiren.ch)

## Spain / Portugal

### Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5  
Sant Gregori Parc Industrial Edifici A  
17150 Sant Gregori (Girona)  
Tel: +34 972 42 84 33  
Fax: +34 972 40 50 54  
website: [www.vermeiren.es](http://www.vermeiren.es)  
e-mail: [info@vermeiren.es](mailto:info@vermeiren.es)

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### **All rights reserved, including translation.**