VERMEIREN

Liftchair

INSTRUCTION MANUAL - MODE D'EMPLOI - GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG - ISTRUZIONI PER L'USO - MANUAL DE INSTRUCCIONES

EN

Instructions for specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

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Versión: E, 2016-08

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the lift chair, read the instruction manual carefully: it will familiarise you with this product.

If you have any further questions, please consult your specialist dealer.

TECHNICAL DETAILS

Vancouver

Total width :	730 mm
Power :	220/230 V
Seat width :	450 mm
Height armrests :	140 mm
Height seat :	500 mm
Total height:	1090 mm
Length :	920/1720 mm
Seat depth :	500 mm
Height back :	650 mm
Weight :	ca. 56 kg
Weight user :	150 kg

Montreal

Total width :	790 mm
Power :	220/230 V
Seat width :	450 mm
Height armrests :	180 mm
Height seat :	500 mm
Total height:	1090 mm
Length :	920/1720 mm
Seat depth :	530 mm
Height back :	650 mm
Weight :	ca. 58 kg
Weight user :	150 kg

Ontario 1 and 2 motors

Total width :	810 mm
Power :	220/230 V
Seat width :	450 mm
Height armrests :	180 mm
Height seat :	460 mm
Total height:	1120 mm
Length :	920/1720 mm
Seat depth :	500 mm
Height back :	750 mm
Weight :	1 motor: ca. 52 kg
	2 motors: ca. 52,5 kg
Weight user :	150 kg

Deviations in sizes and weight (1,5 mm en 1,5 kg) are possible

INSTALLATION / ASSEMBLY

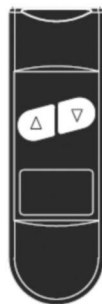
The liftchair is delivered fully assembled with exception of the back, which is easy to mount on the liftchair.

USAGE

The four wheels of the liftchair will touch the ground when the liftchair is in the seating position (liftchair can be moved). The two front wheels will be blocked when using the lift-function (liftchair cannot be moved).

Usage of the buttons:

• 1 motor (legrest and backrest moves together)

LIFT FUNCTIONS

Stepless adjustment from sit-position to lift-position.

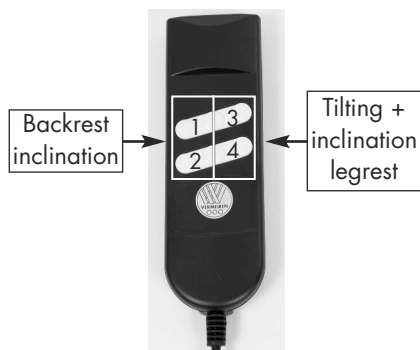
LAY BACK FUNCTION

Stepless adjustment to stretched position

→ When the liftchair is not being used, leave it the sitting position and not in the elevated position.

• 2 motors (legrest and backrest moves separate)

1 = Backrest inclination forwards

2 = Backrest inclination backwards

3 = Lift function (stepless adjustment from sit-position to lift-position: seat upwards + legrest downwards)

4 = Lay function (stepless adjustment to stretched position: seat downwards + legrest upwards)

SAFETY

→ Please do not exceed the maximum users weight of 150 kg.

→ Be extremely careful in the presence of child-

ren or pets.

→ Do not allow children to be under the chair, use the chair or play with it because it might cause injuries when the legrest are folding.

→ If the wheelchair blocks in the lying position, you must ask assistance to leave the chair.

→ Please do not place the liftchair near any heat source or type of electric radiator, fireplace, radiator or radiator with flammable liquid because the coatings are not fire-resistant. Never smoke when seated in the liftchair.

→ It is strongly recommended not to operate the liftchair near a desk or a wall too close to it, as not to cause damage by tilting the back or the legrests.

→ If you need to retrieve a fallen object underneath the seat, disconnect the front to avoid getting stuck.

→ For prolonged use of the liftchair motor (about 2 min), you'll have to wait a few minutes before reusing it. This is needed to avoid overheating the liftchair motor.

→ Do not move the liftchair:

- when the liftchair is in the lying position;
- when the liftchair is connected to the power and/or in service.

→ Be careful not to sit on the armrests or on the legrest.

→ Ensure that no cables become squeezed.

MAINTENANCE

→ Inspect your liftchair regularly to detect any damage on the coating, framework or loose.

CLEANING THE REMOTE CONTROL

Before starting, it's important to unplug the power cord. The remote can be cleaned with a dry cloth but do not use any aggressive cleaning products.

Do not use your liftchair if you notice any defects but contact your local dealer.

CLEANING THE COATING

You can clean the artificial leather parts with warm water. In case of stubborn dirt, you can wash it with a non-abrasive cleaning product but do not use solvents or hard brushes. You can remove stains with a sponge or soft brush. We strongly advise you to first test the cleaning product on a hidden part of your liftchair. Always avoid wet surfaces, especially the fabric. Never wash your liftchair with plenty of water. Do not wet the electrical parts.

WARRANTY

Defects must be reported in writing within 2 weeks. Material defects or construction errors will be under our responsibility starting from the date of the invoice over a period of two years.

Not included in the warranty:

- Damage due to changes in the structure of our product;
- Wear;
- Damage caused by the user;
- Damage caused by the usage of non-original parts.

The manufacturer takes no liability for damages due to non compliance of these instructions.

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Vermeiren Suisse S.A.