VERMEIREN

139E

USER MANUAL
MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

EN

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Content

Content	1
1 Preface	1
2 Your product	1
3 Before use	1
4 Assembly	2
5 Usage	2
6 Maintenance	2
7 Technical details	3

1 Preface

First of all we wish to thank you for the trust you placed in us by selecting a VERMEIREN product.

Before using the toilet chair, read the instruction manual carefully: it will familiarise you with this product.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

Information available

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

2 Your product

- | | |
|------------------|-------------------------|
| 1. Backrest | 6. Footrest |
| 2. Armrest | 7. Footplate |
| 3. Arm pad | 8. Castor with brake |
| 4. Seat | 9. Castor without brake |
| 5. Toilet bucket | 10. Frame |

3 Before use

3.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- Frame (incl. commode seat, seat cushion, armrests, 4 steering castors, 2 fixing screws for backrest)
- Backrest
- Footrests with footplates
- Toilet bucket with cover
- Instruction manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

3.2 Intended use

Mobile, re-usable toilet commode. Should not be used for taking a shower, for storage, as a ladder, or a wheelchair.

The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

3.3 Safety instructions

- Do not use as a wheelchair.
- Do not lift the chair by using the armrests or footrests.
- Do not stand on the footrests, the toilet chair could tip over – risk of injury.
- When setting up the chair and when it is transported, carefully guard against bruising and shearing – danger of injury.
- Do not grasp sideways further than the length of your arm allows – danger of injury.
- Do not lean too far forwards -- danger of tipping over.
- Remove the bucket before transporting the chair.
- Repairs and disinfection should only be done by authorised persons.
- Empty the bucket every time after use.

4 Assembly

4.1 Frame

Position the base frame in front of you facing forwards. Hold the backrest on the side at the armrest screws (in doing this, the armrests should hang down) and guide the back tubing into the tube openings provided on the base frame. Turn the locking screws until they are tight.

The locking screws must be released, otherwise it may not be possible to insert the back tubing properly.

Check that the back tubes are not jammed or skewed and that they are pushed in fully.

4.2 Armrests

~~You can now~~ Place the armrests in the receivers provided on the base-frame, by rotating the armrests on their screw fixings, ~~in a curve to the front from behind the backrest.~~

4.3 Footrests

The footrest receivers are located on the front frame tubes of the toilet chair.

1. Fold the footplates open.
2. Place the footrests from the side into the two guide bolts and swing inwards until they lock in place. If this does not happen immediately, try again while gently

pressing the retaining mechanism (inner side of footrest hanger).

3. Using the Velcro® fastening, the calf strap can be fitted at the desired position.

4.4 Seat cushion

To place the seat cushion in the correct position, fit the guide pin between the commode seat and its hand grip (at the back).

4.5 Toilet bucket

The toilet bucket can be pushed below the seat from the back, with or without its cover.

5 Usage

5.1 Brakes

⚠ Do not operate the brakes with your hands – risk of injury.

Before getting onto or off the toilet chair, the parking brakes must be applied to the rear castors. The brakes are applied by pushing down the silver brake lever with your foot.

5.2 Toilet bucket

- Remove the bucket before transporting the chair.
- Remove and empty the bucket every time after use.
- Pour some water in the upper edge of the bucket. The bucket is then sealed airtight against bad odours when the lid is closed.

6 Maintenance

6.1 Cleaning

The seat cushions, arm cushions, frame and bucket can be wiped using domestic cleaning agents and a damp cloth. Do not use a brush or any other type of scourer.

Chrome parts should be wiped periodically with a dry cloth. Use a mild chrome polish on stubborn dirt.

Treatment with hot jets and high pressure equipment is not permitted.

6.2 Disinfection

All parts of the chair can be disinfected by scrubbing. In principle all surfaces of a system or a product have to be disinfected before handing it over to another user (re-use).

Only authorised technicians trained in the functioning and application of disinfectants should apply disinfectants.

Toilet buckets and cushioned seats are not meant to be treated hygienically.

6.3 Inspection/Service/Repairs

The service frequency depends on the frequency and intensity of use. Consult your specialist dealer to make an agreement on service frequency.

The following parameters must be checked:

- Check the frame, armrests, footrests and castors for plastic deformation and reduced functioning.
- Visual inspection of damage to the paintwork (danger of corrosion)
- Check the condition of the cushions (tears, brittleness, soaking)
- Bucket sealing (not in the case of re-using)
- Completeness of consignment ?
- Instruction manual available?

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

Replacement parts can be obtained from your specialist dealer.

7 Technical details

Brand	Vermeiren
Type	Toilet chair
Model	139E

<i>Description</i>	<i>Dimensions</i>
Max. load	120 kg
Total height	950 mm
Seat height	492 mm (without seat cushion) 528 mm (with seat cushion)
Overall width	560 mm
Seat width	420 mm
Backrest height	375 mm
Overall length	820 mm
Seat depth	420 mm
Dismantled length	550 mm
Dismantled height	590 mm
Armrest to seat distance	226 mm
Front location of armrest structure	370 mm
Weight	ca. 13 kg
Dismantled weight	Frame: 7,75 kg Backrest: 2,5 kg Footrests: 1,65 kg (Pair) Seat cushion: 1,05 kg
Castors	125 mm (2 with brakes, 2 without brakes)
Volume toilet bucket	5 l
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.	

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Service registration form

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

Vermeiren GROUP NV
Vermeirenplein 1 / 15
2920 Kalmthout
BE

website: www.vermeiren.com

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All rights reserved, including translation.