# VERMEIREN

## Eden Eden-E

USER MANUAL  
MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

EN

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*Bedienungshandbuch*

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*Manual de instrucciones*

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This page is intentionally left blank

# Contents

<b>Contents</b> .....	<b>1</b>
<b>Preface</b> .....	<b>2</b>
<b>1 This product</b> .....	<b>3</b>
<b>2 Before use</b> .....	<b>4</b>
2.1 Intended use .....	4
2.2 General safety instructions.....	4
2.3 Symbols on the chair.....	4
2.4 Transport and storage .....	5
<b>3 Using the chair</b> .....	<b>6</b>
3.1 Brakes.....	6
3.2 Using the safety harness.....	6
3.3 Moving to/from the chair.....	7
3.4 Comfort adjustment (depending on version) .....	7
3.5 Mounting or removing the headrest.....	8
3.6 Mounting or removing the tray .....	9
<b>4 Maintenance</b> .....	<b>10</b>
4.1 Points of maintenance.....	10
4.2 Instructions for maintenance .....	10
4.3 Troubleshooting .....	11
4.4 End of use .....	11
<b>5 Technical specifications</b> .....	<b>12</b>

## Preface

Thank you for your trust in the products of Vermeiren. To support you on the use of this shell chair and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your chair.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

### **Important note**

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

### **Information available**

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

This instruction manual  
For user and specialist dealer

Installation instructions  
For specialist dealer

Service manual  
For specialist dealer

EC declaration of conformity

# 1 This product

*Eden-E*

*Eden*

- 1. Shell chair
- 2. Headrest
- 3. Backrest
- 4. Safety harness
- 5. Tray
- 6. Castors with brakes

- 7. Lower frame
- 8. Leg support
- 9. Seat cushion
- 10. Handset (Eden-E)
- 11. Push bar (Eden)
- 12. Identification plate

## 2 Before use

### 2.1 Intended use

- i** In this paragraph a brief description of the intended use of your shell chair is given. Additionally, relevant warnings are added to the instructions in the other paragraphs. In this way we would like to make you aware of the possible misuse that may appear.
- This shell chair is suitable for indoor use only, on flat, level grounds.
  - This chair is designed and produced solely as seat and resting place for one (1) person with a maximum weight of 120kg. It is not designed for transport of goods or objects, nor for any other use as previously described.
  - This chair is meant to be operated by an attendant or care taker.
  - If applicable, only use accessories and spare parts approved by Vermeiren.
  - Please read all technical details and limits of your chair in §5.
  - The warranty of this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

### 2.2 General safety instructions

 CAUTION Risk of injuries and/or damage

Keep the following general warnings in mind during use:

- Be aware that some parts of this chair may get very hot or cold due to ambient temperature, solar radiation, heating devices. Be careful when touching. Use protective clothing if the weather is cold.
- Negotiate thresholds or obstacles carefully.
- Be careful when using possible causes of fire such as cigarettes since they may set the seat and back covers alight.
- Do not modify this shell chair in any way.

### 2.3 Symbols on the chair

The symbols in following list are applicable to your chair. Symbols can be found in the relevant ISO standard (ISO 7000, ISO 7001 and IEC 417).

Maximum weight of user in kg

Indoor use only

Not intended to be used as seat in a motor vehicle

Maximum safe slope in ° (degrees).

Type designation

The shell chair complies to the requirements set up in:

ISO 7176-8: Requirements and test methods for static, impact and fatigue strengths.

EN 1021-1, EN1021-2: Resistance to ignition of upholstered parts

## 2.4 Transport and storage

 CAUTION Risk of injuries and damage

- Pay attention when passing with the chair through narrow passages.
- Make sure the passage is clear from objects, obstacles and persons.
- Never use ramps with a too large slope angle, see technical specifications in §5.
- Never use the shell chair to transport a patient.

### 2.4.1 Moving aside

1. Return the shell chair to sitting position, see §3.4.
2. Release the brakes and move the shell chair by using its wheels. Roll it to its destination, see §3.1.

**i** You can use a ramp to push the chair to higher or lower ground level.

If this is not possible, ask help to lift the chair with two persons and take it to its destination.

1. Firmly grasp the frame and shell with both hands and lift it from the floor. Do not use leg rest(s) or wheels to grasp the chair.

### 2.4.2 Transport by vehicle

 WARNING Risk of injuries

- Do NOT use your shell chair as seat in a vehicle, see the following symbol.
- Never use the same safety belt for passenger and shell chair.

1. Place the chair in the vehicle according to the instructions in §2.4.1.
2. Store the shell chair and its parts in the luggage compartment. If luggage and passenger compartments are NOT separated or enclosed, fasten the shell chair frame securely to the vehicle. You can use the available safety belt in the vehicle.
3. Apply the brakes after loading, see §3.1.

### 2.4.3 Storage

CAUTION Risk of damage

Make sure that your shell chair is stored dry to prevent mould from growing and the upholstery from being damaged, see also technical details in §5.

### 3 Using the chair

**WARNING**

Risk of injuries

- First read the previous chapters and inform yourself about the intended use. Do NOT use your chair unless you have read and fully understood all instructions.
- In case of doubts or questions, do not hesitate to contact your local specialist dealer, your care provider or technical adviser.

**CAUTION**

Risk of injuries and/or damage

- Make sure that your fingers, clothes, belts, buckles or jewellery won't get caught by moving parts, during assembly, use or adjustment.

#### 3.1 Brakes


**CAUTION**

Risk of injuries and/or damage

- Do not use the brakes to slow the shell chair down.

Each of the swivelling wheels is equipped with a parking brake that blocks rotation and rolling of the wheel. When the shell chair is in parking position, all four brakes must be applied.

- Releasing brakes [A]: Use your foot to press the brake lever upwards.
- Applying brakes [B]: Use your foot to press the brake lever down towards the wheel.

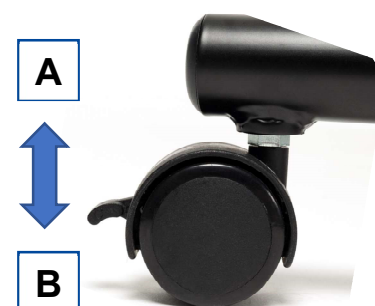
#### 3.2 Using the safety harness

**CAUTION**


Risk of injuries

- Adjust the safety harness so that the patient cannot be strangled and so that he can sit comfortably in the chair.
- Adjust the safety harness as close as possible to the body of the patient so that the patient cannot slide out the safety harness. Use the safety harness especially for patients who make extreme movements.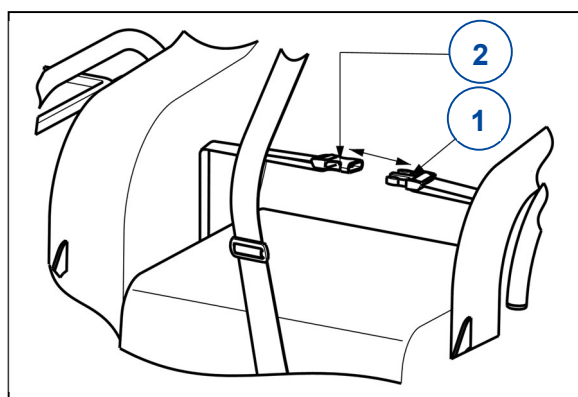


1. Mount the waist belt around the waist of the patient. Click the buckle of belt ① into buckle of belt ② to close.
2. If needed, adjust the waist belt.

3. Mount the safety harness around the body of the patient.
4. Cross the harness around the back of the shell chair.
5. Click the buckle of belt ③ into buckle of belt ④ to close.
6. If needed, adjust the safety harness.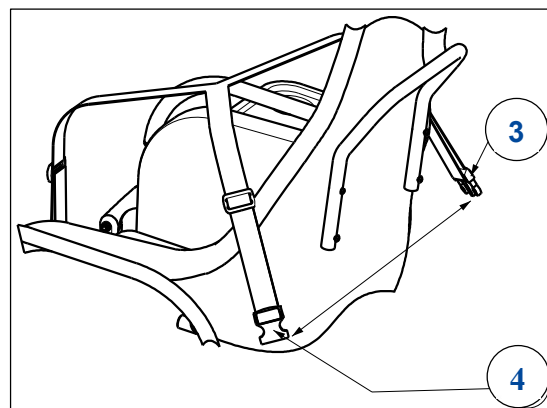

The adjustment of the belt shall be done using the plastic adjustment system ① mounted on the belt.

1. Pull the belt through the adjustment system ① to lengthen or shorten.

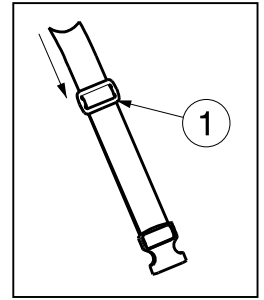### 3.3 Moving to/from the chair

CAUTION

Risk of injuries or damage

- In case you cannot perform the transfer in a safe manner, ask someone to assist you.
- Make sure that the person in the chair cannot slip out of the body/hip belt. Adjust both belts and tilt the chair backwards to prevent this.
- The parking brakes should be applied before getting into and out of the shell chair.

#### 3.3.1 Transfer

1. Make sure that the shell chair is in upright sitting position.
2. Position the shell chair as close as possible to the person sitting in his/her chair, couch, seat or bed.
3. Make sure that the chair stands on a flat and level ground.
4. Turn the front swivelling wheels outwards, to the front.
5. Apply the brakes, see §3.1.
6. Open the buckle of the body belt and place it aside. Do the same with the hip belt.
7. Transfer the person to/from the chair with the help of attendant(s) or lifting equipment.

#### 3.3.2 Sitting in the chair

1. Sit down on the seat with the lower back against the backrest.
2. Make sure that the upper legs are horizontal and that the feet are in comfortable position. If necessary, adjust the seat height (see installation manual).
3. Close the hip belt and body belt and adjust it as close as possible to the body.

### 3.4 Comfort adjustment (depending on version)

CAUTION

Risk of injuries or damage

- The following comfort adjustments can be done by the attendant or caretaker. All other adjustments are done by your specialist dealer according to the installation manual.
- Be sure that the shell chair does not tip over backwards when it is placed in its most backwards position.
- Always apply the brakes before performing adjustments.

### 3.4.1 Manual seat angle / shell inclination adjustment (Eden)

- i** The tilt lever (1) holds the gas spring of the tilt mechanism in position. As soon as the tilt lever is pulled, the gas spring is released.

*To change the seat inclination:*

1. Hold the push handle (2) tight and pull the tilt lever (1) with your finger in a squeezing motion.
2. Push down on the push handle to tilt the seat.
3. Release the tilt lever to block the position.

*To return to sitting position:*

1. Hold the push handle tight and pull the tilt lever (1) with your finger in a squeezing motion.
2. Slowly move the push handle up. Release the tilt lever to block the position.

### 3.4.2 Electrical seat angle / shell inclination adjustment (Eden-E)

To change the inclination of the seat:

1. Press and hold the button on the handset.
  - The left button [A] to incline the seat backwards.
  - The right button [B] to incline the seat forwards.
2. Letting go of the button at any time will stop the seat in any position.
3. With the movement of the seat, the leg rest is raised accordingly.

### 3.5 Mounting or removing the headrest


1. Make sure that the large edge of the headrest attachment is placed to the bottom.
2. Using the elastic band, mount the headrest to the desired position on the shell chair top.
3. Using the elastic band, remove the headrest from the shell chair.

### 3.6 Mounting or removing the tray


 CAUTION Risk of injuries or damage

- Do not overload the tray. The maximum allowed load is 1 kg.

Mounting the tray: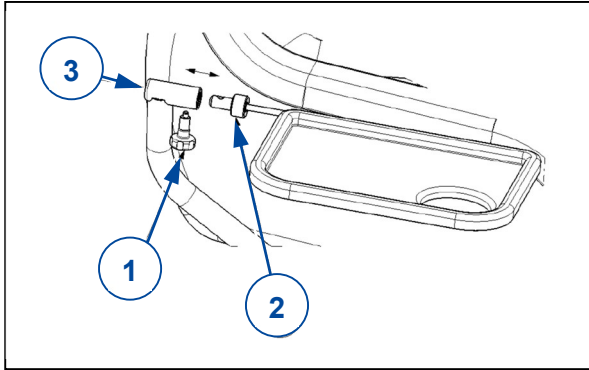

1. Loosen screw ①.
2. Mount the pen of the tray support ② to the bushing ③.
3. Retighten screw ① firmly by hand.

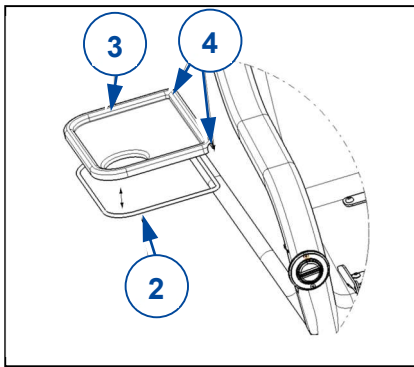4. Mount the tray ③ to the tray support ② by clicking it into place. Place the two holes ④ closest to the chair.

To remove the tray, repeat the steps in reverse order.

Folding the tray:

1. Loosen screw ①.
2. Pull down the handle of the screw ①.
3. Fold the tray.

## 4 Maintenance

- i** Regular care ensures that your shell chair is preserved in a perfectly functional condition. For the maintenance manual, refer to the Vermeiren website: [www.vermeiren.com](http://www.vermeiren.com).

### 4.1 Points of maintenance

 CAUTION Risk of injuries and damage

Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.

- i** The last page of this manual contains a registration form for the specialist dealer to record each service.

The service frequency depends on the frequency and intensity of use. Contact your dealer to agree to a common timetable for inspection/maintenance/repair.

#### *Before each use*

Inspect the following points:

- All parts: Present and undamaged or unworn.
- All parts: Clean, see § 4.2.1.
- Wheels, shell, body belt, hip belt: Well secured.
- Working of tilt lever (gas springs), brakes, buckles of belts.
- Wheels/tyres; keep them free of wires, hair, sand, fibres and other contamination (oil, mud).
- Condition of frame parts: No deformation, instability, weakness or loose connections
- Shell-chair upholstery, head cushion, seat cushion, calf-rest cushion: No excessive wear (like dented spots, damage or tears).

Contact your specialist dealer for possible repairs or part replacements.

### 4.2 Instructions for maintenance

#### 4.2.1 Cleaning

 CAUTION Risk of damage

- Never use a steam cleaner to clean the chair.

Wipe all rigid parts of the chair frame with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

The upholstery can be cleaned with lukewarm water and a mild soap. Do not use abrasive cleaning agents, sponge, or hard brush to clean.

#### 4.2.2 Disinfection

 CAUTION Risk of damage

Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

### 4.3 Troubleshooting

 **WARNING** Risk of injuries and damage to the chair

- NEVER attempt to repair your chair yourself.

Even if you use your shell chair properly, a technical problem may occur. The following symptoms may indicate a serious problem. Therefore, always contact your specialist dealer if you spot any of following deviations:

- Strange sound;
- Uneven tread wear at one of the tyres;
- Jerky movements;
- Chair deflects to one side;
- Damaged or broken wheels;
- Damaged or broken tilting mechanism or footrest mechanism.

### 4.4 End of use

At end of life, you need to dispose your chair according to the local environmental legislation. The best way to do so, is to disassemble the chair to facilitate the transport of recyclable parts.

## 5 Technical specifications

The technical details below are only valid for this chair, at standard settings and optimal ambient conditions. Take these details into account during use. The values are no longer applicable if your chair has been modified, damaged, or is severely worn.

<b>Brand</b>	<b>Vermeiren</b>
<b>Type</b>	Shell chair
<b>Model</b>	<b>Eden</b>

<i>Description</i>	<i>Min.</i>		<i>Max.</i>	
Maximum occupant mass	120 kg			
Overall length	710 mm upright, 1330 mm reclined			
Overall width	940 mm, 770 mm with folded table			
Total mass	Approx. 34 kg			
Mass of heaviest part	Lower frame : 15,5 kg			
Obstacle climbing	Not applicable			
Seat plane angle	-3°		+30°	
Effective seat depth	460 mm			
Effective seat width	380 mm	440 mm	480 mm	
Seat surface height at front edge	400 mm		430 mm	
Backrest angle	-3°		+30°	
Backrest height	830 mm			
Armrests to seat distance	230 mm			
Maximum load of tray	1 kg			
Overall height	1350 mm			
Thickness seat cushions	80 mm			
Storage and use temperature	+ 5 °C		+ 41 °C	
Storage and use humidity	30%		70%	
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / 1,5°.				

Table 1: Technical specifications Eden

<b>Brand</b>	<b>Vermeiren</b>
<b>Type</b>	Shell chair
<b>Model</b>	<b>Eden-E</b>

<b>Description</b>	<b>Min.</b>	<b>Max.</b>
Maximum occupant mass	120 kg	
Overall length	710 mm upright, 1330 mm reclined	
Overall width	940 mm, 770 mm with folded table	
Total mass	Approx. 36 kg	
Mass of heaviest part	Lower frame : 15,5 kg	
Obstacle climbing	Not applicable	
Seat plane angle	-3°	+30°
Effective seat depth	460 mm	
Effective seat width	380 mm	440 mm
Seat surface height at front edge	400 mm	430 mm
Backrest angle	-3°	+30°
Backrest height	830 mm	
Armrests to seat distance	230 mm	
Maximum load of tray	1 kg	
Overall height	1350 mm	
Thickness seat cushions	80 mm	
Operator control	IP20, Uin: 24V ---	
Power supply / Transformer	Input: 100-240 V~, 50/60 Hz, max. 1,5A, Output: 24V --- Protection class: IP20 Insulation class: II Duty cycle: 2min ON / 18min OFF	
Storage and use temperature	+ 5 °C	+ 41 °C
Storage and use humidity	30%	70%
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / 1,5°.		

Table 2: Technical specifications Eden-E

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:

Vermeiren GROUP NV  
Vermeirenplein 1 / 15  
2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)

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### All rights reserved, including translation.