VERMEIREN

Inovys II-F

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES

EN

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Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: A, 2016-05

All rights reserved, including translation.

No part of this manual may be reproduced in any form whatsoever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© N.V. Vermeiren N.V. 2016

Contents

Preface	2
1 Product description	3
1.1 Intended Use.....	3
1.2 Technical specifications	4
1.3 Drawing.....	5
1.4 Accessories.....	5
1.5 Explanation of symbols.....	5
1.6 Safety rules.....	6
2 Use	7
2.1 Carrying the wheelchair	7
2.2 Mounting the rear wheels (only for quick release 22", 24" rear wheels).....	7
2.3 Mounting or removing of the footrests	7
2.4 Angle of footrests	8
2.5 Fold away leg rest	8
2.6 Operating the brakes.....	9
2.7 Mounting or removing of armrests.....	9
2.8 Pushing handgrips	10
2.9 Mounting or removing backrest	10
2.10 Transfer in and out the wheelchair	11
2.11 Correct position in the wheelchair	11
2.12 Riding the wheelchair (only for 22" and 24" rear wheels).....	11
2.13 Riding the wheelchair by an attendant	11
2.14 Moving on slopes	12
2.15 Negotiating steps or curbs	12
2.16 Taking off the wheels (only for quick release 22", 24" rear wheels).....	13
2.17 Backrest adjusting / Seat inclination	14
2.18 Transport in the car.....	14
3 Installation and adjustment	15
3.1 Tools	15
3.2 Manner of delivery	15
3.3 Mounting backrest frame	16
3.4 Adjusting the seat height and seat angle	16
3.5 Position wheels.....	17
3.6 Adjust the seat depth	18
3.7 Adjust the backrest	19
3.8 Adjust the brakes.....	19
3.9 Adjusting of the footrests	20
3.10 Adjusting the armrests.....	22
3.11 Adjusting of the tip cap	23
4 Maintenance	23

Preface

First of all we want to thank you for putting your trust in us by selecting one of our wheelchairs.

The Vermeiren wheelchairs are the result of many years of research and experience. During the development, special attention was given to the ease of use and the serviceability of the wheelchair.

The expected lifetime of your wheelchair is strongly influenced by the care and maintenance of the wheelchair.

This manual will help you get acquainted with the operation of your wheelchair.

Following the user instructions and the maintenance instructions are an essential part of the warranty.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

1 Product description

1.1 *Intended Use*

The wheelchair is intended for people with walking difficulties or no walking abilities.

The wheelchair is designed to transport 1 person.

The wheelchair is suited for indoor and limited outdoor use.

The user can propel the wheelchair by himself (only for 22", 24" rear wheels) or have the wheelchair pushed by an attendant.

The different types of fittings and accessories, and the modular construction allow full use by persons disabled by:

- paralysis
- loss of limbs (leg amputation)
- limb defects or deformations
- stiff or damaged joints
- heart insufficiencies and poor blood circulation
- balance disturbances
- cachexia (decrease in muscle)
- and also for aged persons.

When providing for individual requirements:

- body size and weight (max. 135 kg)
- physical and psychological condition
- residential circumstances
- environment

should be taken into consideration.

Your wheelchair should only be used on surfaces where all four wheels are touching the ground and where there is sufficient contact to propel the wheels equally.

Ride only over flat surfaces with firm ground. It should not be used to drive on ground such as gravel, mud, cobble stones, grassy surfaces, or other ground that is not firm or level.

The wheelchair should not be used as a ladder, nor is it a transport for heavy or hot objects.

When used on mats, carpeted floors or loose floor coverings, the floor covering can get damaged.

Use only Vermeiren approved accessories.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

Compliance with the user and maintenance instructions are an essential part of the guarantee conditions.

Visually impaired people can contact the dealer for instructions for use.

1.2 Technical specifications

Technical terms below are valid for the wheelchair in standard settings, as delivered, without a separately mounted seating system. The values can differ when a seating system is mounted. If other footrests / armrests or other accessories are used, the tabulated values will change.

Make	Vermeiren			
Address	Vermeirenplein 1/15, B-2920 Kalmthout			
Type	Manual wheelchair			
Model	Inovys II-F			
Maximum safe load	135 kg (user weight + weight seat frame)			
Description	Dimensions			
Effective seat width	400 mm	450 mm	500 mm	
Overall width (depends on the seat width)	630 mm	680 mm	730 mm	
Dismantled width (without armrest)	600 mm	650 mm	700 mm	
Description	Dimensions Min.		Dimensions Max.	
Overall length with footrests (depends on the adjusted seat depth)	1050 mm		1090 mm	
Dismantled length without footrests (depends on the adjusted seat depth)	830 mm		870 mm	
Dismantled height (Backrest folded, without armrest)	710 mm		780 mm	
Total mass	± 32 kg (24" rear wheels)		± 32,40 kg (16" rear wheels, drum brakes)	
Mass of heaviest part	± 19,5 kg (24" rear wheels)		± 19,9 kg (16" rear wheels, drum brakes)	
Masses of parts that can be dismantled or removed	Footrests: 4,90 kg; Armrests: 3,45 kg; Rear wheels 24": 4,15 kg			
Static stability downhill	10°			
Static stability uphill	15°			
Static stability sideways	12°			
Obstacle climbing	60 mm			
Seat plane angle	0°		+21°	
Effective seat depth	460 mm		520 mm	
Seat surface height at front edge	390 mm		460 mm	
Backrest angle	0°		+54°	
Backrest height	Depends on manufacturer shell / backrest			
Distance between footrest and seat frame	305 mm		485 mm (Do not use maximum distance with the footrest in vertical position -- touch the ground)	
Angle between footplate and leg footrest	80°		100°	
Angle between seat and leg footrest	104°		194°	
Distance between armrest and seat frame	300 mm		400 mm	
Front location of armrest structure	460 mm		490 mm	
Hand rim diameter (only for 24" rear wheels)	535 mm			
Horizontal location of axle (deflection)	-8 mm (16" rear wheels) -4 mm (24" rear wheels)		41 mm (16" rear wheels) 42 mm (24" rear wheels)	
Minimum turning radius	1520 mm		1620 mm	
Diameter PU Rear wheels	16" (T30) + drum brakes	22"	24"	
Diameter PU steering wheels	200 mm			

Storage and use temperature	+ 5 °C	+ 41 °C
Storage and use humidity	30%	70%
We reserve the right to introduce technical changes. Measurement tolerance $\pm 15 \text{ mm} / 1,5 \text{ kg} / ^\circ$		


Table 1: Technical specifications Inovys II-F

The wheelchair complies to the requirements set up in:

ISO 7176-8: Requirements and test methods for static, impact and fatigue strengths.

To remain compliant to EN12183, make sure that the mounted seating system meets the necessary requirements of the MDD 93/42/EEC.

1.3 Drawing

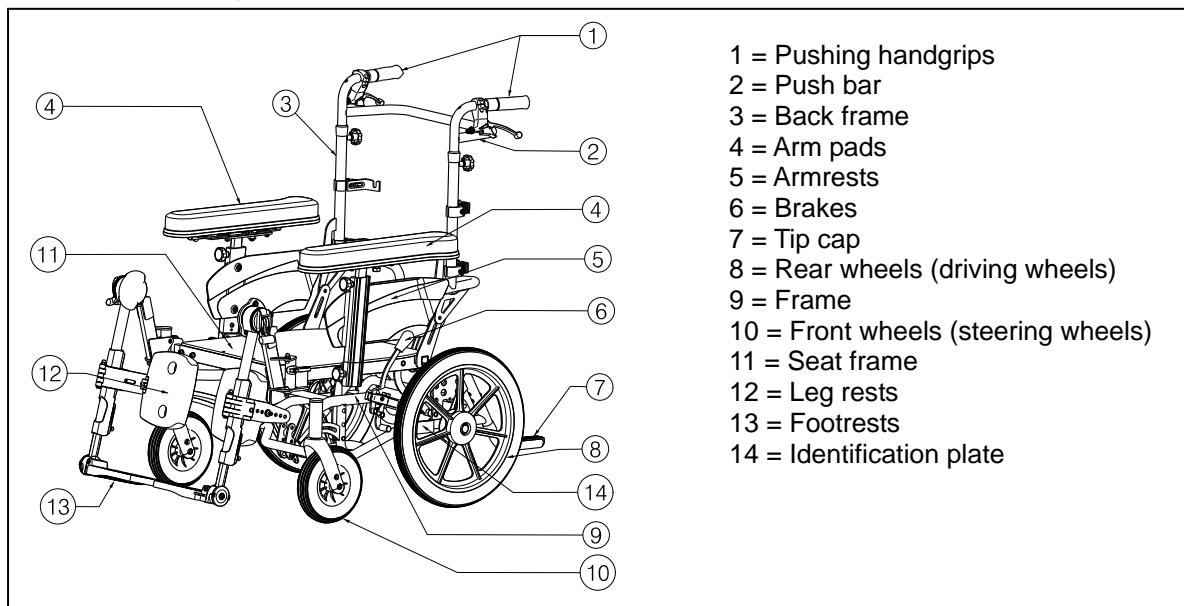1.4 Accessories

The following accessories are available for the Inovys II-F:

- Wooden table (B12)
- Hemiplegic armrests (B66)

Contact your specialist dealer about other accessories. He will gladly advise you.

1.5 Explanation of symbols

Maximum mass

Indoor and outdoor use

Maximum Safe Slope

CE conformity

Type designation

Not intended to be used as a seat in a Motor vehicle

1.6 Safety rules

- ⚠ To prevent injury and/or damage to your wheelchair, make sure that no objects and/or body parts are caught in the spokes of the driving wheels (for 22", 24" rear wheels).
- ⚠ The parking brakes should be applied before getting into and out of the wheelchair.
- ⚠ When getting into and out of the wheelchair, do not stand on the footplates. These should be folded up beforehand, or the footrests should be swung completely outwards out of the way.
- ⚠ Investigate the effects of shifting the centre of gravity on the behavior of the wheelchair, for example on up or down gradients, on laterally sloping ground, or when overcoming obstacles. Obtain support from an attendant.
- ⚠ If you want to pick up something (lying in front of, on the side, or to the rear of the wheelchair), you should not lean too far out to avoid tipping over.
- ⚠ When moving through doors, arches, etc. ensure that there is enough room at the sides so that you do not get your hands or arms caught or crushed and that there is no damage to the wheelchair.
- ⚠ Only use your wheelchair according to regulations. For example, avoid uncontrolled rolling against obstacles (steps, curbs, doorframes, etc) or dropping down from ledges. The manufacturer cannot assume liability for damage caused by overloading, collision or other improper use.
- ⚠ Stairs may only be negotiated when aided by another person. If furnishings like drive-on, ramps or lifts are available, use them.
- ⚠ When moving on the public highway, you are subject to the rules of the road.
- ⚠ When driving your wheelchair, you should not be under the influence of alcohol or medicine as in the case of driving other vehicles. This also applies to indoor driving.
- ⚠ When travelling outdoors, adapt your driving to weather and traffic conditions.
- ⚠ When transporting the wheelchair, never pick it up by grasping removable parts (armrests, footrests, etc.)
- ⚠ To be better visible when driving in the dark, wear the brightest possible clothing or clothes with reflectors, and check that the reflectors mounted on the sides and rear of the wheelchair are clearly visible.
- ⚠ Be careful when using possible causes of fire such as cigarettes since they may set the seat and back covers alight (in the case the customer use flammability upholstery).
- ⚠ Never exceed the maximum load of 135 kg.
- ⚠ Take care during all adjusting functions that no objects or body parts come into the area of the adjustment as there is the risk of squashing.
- ⚠ Check that the profile depth of the tyres is adequate.
- ⚠ To avoid hand injury, do not push your hand between the driving wheel and the knee lever brake when propelling your wheelchair (only for 22", 24" rear wheels with hand rim).
- ⚠ When driving outdoors, wear gloves which improve your grip on the tyre grips and protect your fingers against dirt and heating (only for 22", 24" rear wheels with hand rim).
- ⚠ Take care when attaching the quick-assembly wheels that the axle is completely and properly engaged (only for 22", 24" rear wheels with hand rim).

2 Use

This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**


The wheelchair is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer on how to set up the wheelchair are given in § 3.

2.1 Carrying the wheelchair

The best way to carry the wheelchair is to make use of the wheels and roll the wheelchair.

If this is not possible (e.g. when the rear wheels are taken off for transportation in a car), firmly grasp the frame on the front and the back. Do not use the foot- or armrests or the wheels to grasp the wheelchair. At least two persons are required for transporting the wheelchair.

2.2 Mounting the rear wheels (only for quick release 22", 24" rear wheels)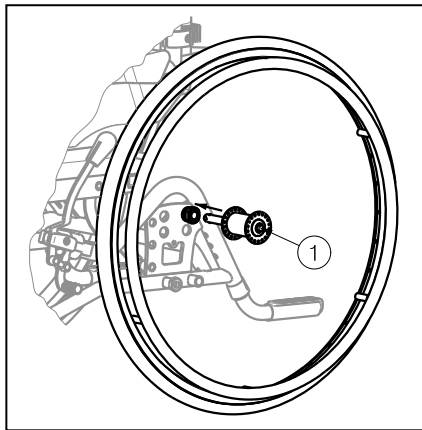


1. Take the rear wheel and push on button ①.
2. Keep the button pushed in and mount the rear wheels axle till it stops.
3. Release the button ①.
4. Check that the wheel is secured.

2.3 Mounting or removing of the footrests

⚠ WARNING: Risk of injury - Make sure that the footrests are firmly locked before using the footrests.

The wheelchair has two footrests that can be fold away to the "outside" with a 3 point attachment system (①, ②, ③ on the figure below). This system provides a steady, play-free connection of the footrest to the seat frame.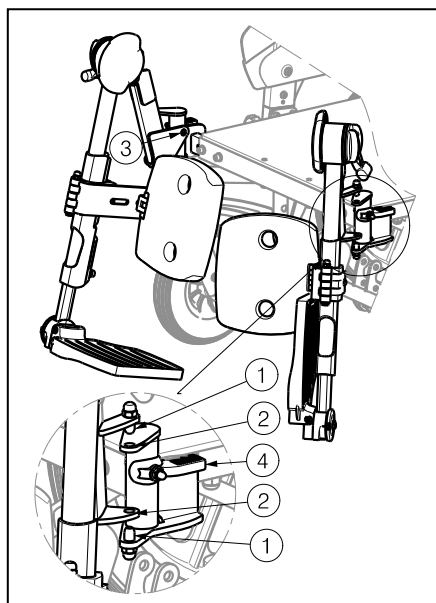

The mounting of the footrests is done as follows:


1. Hold the footrest sideways at the outside of the wheelchairs frame and mount the pins ① into the holes ②.
2. Turn the footrest inwards till the pin click in the hole of the attachment point ③.
3. Turn the footplates downwards.

To take off the footrests:

1. Fold the footplates upwards.
2. Pull or push on lever ④.
3. Turn the footrest to the outside of the wheelchair until the hole comes loose of the pin from attachment point ③.
4. Pull the footrest upwards until the pins ① come out the holes ②.

2.4 Angle of footrests


- ⚠ **CAUTION:** Risk of injury – Check that the lever ① is well fixated after adjusting the angle of the footrest.
- ⚠ **CAUTION:** Risk of clamping – Keep fingers, buckles and clothes away from the adjusting mechanism. To adjust the angle of the footrest, only touch them at the level of the footplates-to avoid injury on your fingers.
- ⚠ **CAUTION:** Risk of injury – Make sure that no objects and/or persons are inside the swinging range of the adjust functions.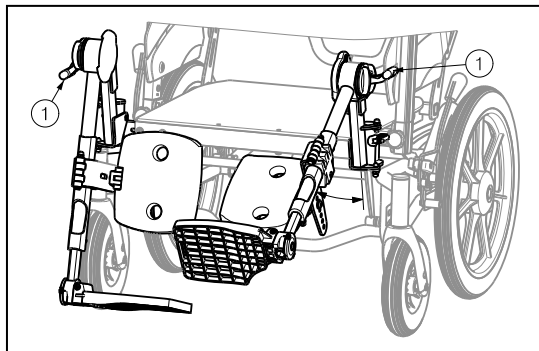

The angle or inclination of the footrest can be easily adjusted as follow:

1. Pull the lever ① on the side of the footrest upwards.
2. Adjust the angle (range 104° - 194°: step less) of the footrest to a comfortable position. Grasp the footrest at the level of the footplates.
3. Retighten the lever ① firmly by hand.

2.5 Fold away leg rest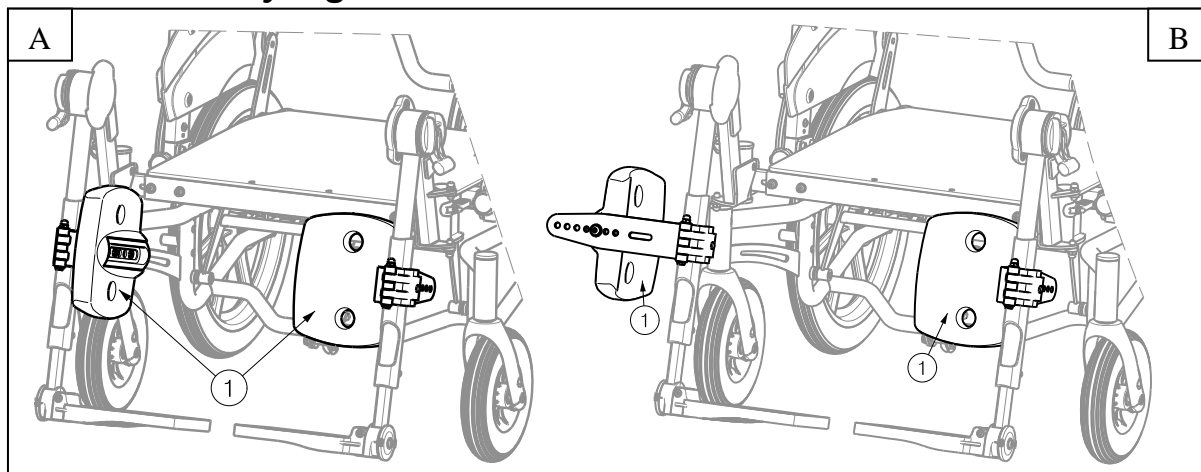


The leg rest can be easily fold away as follow:

1. Take the leg rest ① on the outside (Figure A).
2. Fold the leg rest ① to the front (Figure A).
3. Take the leg rest ① on the back (Figure B).
4. Swing the leg rest ① forwards (Figure B).

To put the leg rests in the begin position you can do the reverse. **Check you do not place your fingers between the movable parts or were they can be squeezed.**

2.6 Operating the brakes

- ⚠ **WARNING:** The brakes are not used to slow down the wheelchair during movements – Use the brake only to prevent the wheelchair from unintended movements.
- ⚠ **WARNING:** Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...) – Check the condition of the tires before each use.
- ⚠ **WARNING:** The brakes are adjustable and can wear – Check the operation of the brakes before each use.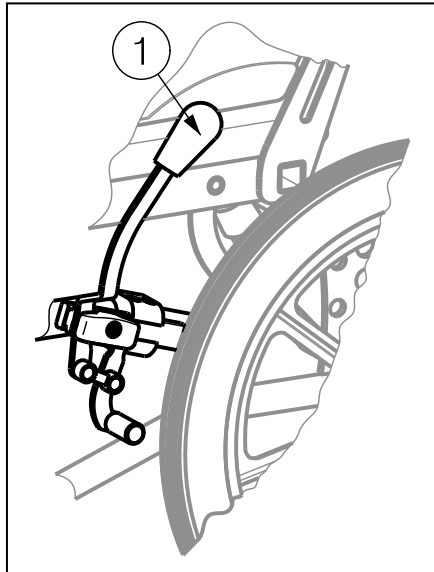

To apply the brakes:

1. Push the brake levers ① forward till you feel a distinctive click.


- ⚠ **CAUTION:** Risk of unintended movement – Make sure the wheelchair is on a flat horizontal surface before releasing the brakes. Never release both brakes simultaneously.

To release the brakes:

1. Release one brake by pulling the lever ① backwards.
2. Hold the hand rim (if available) of the release wheel with your hand.
3. Release the second brake by pulling the lever ① backwards.

2.7 Mounting or removing of armrests

- ⚠ **CAUTION:** Risk of clamping – Keep fingers, buckles and clothes away from the bottom side of the armrest.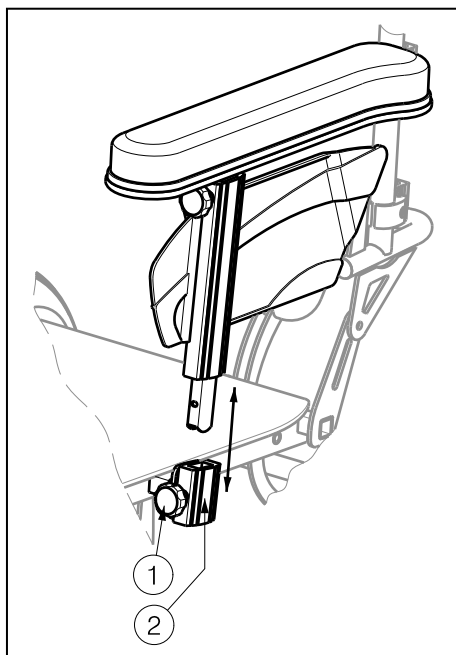


The armrests of the wheelchair can be mounted by following instructions.

1. Pull out the star knob ①.
2. Mount the armrest from above into the provided bracket ②.
3. Let go of the star knob ①.
4. Turn the star knob ① clockwise until it is finger-tight.
5. Check that the armrest is firmly secured.

The armrests of the wheelchair can be removed by following instructions.

1. Turn the star knob ① anti-clockwise.
2. Pull out the star knob ①.
3. Remove the armrest out the bracket ②.
4. Let go of the star knob ①.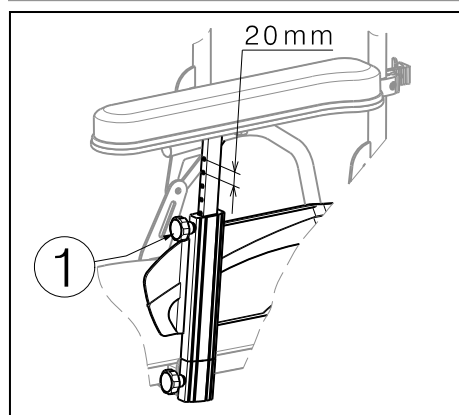


The arm pad of the wheelchair can be adjusted in 6 different heights (steps from 20 mm):

1. Turn the star knob ① anti-clockwise.
2. Pull out the star knob ①.
3. Pull the arm pad into the required position.
4. Let go of the star knob ①.
5. Turn the star knob ① clockwise until it is finger-tight.
6. Check that the arm pad is firmly secured.

2.8 Pushing handgrips

⚠ CAUTION: Risk of injury - Make sure that the star knobs ① are properly tightened.

⚠ CAUTION: Risk of tipping over - Use only the last indentation of the pushing handgrip tube for the maximum height adjustment.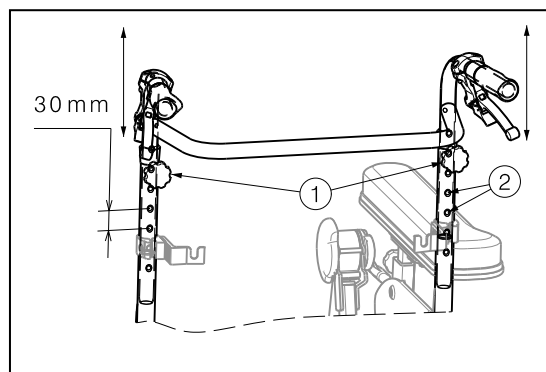


The pushing handgrips can be adjusted in height depending on the size of the attendant.

1. Loosen the star knob ① located on the back of the handgrip tubes.
2. Pull the pushing handgrips into the desired position (7 positions). Indentations ② (steps 30 mm) have been placed into the tubing of the pushing handgrips.
3. Retighten the star knob ① firmly by hand.

2.9 Mounting or removing backrest

⚠ WARNING: Risk of injury – Check that the backrest is firmly mounted into the four brackets before using your wheelchair.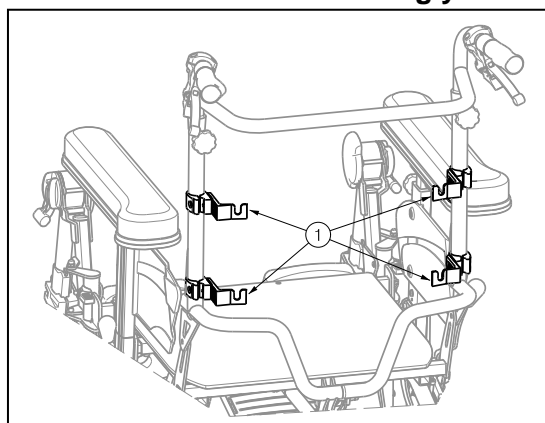


The backrest of the wheelchair can be mounted by following instructions.

1. Mount the backrest into the four brackets ①.
2. Check that the backrest is firmly fixated.

Depending on the backrest that is chosen by the customer, above assembly instructions may deviate.

2.10 Transfer in and out the wheelchair

- ⚠ **CAUTION:** In case you cannot perform the transfer in a safe manner, ask someone to assist you.
- ⚠ **CAUTION:** Risk of tipping over of the wheelchair – Do not stand on the footplates.

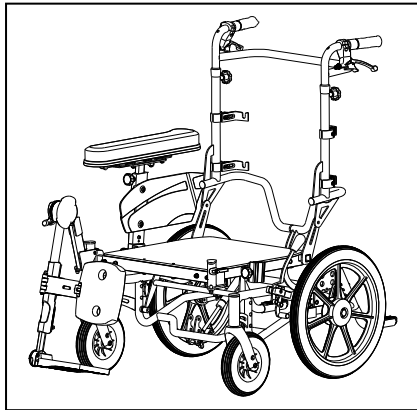1. Place the backrest in the most vertical position.
2. Position the wheelchair as close as possible to the chair, couch or bed to/from you wish to transfer.
3. Check both brakes from the wheelchair are in the on position.
4. Fold the footplates upwards to prevent standing on them.
5. If the transfer is on the side of the wheelchair, you can remove the armrest (see § 2.7) and/or footrest (see § 2.3) on that side.
6. Transfer to/from the wheelchair.

2.11 Correct position in the wheelchair

Some recommendations for a comfortable use of the wheelchair:

- Position your backside as close as possible to the backrest.
- Make sure your upper legs are horizontal – If needed adjust the length of the footrests. (see § 3.8.1)

2.12 Riding the wheelchair (only for 22" and 24" rear wheels)

- ⚠ **WARNING:** Risk of clamping – Prevent your fingers from being caught by the wheels spokes.
- ⚠ **WARNING:** Risk of clamping – Be careful passing through restricted passages (e.g. doors).
- ⚠ **WARNING:** Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.

1. Release the brakes.
2. Take both hand rims at their highest position.
3. Lean forward and push the hand rims forward until straight arms.
4. Swing your arms loosely back to the hand rims topside and repeat the movement.

2.13 Riding the wheelchair by an attendant

- ⚠ **WARNING:** Risk of clamping – Be careful passing through restricted passages (e.g. doors).
- ⚠ **WARNING:** Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a long amount of time. Touching surfaces which can assume the environment temperatures, can cause burns.

The wheelchair can be pushed with the pushing handgrips or push bar by an attendant.

2.14 Moving on slopes

- ⚠ **WARNING:** Control your speed – Moving on slopes as slow as possible.
- ⚠ **WARNING:** Consider the capacities of your attendant – If your attendant does not have enough force to control the wheelchair, put on the brakes.
- ⚠ **WARNING:** Risk of tipping over – Lean forward to move your centre of gravity forward. To improve a better stability.
 1. If available on the wheelchair, wear the safety belt.
 2. Do not attempt moving on too high slopes. The maximum slope angles (upwards and downwards) are mentioned in table 1.
 3. Ask an attendant to help you moving on the slope.
 4. Lean forward to move your centre of gravity forward.

2.15 Negotiating steps or curbs

- ⚠ **WARNING:** Risk of injury – Transporting a patient up or down steps is to be done with at least two attendants.
- ⚠ **WARNING:** Risk of injury – Wear suitable safety belt systems while being transported (information obtainable from your specialist dealer).
- ⚠ **WARNING:** Risk of injury – Sit quietly in your wheelchair and avoid jerky movements. If you cannot sit quietly when being transported because of illness or disability (e.g. slightly spastic, etc.), then at least two persons are required to secure the wheelchair on both sides by taking hold of fixed parts of the frame.
- ⚠ **WARNING:** Risk of injury – Make sure that your arms and legs do not protrude outside the wheelchair while being transported.
- ⚠ **WARNING:** Risk of injury or damages – No persons or objects should be below the wheelchair while being transported.
- ⚠ **WARNING:** Risk of injury – When the footrests have been removed, the legs of the patient are not secured.

2.15.1 Getting down steps or curbs

Riding down steps or curbs can be done moving forwards with an attendant:

1. Make sure that the pushing handgrips are sufficiently tight.
2. Remove the footrests.
3. Ask the attendant to tip the wheelchair slightly backwards with the tip cap.
4. Get past the curbs while moving on the rear wheels and push the wheelchair forwards.
5. The second person secure the wheelchair in front by taking hold of the front of the frame after removing the footrests. He must walk backwards.
6. Put the wheelchair back on the four wheels after passing the curbs.

2.15.2 Moving up steps or curbs

Moving up steps or curbs can be done moving backwards with an attendant:

1. Make sure that the pushing handgrips are sufficiently tight.
2. Remove the footrests.
3. Turn the wheelchair to have the rear wheels facing the curb.
4. Lean backwards and move your centre of gravity above the rear wheels.
5. Ask the attendant to pull the wheelchair on the curb.
6. The second person secure the wheelchair in front by taking hold of the front of the frame after removing the footrests. He must walk forwards.
7. Take back the normal position in the wheelchair.

2.15.3 Taking of stairs

- ⚠ **WARNING:** Risk of tipping over – Taking of stairs shall always be done with 2 attendants.
- ⚠ **WARNING:** Risk of injury – Lifting the wheelchair while the patient is seated in the wheelchair is forbidden! Always keep the rear wheels on the ground/stairs.
- ⚠ **WARNING:** Risk of injury – Wear suitable safety belt systems while being transported (information obtainable from your specialist dealer).
- ⚠ **WARNING:** Risk of injury or damages – No persons or objects should be below the wheelchair while being transported.
- ⚠ **WARNING:** Risk of injury – When the footrests have been removed, the legs of the patient are not secured.

Taking the wheelchair of stairs shall be according following rules:

1. Remove the footrests.
2. One attendant tip the wheelchair slightly backwards.
3. The second attendant take the front of the frame.
4. Stay calm, avoid sudden movements and keep your arms inside the wheelchair.
5. Take the steps on the rear wheels of the wheelchair.
6. Mounting the footrests back after taking the stair.

2.15.4 Using ramps

- ⚠ **WARNING:** Risk of injury – Do not exceed the maximum load or the ramps.
- ⚠ **WARNING:** Risk of injury - Choose the correct ramps so there cannot be caused any injury or damage. We do not accept any liability for injury or damage caused by an improper choice of ramps.
- ⚠ **WARNING:** Risk of injury - Be sure that the wheel height is high enough to get over the free height of the ramps. The wheelchair frame may not touch the ramps.
- ⚠ **WARNING:** Risk of injury - Use a restraining safety belt to secure yourself in your wheelchair.
- ⚠ **WARNING:** Risk of tipping over - Put the adjustment functions (seat, back, footrests, ...) that the wheelchair has the best stability.


If you are considering the installation of ramps to overcome obstacles, kindly note the following:

Because of the large weight of the wheelchair we advise against using your own power for driving on a ramp, since it might not be possible for you to prevent the wheelchair from rolling backwards.

Never go up or down ramps without the aid of a helper.

2.16 Taking off the wheels (only for quick release 22", 24" rear wheels)

To facilitate the transport off the wheelchair the rear wheels can be taken off: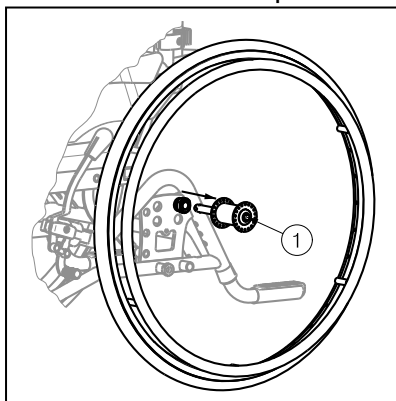


1. Make sure the brakes standing in the off position.
2. Take the wheelchair to the side frame where you want to remove the wheel.
3. Press the button ① in the center of the wheel hub.
4. Pull the wheel away from the frame.

2.17 Backrest adjusting / Seat inclination

- ⚠ **WARNING:** Risk of tipping over – Be aware that the stability decrease when the backrest is adjusted backwards.
- ⚠ **WARNING:** Risk of injury – Apply the parking brakes before adjusting the backrest and seat inclination.

The seat and the backrest can only be adjusted simultaneously or independently from one another in their angle by an attendant. Be sure that the patient sits in the chair when the attendant shall perform the back adjusting, seat inclination and that the wheelchair does not tip over.

Seat inclination:

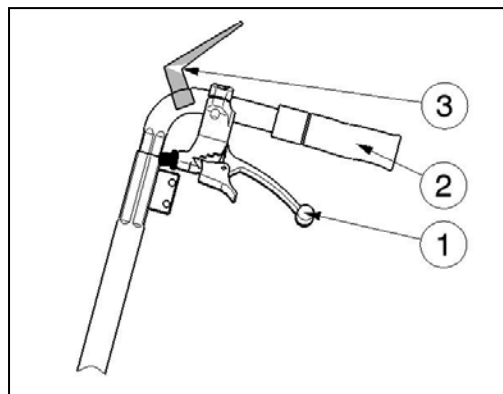1. Pull lever ① on the left side (see label) towards the handgrip ② to adjust the seat inclination.

Label

2. Pull/Push the seat in the desired position (0° to + 21°), the gas pressure spring shall be pressed in/out.
3. Loosen the lever ①, to block the gas pressure spring.

Backrest adjusting:

1. Pull lever ① on the right side (see label) towards the handgrip ② to adjust the angle of the backrest.

Label

2. Pull/Push the backrest in the desired position (0° to + 54°), the gas pressure spring shall be pressed in/out.
3. Loosen the lever ①, to block the gas pressure spring.

If your wheelchair features drum brakes, the levers ① are used as brakes; additional levers ③ have been mounted for adjusting angle of backrest and seat inclination. These levers ③ can be triggered with the thumb.

2.18 Transport in the car

- ⚠ **DANGER:** Risk of injury – The wheelchair is not suited for use as a seat in a motor vehicle.
- ⚠ **WARNING:** Risk of injury – See that the wheelchair is attached properly. So you can avoid injury from the passengers during collision or sudden braking.
- ⚠ **WARNING:** Risk of injury – Use for attaching the wheelchair and passenger **NEVER** the same seatbelt.

Never use your wheelchair as a seat in an automobile or other vehicle. The wheelchair shall be marked with following symbol.

To transport the wheelchair in the car use following steps:

1. Set the seat in a horizontal position and the backrest vertically.
2. Remove footrests, armrests and accessories.
3. Store footrests, armrests and accessories safely.
4. If possible, remove the rear wheels.
5. Place the wheelchair in the luggage place with 2 persons.
6. If the wheelchair and the passenger compartment is NOT separated, attach the frame of the wheelchair securely to the vehicle. You can use the available safety belts in the vehicle.
7. Place the patient in the recommended existing seats with their seatbelt systems.

3 Installation and adjustment

The instructions in this chapter are for the specialist dealer.

The Vermeiren multi-functional wheelchairs Inovys II-F has been designed entirely for your comfort. This wheelchair can be adjusted to individual requirements as explained below.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

⚠ WARNING: Variation of allowed adjustments can still change the stability of your wheelchair (tilt back or sideways).

3.1 Tools

To set up the wheelchair the following tools are needed.

- Wrench set n° 10 to n° 22
- Allen key set n° 3 to n° 8
- Screwdriver Phillips head
- Screwdriver
- Nipper
- Pliers

Assembly and settings may only be performed by a specialist dealer.

3.2 Manner of delivery


The Vermeiren Inovys II-F shall be delivered with:

- 1 frame with armrests, rear and front wheels
- 1 pair footrests
- Seat frame and back frame
- Tools
- Manual
- Accessories (optional)

When you receive the wheelchair, please check that the product supplied is complete and fully functional. Warranty claims are only valid if the manufacturer (supplier) is notified in writing within a period of two weeks. In the case of any defect that is not immediately evident, warranty claims are only valid if the manufacturer (supplier) is notified in writing within two weeks after it is detected. We shall remedy defects in the manner of our choosing - either by carrying out a repair or by providing a replacement.

3.3 Mounting backrest frame

- ⚠ WARNING: Risk of injury - Make sure that the ring of safety pin (A) is firmly secured.**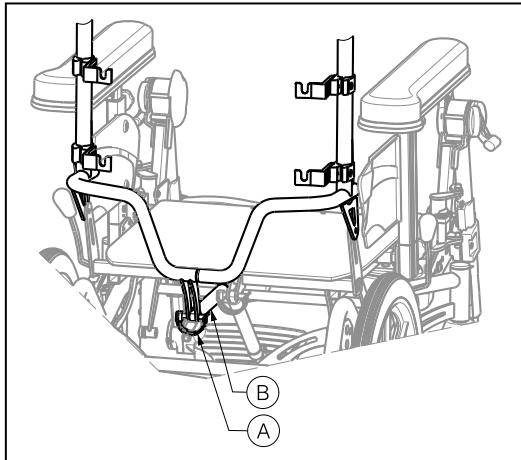

After you have taken the wheelchair out of the original packaging, push the backrest frame to the back into the 90° position. Guide the safety pins (A) into the appropriate socket to attach the spring (B), which is located at the back under the seat.

Check that no cables are squeezed.


3.4 Adjusting the seat height and seat angle

- ⚠ WARNING: Risk of injury - Make sure that all screws are properly tightened with the appropriate tool.**
- ⚠ WARNING: Risk of injury - Self-locking nuts are only to be used once or are to be secured with an additional screw locking device.**
- ⚠ WARNING: Risk of injury - Adjust the brakes if necessary after adjusting the seat height and seat angle.**
- ⚠ WARNING: Risk of injury - Check that the seat heights are mounted in the same position on both sides.**
- ⚠ WARNING: Risk of tipping over - When the driving wheels are adjusted, the position of the wheels are altered.**

The seat height of the Inovys II-F multi-position wheelchair is adjustable in 3 positions in steps of 35 mm. The seat height can be adjusted by moving the seat frame in different positions in the front tube (A) and rear wheel plate (C), and the gas spring in the attachment plate (B). These components you can find below the seat frame.

Seat height	Part (A), (B), (C)
390 mm	Hole 1
425 mm	Hole 2
460 mm	Hole 3

Table 2: Seat heights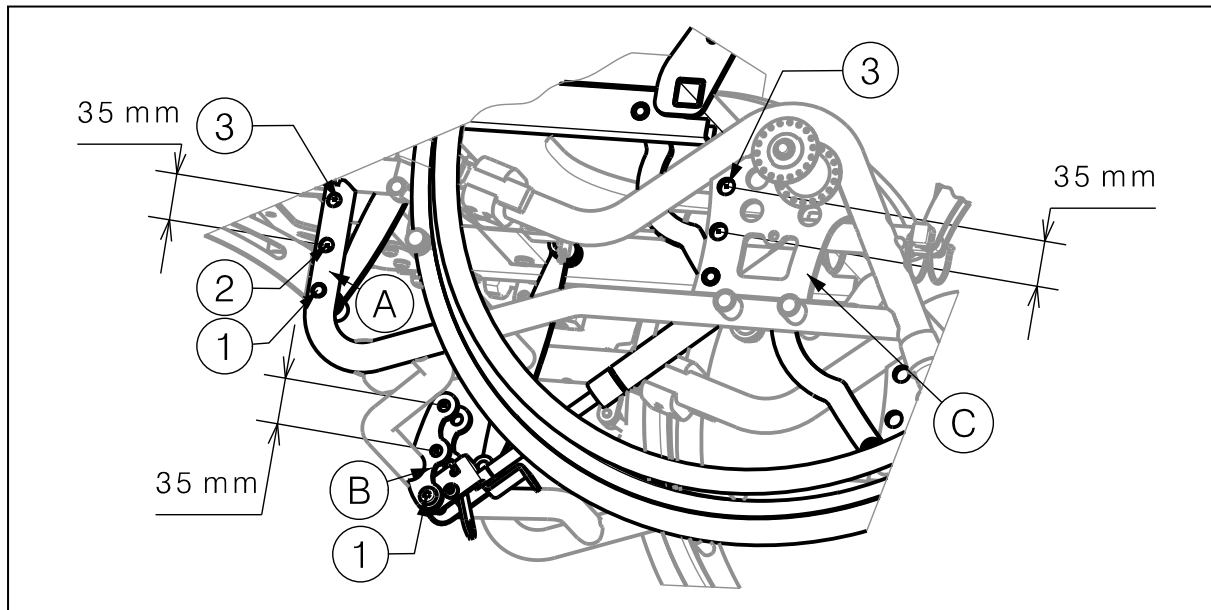

Ⓐ Front tube

The basic frame and the seat frame are screwed together with the front tube Ⓐ. In the basic frame three different sockets are provided for these screws which determine the wheelchair's seat height.

Ⓑ Attachment plate gas pressure spring


In addition to adjusting the frame, the gas pressure spring for the seat inclination can be fixed at three different attaching heights Ⓑ. So that the angle of inclination can be kept for the seat inclination, the assembly height of the gas pressure spring Ⓑ must correspond to the assembly height of the front tube Ⓐ.

Ⓒ Rear wheel plate

The rear wheel plate Ⓒ must correspond to the assembly height of the front tube Ⓐ.

3.5 Position wheels

- ⚠ **WARNING: Risk of tipping over – Be aware that you change the stability of the wheelchair when changing the position of the driving wheels.**
- ⚠ **WARNING: Risk of injury – Make sure that the rear wheels on both sides are mounted in the same position.**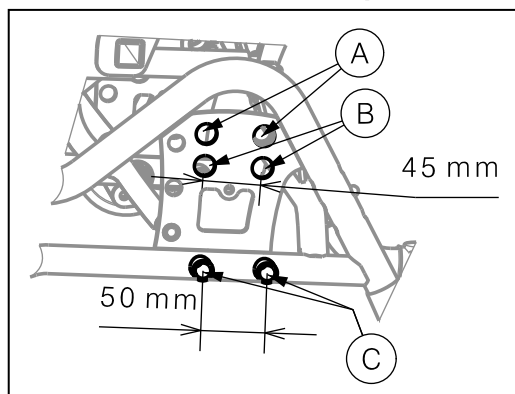

DRIVING WHEELS

Four possible wheel sockets are provided in a swage block of the frame for the use of driving wheels (24") Ⓐ, (22") Ⓑ. Depending on which suspensions are used to attach the driving wheels, the wheel position (stability or manoeuvrability of wheelchair: 45 mm). The wheel sockets attached in the delivery condition can be attached in all sockets (bore holing).

The wheel sockets in the frame tubing Ⓒ serve only to attach 16" wheels (transit version). The stability or manoeuvrability of the wheelchair can be adjusted over 50 mm.

STEERING WHEELS

In the standard model, the INOVYS II-F is delivered with two 8" steering wheels (200 mm x 50 mm or 200 mm x 35 mm).


3.6 Adjust the seat depth

⚠ WARNING: Risk of injury – Make sure that all screws are secure and properly tightened.

The seat depth can be adjusted in 3 positions (steps 20 mm). Adjust the seat depth according following rules:

Seat depth	Hole
480 mm	①
500 mm	②
520 mm	③


Table 3: Seat depth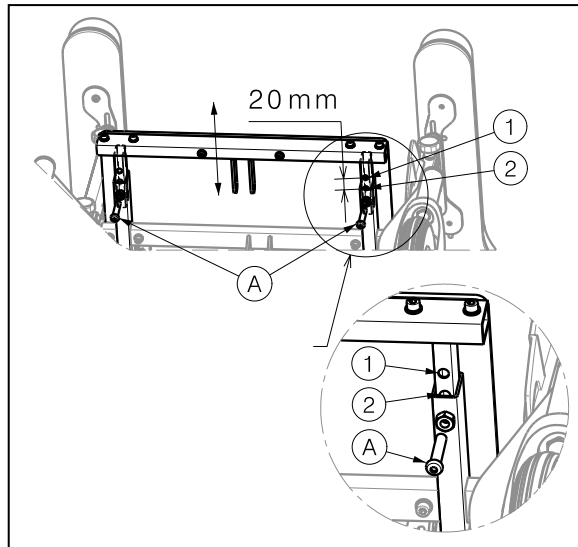

1. Loosen the screws Ⓐ beneath the front seat frame.
2. Pull or push the front of the seat frame into the desired position.
3. Retighten the screws Ⓐ firmly again.

Note: When rebuilding the seat depth in the back frame area, the displacement of the spring for adjusting the back is changed and a gap can occur between the back and seat cushions.

The seat depth can also be changed by adjusting the brackets ①. With these adjustment you can become a minimum seat depth of 460 mm.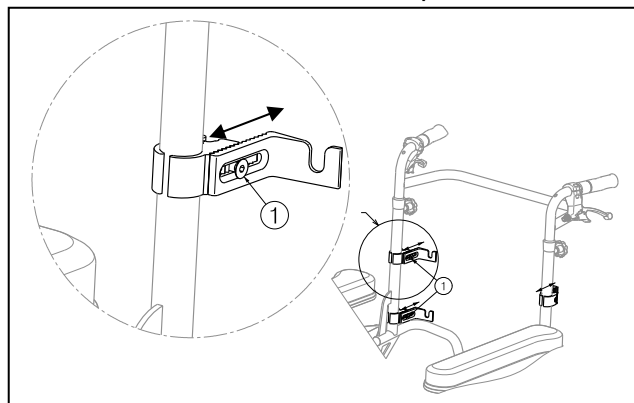


1. Remove the backrest.
2. Loosen the bolts ① of the four brackets.
3. Adjust the brackets more to the front or back until the desired seat depth.
4. Retighten the bolts ① of the four brackets.
5. Replace the backrest.

Depending on the backrest that is chosen by the customer, above assembly instructions may deviate.

3.7 Adjust the backrest


The height of the backrest can be adjusted over a distance of 35 mm (step less) according following instructions: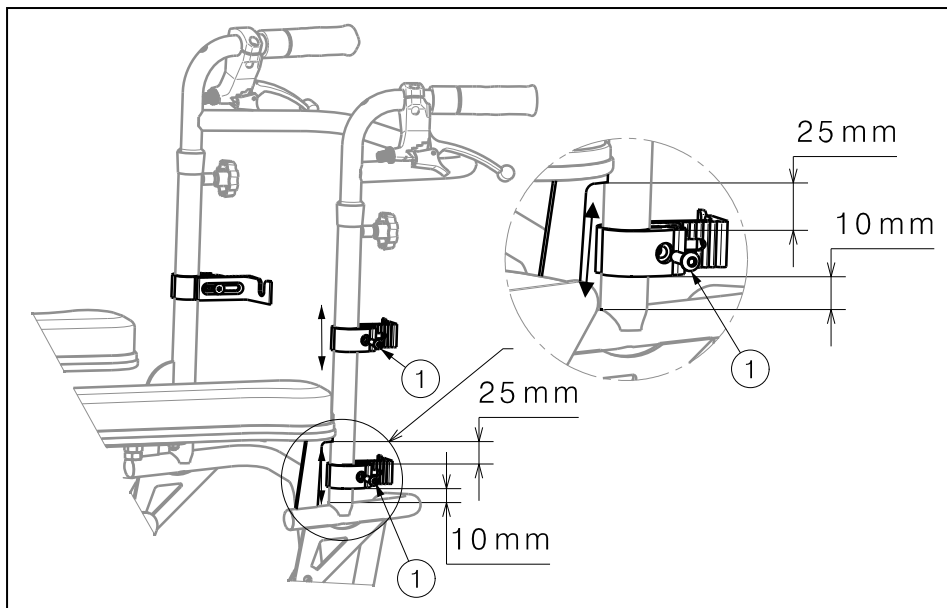


1. Loosen the bolts ① of the four brackets.
2. Adjust the brackets of the backrest to the desired height.
3. Retighten the bolts ① of the four brackets.

3.8 Adjust the brakes

Adjust the brakes according following rules: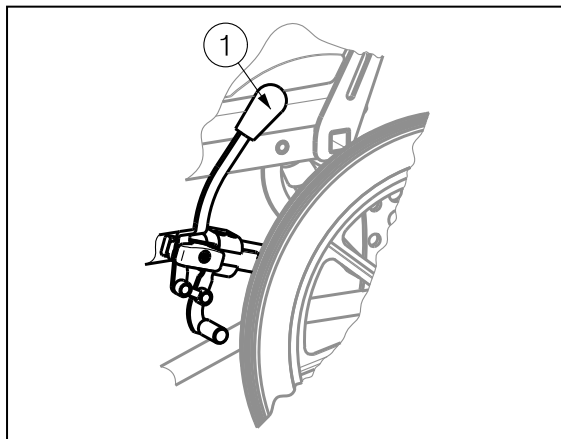

1. Install the wheels according § 2.2 for quick release 22", 24" rear wheels.
2. Disconnect the brakes by pulling lever ① backwards.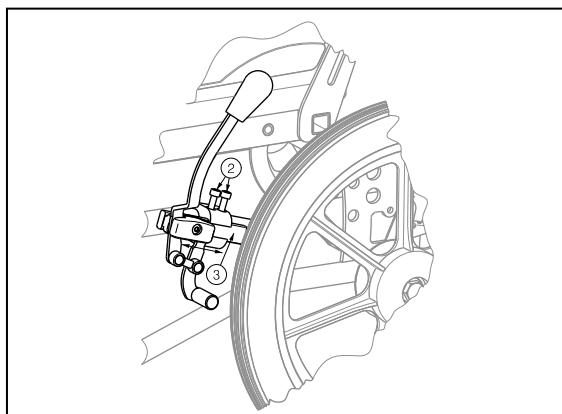


3. Loosen the bolts ② so the brake mechanism can slide over the guide ③.
4. Pull the brake mechanism over the guide ③ to the desired position.
5. Retighten the bolts ②.
6. Check working of the brakes.
7. If necessary repeat the above steps until the brakes are adjusted well.

3.9 Adjusting of the footrests


3.9.1 Length of the footrests

⚠ CAUTION: Risk of damage – Avoid that the footrests make any contact with the ground. Keep a minimum distance from 60 mm above the ground.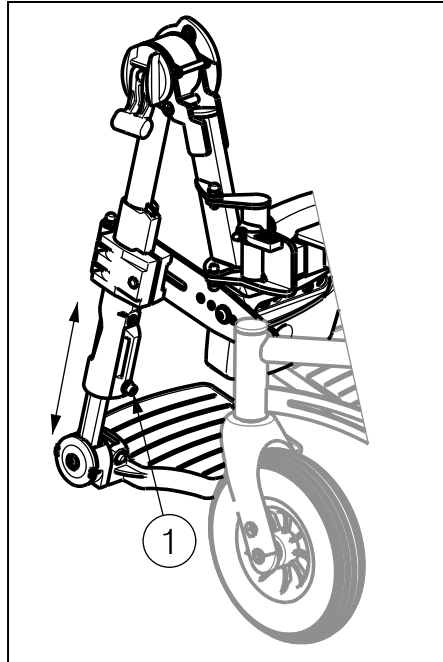

Adjust the length of the footrests as follow:

1. Remove the screw ① (on the back of the footrests).
2. Adjust the length of the footrest to a comfortable length.
3. Tighten the screw ① properly.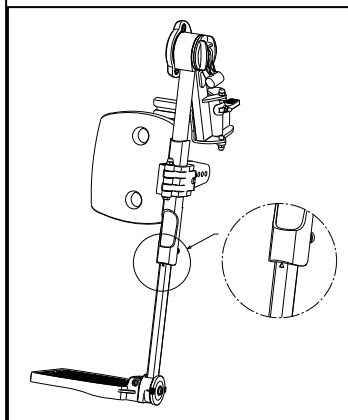


The max adjustable length of the footrest shall be marked with a triangle on the tube.

Not possible when footrest is in vertical position.

3.9.2 Adjusting leg rest


The leg rests are available in different sizes. Small size for seat width 400 mm wheelchair and medium size for wheelchair with seat widths 450 mm and 500 mm.

Adjusting the height: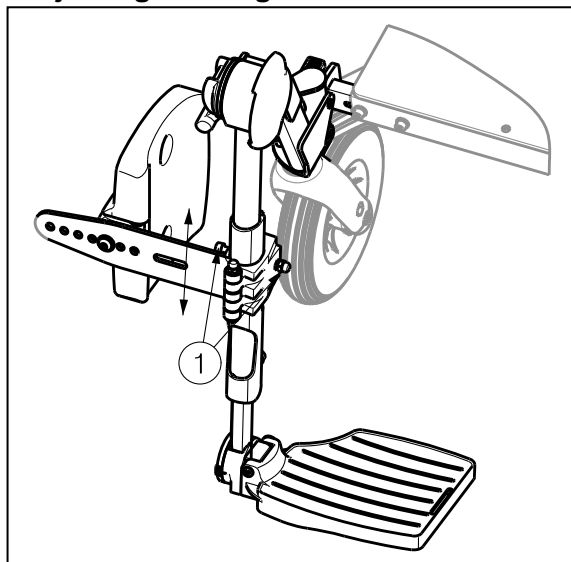


1. Fold away leg rest (according paragraph 2.5).
2. Loosen bolt ① on the side of the footrest.
3. Adjust the height of the leg support to a comfortable position (step less).
4. Retighten the bolt ①.

Adjusting the depth: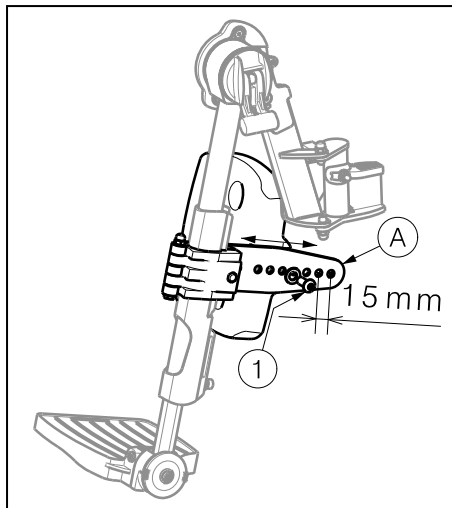


1. Loosen bolt ① on plate ①.
2. Adjust the depth of the leg rest to a comfortable position (7 different positions in steps of 15 mm).
3. Retighten the bolt ①.

Adjusting the width: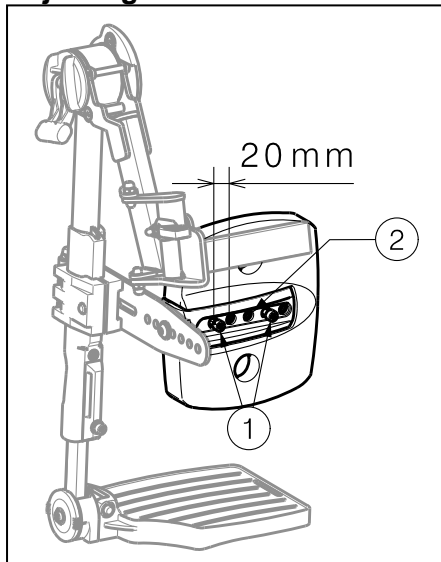

1. Loosen the two bolts ① on the back of leg rest cushion.
2. Adjust the width of the leg rest to a comfortable position (2 different positions in steps of 20 mm, **do not use the middle hole ② for adjusting**).
3. Retighten the two bolts ①.

3.9.3 Adjusting angle footplate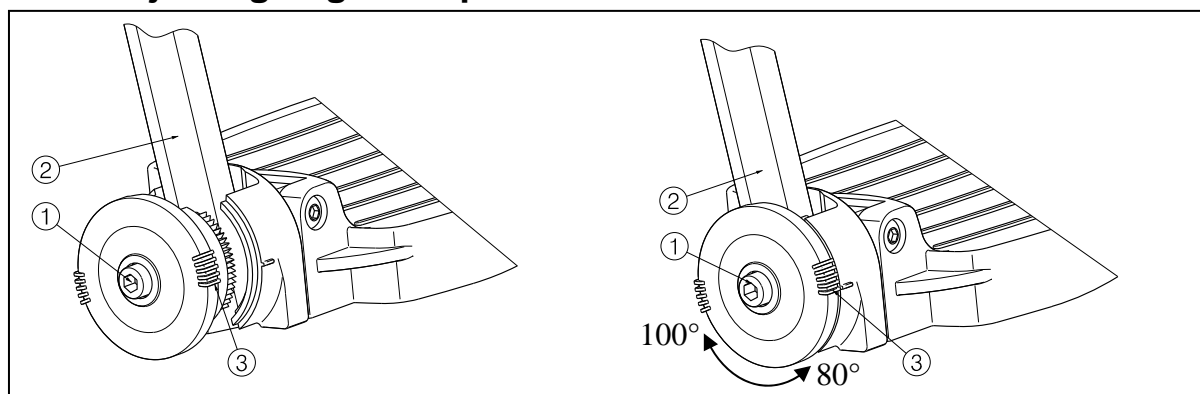


Adjust the angle of the footplate as follow:

1. Loosen bolt ①.
2. Adjust tube ② in the desired angle. The angle indication (80°-85°-90°-95°-100°) is according the dashes ③.
3. Retighten the bolt ①.

3.9.4 Adjusting width of the footrest

- ⚠ **WARNING:** Risk of injury – Make sure that all screws are secure and properly tightened before using the footrests.
- ⚠ **WARNING:** Risk of injury – Make sure that the footrests are positioned symmetrically from the seat.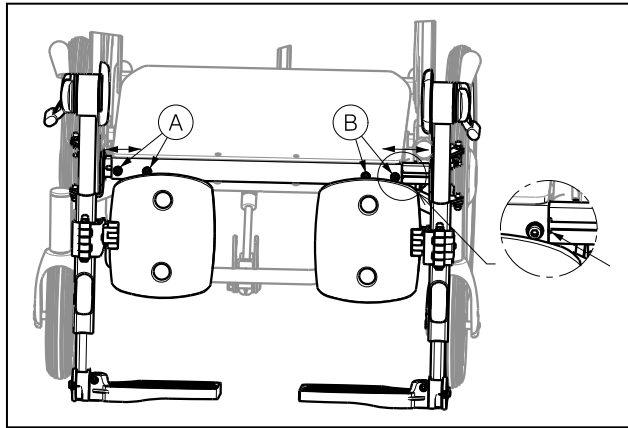


Adjust the width of the footrest as follow:

1. Remove the bolts Ⓐ for right footrest and Ⓑ for left footrest on the front of the seat frame.
2. Adjust the footrest in the desired position.
3. Retighten the bolts Ⓐ and Ⓑ properly by hand.

Pull out the footrest socket until the mark line in the tube (maximum position).

3.10 Adjusting the armrests


3.10.1 Adjusting depth of the armrests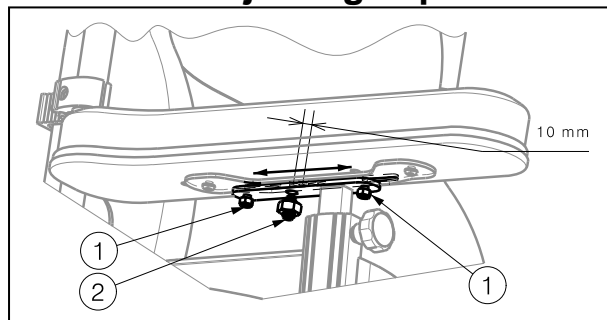

1. Loosen the fixing screws ① beneath the arm cushion.
2. Remove the nut + spring plunger ②.
3. Push the arm cushion into the desired position.
4. Retighten the fixing screws ① and nut + spring plunger ②.

3.10.2 Adjusting width of the armrests

- ⚠ **WARNING:** Risk of injury – Make sure that all screws are secure and properly tightened before using the armrests.
- ⚠ **WARNING:** Risk of injury – Make sure that the armrests are positioned symmetrically from the seat.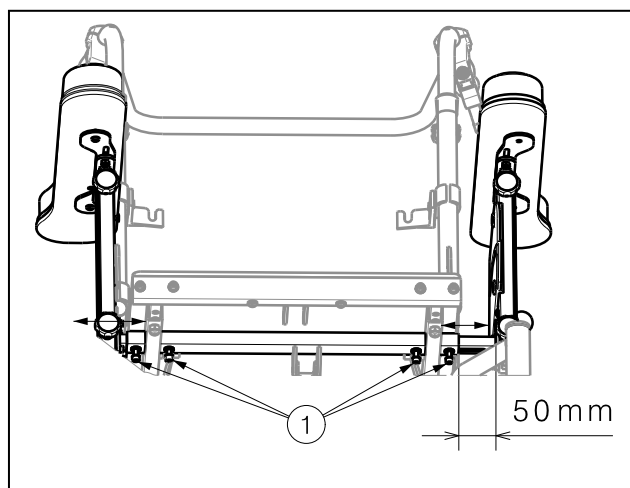

1. Loosen the screws ① of the armrest sockets beneath the seat.
2. Pull the armrest socket into the desired position (max. of 50 mm outwards).
3. Retighten the screws ① firmly by hand.


Pull out the armrest socket only to the extent that both screws on one side still secure the armrest socket.

3.11 Adjusting of the tip cap

⚠ WARNING: Risk of injury - Make sure that the spring button ① is locked after adjusting the tip cap.

A tip cap is mounted on the lower back frame that can be adjusted in its height depending on the seat height of the wheelchair.

The tip cap can be placed in 3 different positions with a difference of 20 mm.

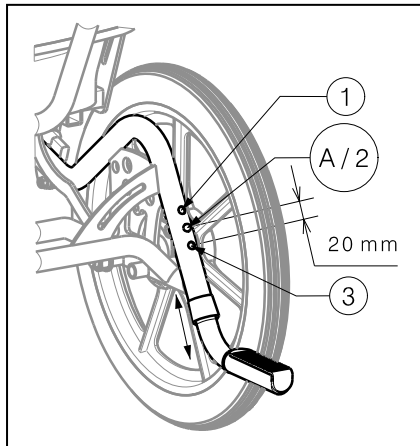1. Press the spring button ①.
2. Press the tip cap tube into the frame tube or out the frame tube until the desired position is reached.
3. Loosen the spring button ①.
4. Check that the spring button ① is securely fixated.

4 Maintenance

For the maintenance manual of the wheelchairs refer to the Vermeiren website:
www.vermeiren.com.

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CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles and other products: 2 years (batteries 6 months) and multipoosition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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Vermeiren Austria GmbH
Vermeiren Suisse S.A.