# VERMEIREN

## Izzie

USER MANUAL  
MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

EN

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## Izzie bathroom bar

### 1 Upon delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- 1 bathroom bar
- 6 screws (8 for Izzie R/L)
- 6 wall plugs (8 for Izzie R/L)
- 2 rubbers (3 for Izzie R/L)
- Manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

### 2 Mounting the bathroom bar

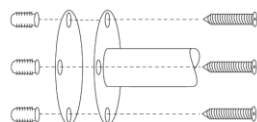
**CAUTION** Risk of injuries

- Make sure that the bathroom bar is mounted securely to the wall. Check the fixation on a regular basis.
- Be sure that there are no electrical cables or other tubes in the mounting area of the bathroom bar.
- The plugs delivered with the product are suitable for concrete walls or full brick walls. For other types of walls, please inquire about the correct type of mounting equipment.

The bars are designed as general support elements. They are useable everywhere, including humid environments. Use the installation materials which are delivered with Izzie.

To install Izzie:

1. Put the bathroom bar in the desired position.
2. Mark the fixation holes on the wall.
3. Drill holes in the wall.
4. Mount the plugs.
5. Mount the rubber gaskets.
6. Place the bathroom bar against the wall.
7. Place the screws and fasten firmly.

If you have any doubts, do not hesitate to contact a professional to install the Izzie for you.

### 3 Using the bathroom bar

**CAUTION** Risk of burns

- Be careful when using in extreme hot or cold environments (sunshine, extreme cold, saunas, etc.).
- If you notice any defects, refrain from using Izzie.

The maximum pulling weight for the bathroom bar is 130 kg.

### 4 Maintenance

Izzie can be cleaned and disinfected with normal household products. Avoid using abrasive cleaning products that can cause discoloration.

The high quality of the surface layer guarantees optimal protection against corrosion. If the outer coat is damaged by scratching or in some other way, get your specialist dealer to repair the affected surface.

#### 4.1 Expected lifespan

The bathroom bar is designed to have an average lifespan of 5 years.

### 5 Technical specifications

<b>Brand</b>	Vermeiren
<b>Type</b>	Bathroom bar
<b>Model</b>	Izzie

Description	Dimensions
Maximum load	130 kg
Distance bar-wall	40 mm
Minimum mounting area	30 cm: 65 x 370 mm 40 cm: 65 x 470 mm 45 cm: 65 x 520 mm 60 cm: 65 x 670 mm 80 cm: 65 x 870 mm Izzie R/L : 480 x 880 mm
Storage temperature	+5°C - +41°C

We reserve the right to introduce technical changes.  
Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.

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### All rights reserved, including translation.