

18.08.2021


## ALYA Parts List



## ALYA Parts List

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
1		FRONT FRAME ASS'Y			
2		FRONT FORK ASS'Y			
3		REAR FRAME ASS'Y			
4		SHROUD ASS'Y			
5		COVER, DASHBOARD			
6		TILLER ASS'Y			
7		ELECTRONIC ASS'Y			
8		WIRE ASS'Y			
9		SEAT ASS'Y			
10		TIRE ASS'Y			
11		TOW HANDLE ASS'Y			
12		ACCESSORIES			


## ALYA Parts List

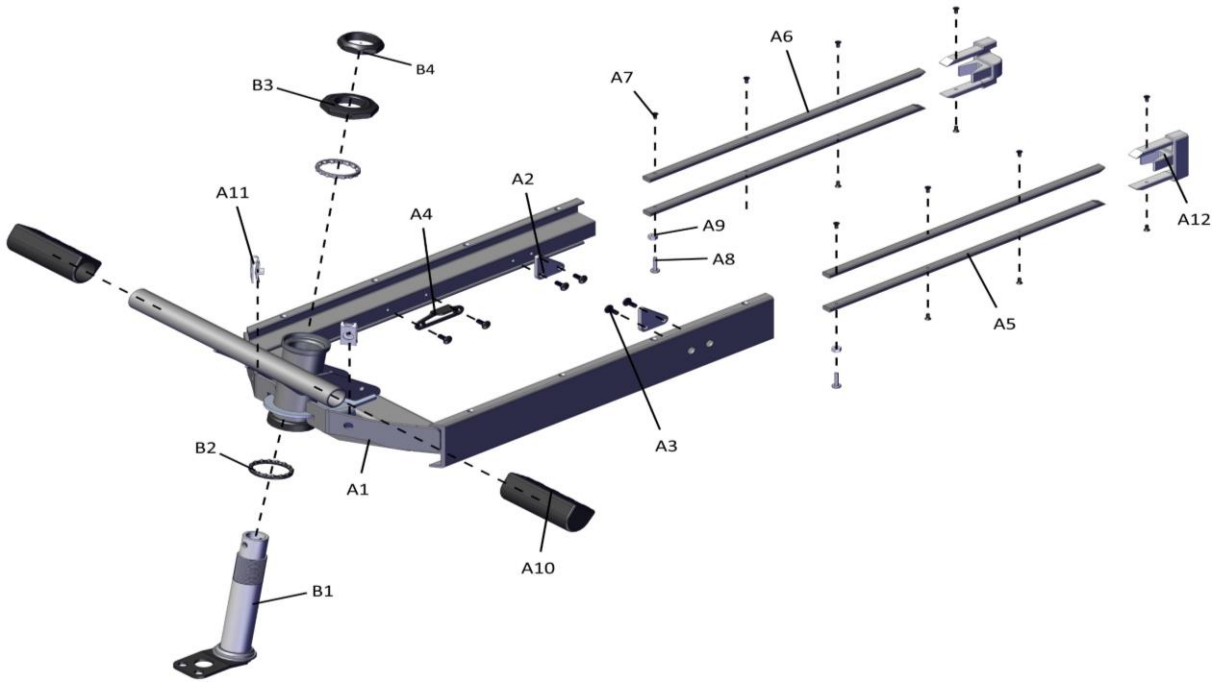

## ALYA Parts List

Fig&In dex NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
1		<b>FRONT FRAME ASS'Y</b>			
A1		FRONT FRAME	1		
A2		STOPPING PLATE	2		
A3		CROSS SCREW,M4x10L	6		
A4		PLATE, MAGNET ADSORPTION	1		
A5		SPACER BLOCK-L	2		
A6		SPACER BLOCK-R	2		
A7		SELF TAPPING SCREW,M3x6L	14		
A8		SELF TAPPING SCREW,M3.5x15L	2		
A9		HEXAGON NUT,M4	2		
A10	1916800	FOOT RUBBER	2		
A11		U-TYPE SPRING NUT,M6(4mm)	2		
A12		FRONT FRAME PIPE PLUG	2		
B1		LTYPE STEERING AXLE	1		
B2	1916801	BEARING	2		
B3	1916802	BEARING BUSH	1		
B4	1916803	LOCKING NUT, BEARING	1		


## ALYA Parts List

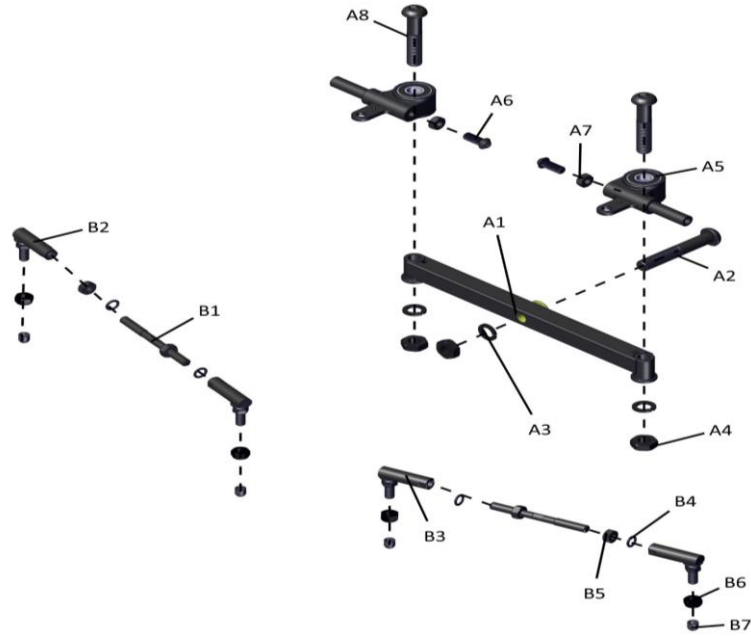

## ALYA Parts List

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
<b>2</b>		<b>FRONT FORK ASS'Y</b>			
A1	1916804	FLOATING ARM	1		
A2	1916805	HEN SOCKET SCREW, M12x75L	1		
A3	1916806	SPRING WASHER, M12xT=2mm	3		
A4	1916807	HEXAGON NUT, M12	3		
A5	1916808	STEERING SHAFT ASS'Y	2		
A6	1916809	HEN SOCKET SCREW, M8x20L	2		
A7	2990534	HEXAGON NUT, M8	2		
A8	1916817	HEN SOCKET SCREW, M12x65L	2		
B1	1916818	STEERING ROD, M8x90L	2		
B2	1916820	BALL JOINT-L	2		
B3	1916821	BALL JOINT-R	2		
B4	1916823	SPRING WASHER, M8xT=2mm	4		
B5	1916822	HEXAGON NUT, M8	2		
B6	2990534	LOCKING NUT, M8	4		
B7		Ø7 ROUND PLUG	4		


## ALYA Parts List

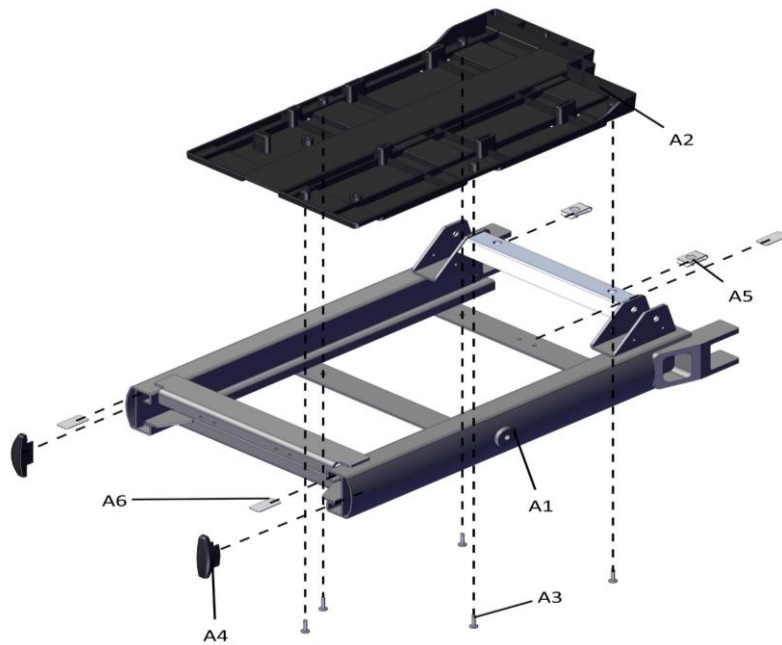

## ALYA Parts List

Fig&In dex NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
<b>3</b>		<b>REAR FRAME ASS'Y</b>			
A1		REAR FRAME	1		
A2		T4Q+ UNDERBODY COVER	1		
A3		SELF TAPPING SCREW,M3.5x15L	5		
A4		REAR FRAME FRONT PIPEPLUG	2		
A5		U-TYPE SPRING NUT,M6(4mm)	2		
A6		ANTI-WEAR TABLETS,13x28mm	3		

## ALYA Parts List

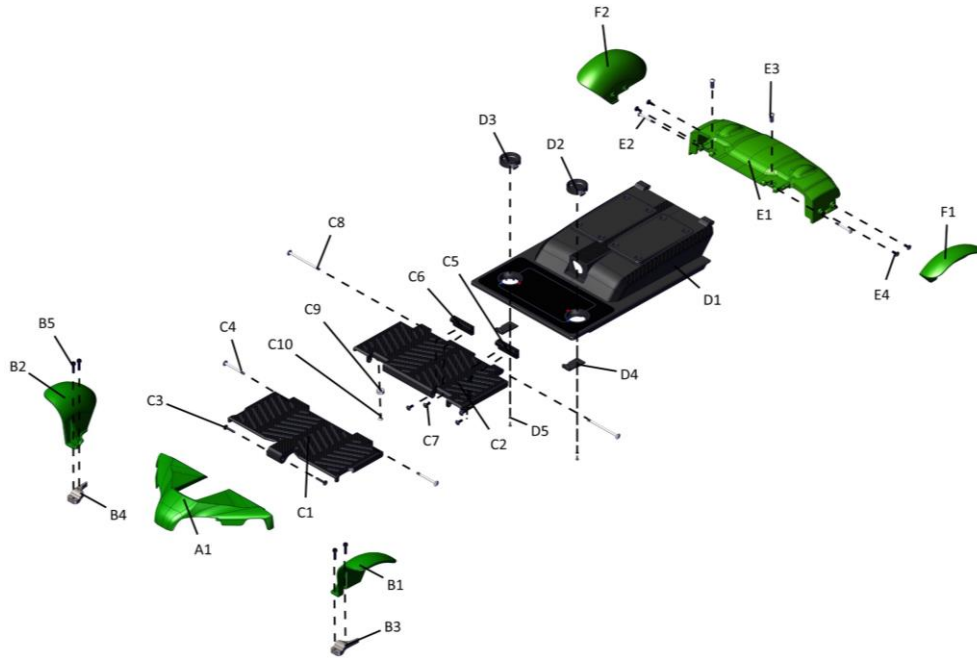


## ALYA Parts List

Fig&In dex NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
<b>4</b>		<b>SHROUD ASS'Y</b>			
A1	1916824	FRONT SHROUD,RED	1		
A1	1916825	FRONT SHROUD,SILVER	1		
B1	1916826	FRONT FENDER-L,RED	1		
B1	1916827	FRONT FENDER-L,SILVER	1		
B2	1916828	FRONT FENDER-R,RED	1		
B2	1916829	FRONT FENDER-R,SILVER	1		
B3		PAD, FRONT FENDER-L	1		
B4		PAD, FRONT FENDER-R	1		
B5	1916830	CROSS SCREW,M4x20L	4		
C1	1916831	FOOTREST SHROUD,FRONT	1		
c2	1916832	FOOTREST SHROUD,REAR	1		
C3	1916833	SELF TAPPING SCREW,M5x20L	2		
C4	1916834	CROSS SCREW,M4x44	2		
C5	1916835	PEDAL HINGE-L	1		
C6	1916837	PEDAL HINGE-R	1		
C7	1916838	CROSS SCREW,M4x10L	4		
C8	1916839	CROSS SCREW,M4x70	2		
C9	1916840	MAGNET	1		
C10	1916841	CROSS SCREW,M4x12L	1		

## ALYA Parts List

Fig&In dex NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
D1	1916842	BATTERY COVER	1		
D2	1916843	BATTERY COVER KNOB-L	1		
D3	1916844	BATTERY COVER KNOB-R	1		
D4	1916845	FIX PLATE	2		
D5	1916846	SELF TAPPING SCREW,M2.5x8L	4		
E1	1916847	REAR SHROUD,RED	1		
E1	1916848	REAR SHROUD,SILVER	1		
E2	1916849	CROSS SCREW,M3.5x35L	2		
E3	1916850	HEN SOCKET SCREW,M6x15L	2		
E4	1916838	CROSS SCREW,M4x10L	4		
F1	1916851	REAR FENDER-L,RED	1		
F1	1916852	REAR FENDER-L,SILVER	1		
F2	1916853	REAR FENDER-R,RED	1		
F2	1916854	REAR FENDER-R,SILVER	1		


## ALYA Parts List

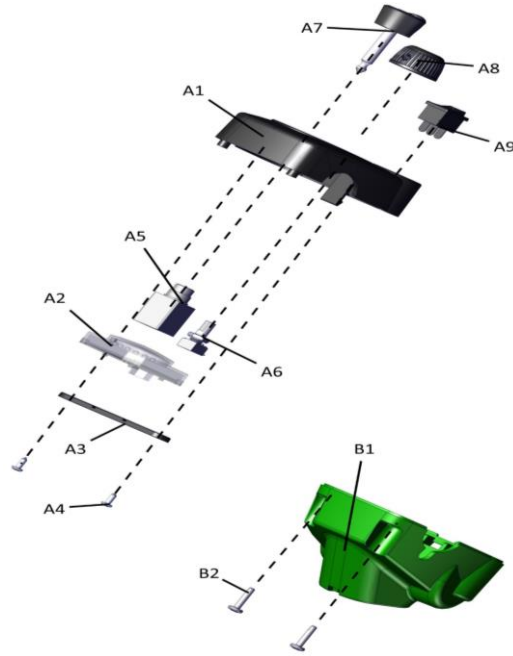

## ALYA Parts List

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
<b>5</b>		<b>COVER, DASHBOARD</b>			
A1	1916855	DASHBOARD COVER,UPPER	1		
A2	1916857	BATTERY INDICATOR	1		
A3		BATTERY INDICATOR PAD	1		
A4		ROUND HEAD CROSS SELF TAPPING SCREW,M3x8L	2		
A5	1916858	KEY SWITCH	1		
A6	1916859	SPEED VR	1		
A7	1916860	T4Q KEY	1		
A8	1916861	SPEED ADJUST KNOB	1		
A9	1916862	BACK/FORWARD SWITCH	1		
B1	1916863	DASHBOARD COVER,LOWER,RED	1		
B1	1916864	DASHBOARD COVER,LOWER,SILVER	1		
B2		SELF TAPPING SCREW,M3.5x15L	2		


## ALYA Parts List

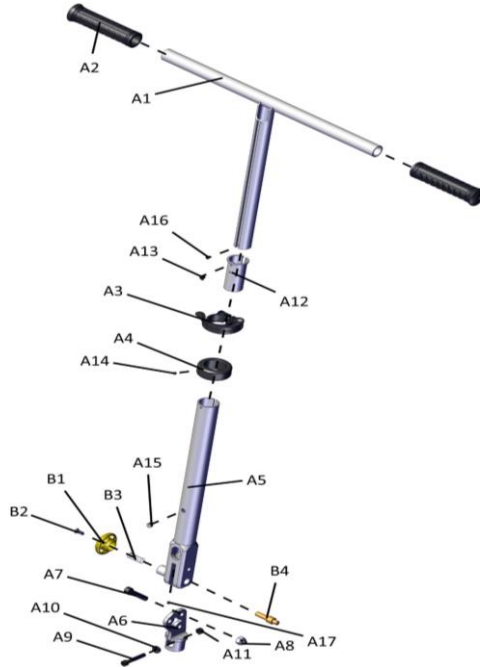

## ALYA Parts List

Fig&In dex NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
<b>6</b>		<b>TILLER ASS'Y</b>			
A1	1916869	TILLER	1		
A2	1916870	TILLER RUBBER	2		
A3		TILLER PIPE CLAMP	1		
A4		TILLER STOP RING	1		
A5	1917954	TILLER SWIVEL, UPPER	1		
A6	1917953	TILLER SWIVEL, LOWER	1		
A7		HEN SOCKET SCREW,M8x30L	1		
A8		NYLON CAP NUT,M8	1		
A9		HEN SOCKET SCREW,M6x40L	1		
A10		WASHER	1		
A11		LOCKING NUT,M6	1		
A12		TILLER BUSH	1		
A13		CROSS SCREW,M4x6L	1		
A14		LOCKING SCREW 4x4	1		
A15		TILLER HOLE PLUG	1		
A16		HEN SOCKET SCREW,M3x3L	1		
A17		LOCKING SCREW 4x4	1		
B1		TILLER POSITIONING HANDLE	1		
B2		TRUSS SCREW,M5x12L	1		
B3		SPRING, POSITIONING BUCKLE	1		
B4		HANDLE DOWEL PIN	1		

## ALYA Parts List

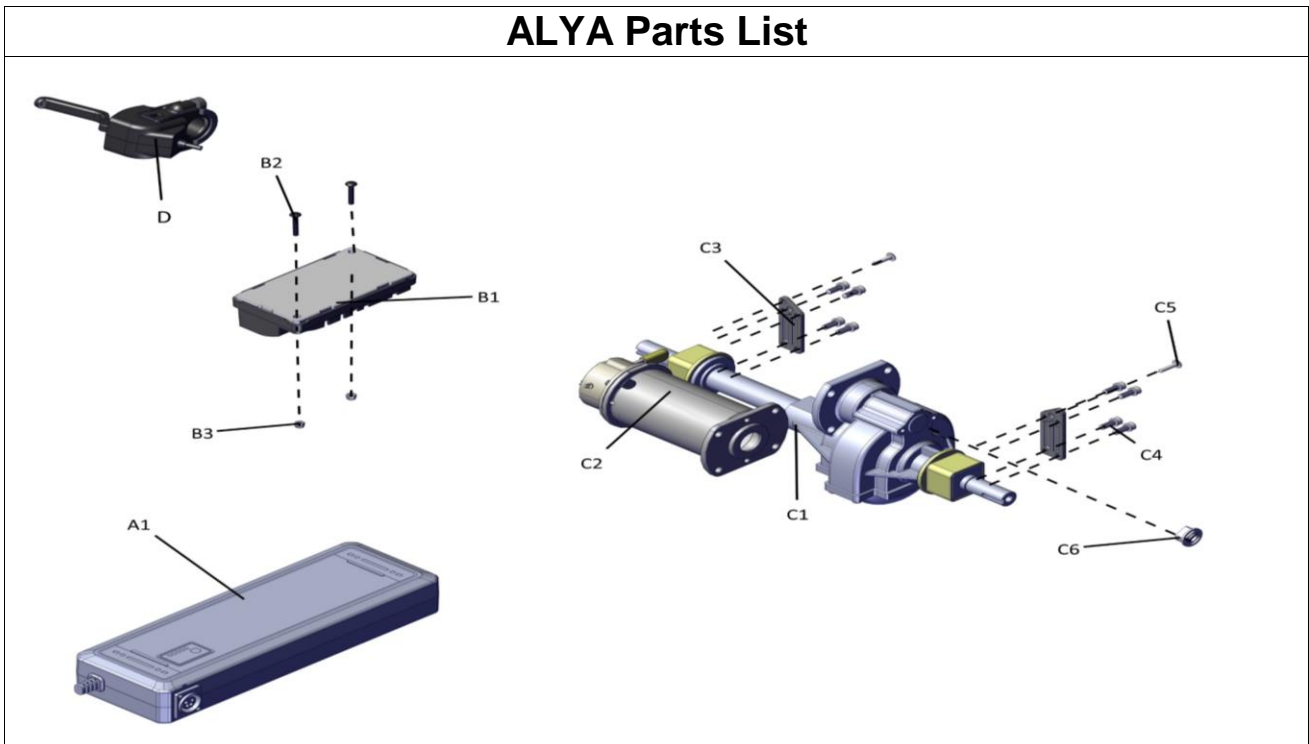


## ALYA Parts List

Fig&In dex NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
<b>7</b>		<b>ELECTRONIC ASS'Y</b>			
A1	1916872	LITHIUM BATTERY-11.6AH	1		
A1	1916873	LITHIUM BATTERY-17.4AH	1		
B1		CONTROLLER-DYNAMIC	1		
B2		CROSS SCREW,M4x25L	2		
B3		LOCKING NUT,M4	2		
C1	1916874	TRANSAXLE ASS'Y,SPEED RATIO1:21	1		
C2	1916871	MOTOR ASS'Y,180W/4700PRM/24V	1		
C3		HOLDER PLATE,TRANSAXLE	2		
C4		HEX SOCKET SCREW,M5x15L	8		
C5		SELF TAPPING SCREW,M4x20L	2		
C6		COUPLER	1		
<b>D</b>	<b>1916875</b>	<b>VR ASS'Y</b>	<b>1</b>		




## ALYA Parts List

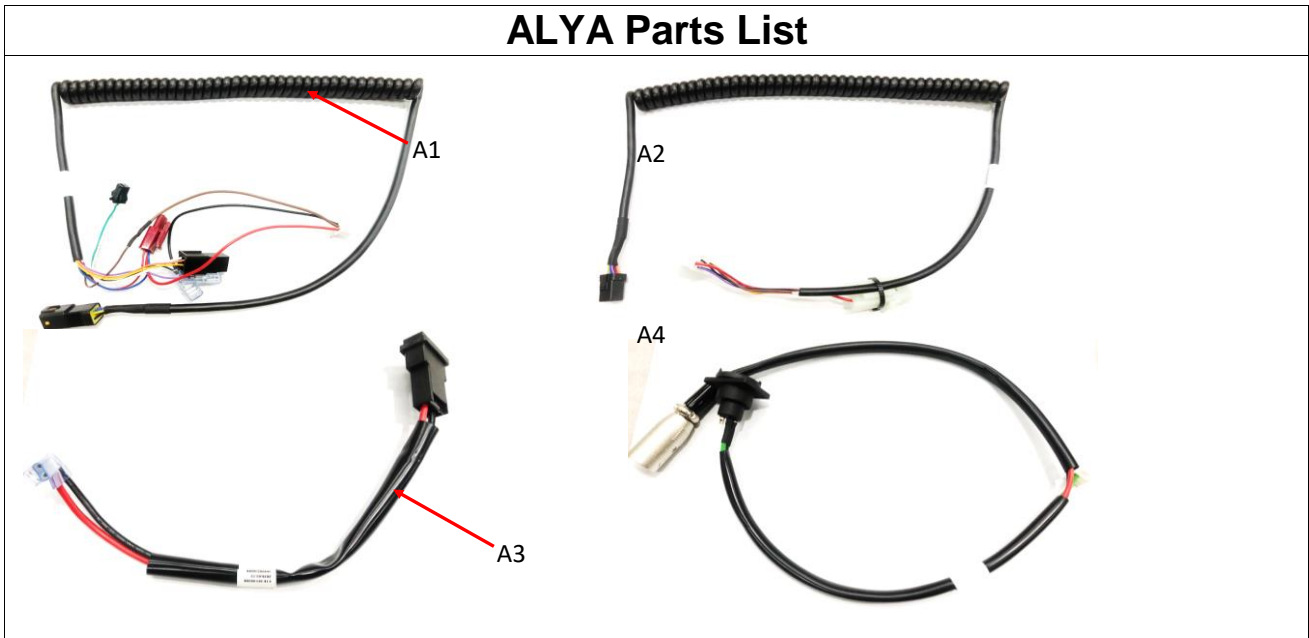

## ALYA Parts List

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
<b>8</b>		<b>WIRE ASS'Y</b>			
A1	1916876	WIRE HARNESS, UPPER	1		
A2	1916877	WIRE HARNESS, LOWER	1		
A3	1916878	POWER HARNESS	1		
A4	1916879	CHARGE HARNESS	1		


## ALYA Parts List

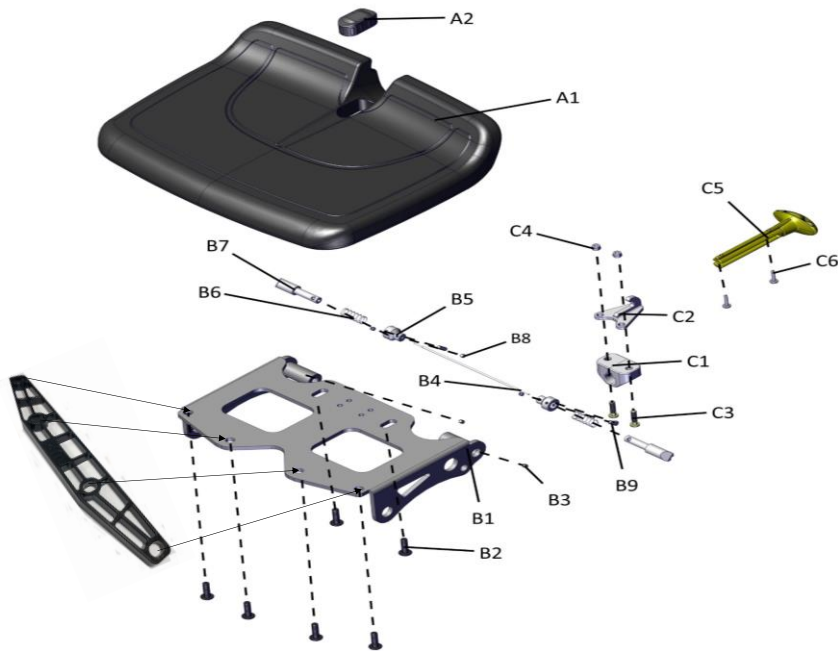

## ALYA Parts List

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
<b>9</b>		<b>SEAT ASS'Y</b>			
A1	1916880	SEAT	1		
A2		SEAT PLUG	1		
B1		SEAT PLATE	1		
B2	1009588	CROSS SCREW,M6x30L	6		
B3		LOCKING SCREW, 4x5	2		
B4	1917216	SEAT RELEASE ROPE	1		
B5		FIXING BOLT, SEAT RELEASE ROPE	2		
B6		SPRING, POSITIONING	2		
B7		POSITIONING BUCKLE	2		
B8		LOCKING SCREW, 4x4	4		
B9		LOCKING SCREW, 4x12	2		
C1		SEAT RELEASE HANDLE MOUNT	1		
C2		FIXED HOOK, SEAT	1		
C3		CROSS SCREW,M5x20L	2		
C4		LOCKING NUT,M5	2		
C5	1916881	SEAT RELEASE HANDLE	1		
C6		SELF TAPPING SCREW,M3.5x15L	2		
C7	1917381	DEAT ANGLE PLATE	1		
	1916907	BACK ASS ( S07-301-00300)			


## ALYA Parts List

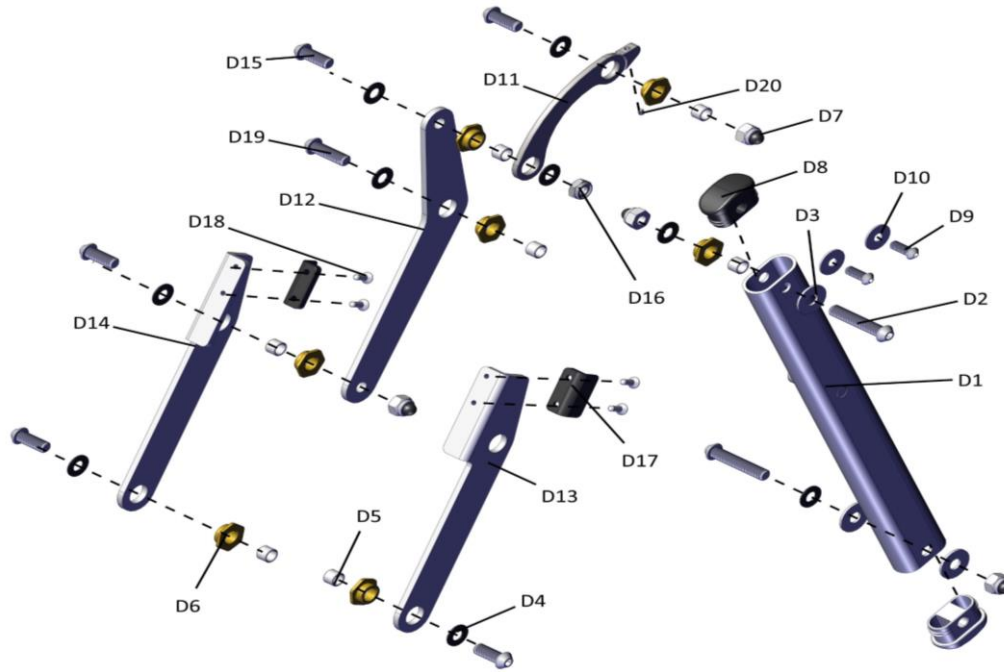

## ALYA Parts List

Fig&In dex NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
D1		SEAT BRACKET	2		
D2		HEN SOCKET SCREW,M8x40L	4		
D3		FLAT WASHER,M8xT=1.5mm	6		
D4		FLAT WASHER,M8xT=1mm	16		
D5		SWINGING ARM SPACER	12		
D6		SWINGING ARM BUSH	12		
D7		NYLON CAP NUT,M8	8		
D8		SEAT BRACKET PLUG	4		
D9		HEN SOCKET SCREW,M6x15L	4		
D10		FLAT WASHER,T=1mm	4		
D11		SEAT CONNECTION ROD,UPPER	2		
D12		SEAT CONNECTION ROD,MIDDLE	2		
D13		SEAT CONNECTION ROD,LOWER, L	1		
D14		SEAT CONNECTION ROD,LOWER, R	1		
D15		HEN SOCKET SCREW,M8x20L	8		
D16		LOCKING NUT,M8xT=6.3	2		
D17		STOP BLOCK	2		
D18		CROSS SCREW,M4x8L	4		
D19		HEN SOCKET SCREW,M8x25L	2		
D20		LOCKING SCREW,M3	2		


## ALYA Parts List

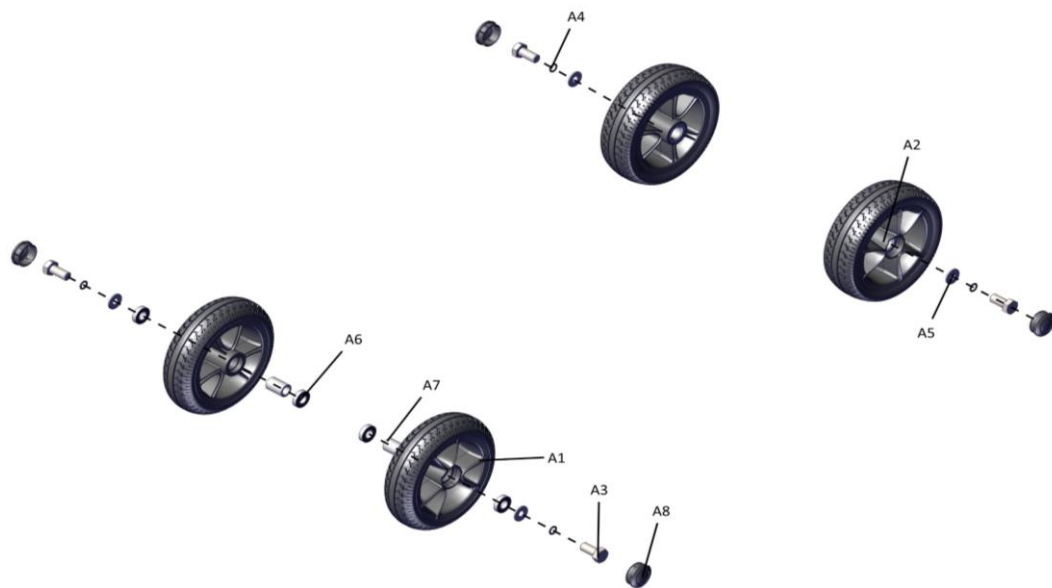

## ALYA Parts List

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
<b>10</b>		<b>TIRE ASS'Y</b>			
A1	1916882	6.5 "FRONT WHEEL PU TIRE, BLACK	2		
A2	1916883	6.5" REAR WHEEL PU TIRE, BLACK	2		
A3	1916884	HEX SCREW, M8x20L	4		
A4	1916885	SPRING WASHER, M8xT=2mm	4		
A5	1916886	FLAT WASHER, M8xT=1.5mm	4		
A6	1916887	BEARING 6901-2RS	4		
A7	1916888	FRONT WHEEL SPACER	2		
A8	1916889	RIM COVER	4		

## ALYA Parts List

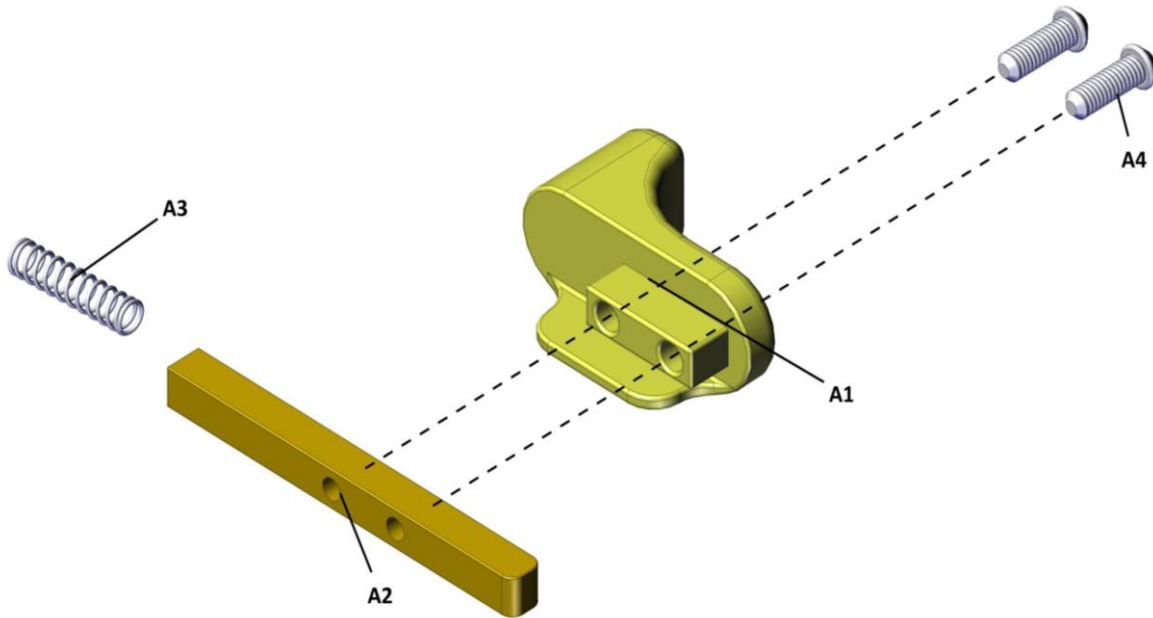

## ALYA Parts List

Fig&In dex NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
<b>11</b>	<b>1916906</b>	<b>TOW HANDLE ASS'Y</b>			
A1		TOW HANDLE RELEASE BUTTON(PA6-YELLOW)	1		
A2		TOW HANDLE FIXED BUCKLE	1		
A3		TOW HANDLE FIXED BUCKLE SPRING	1		
A4		HEN SOCKET SCREW,M4x12L	2		

## ALYA Parts List

Fig&In dex NO.	Parts No.	Parts Name	QTY/CAR	Price	Remark
		Description			
<b>12</b>		<b>ACCESSORIES</b>			
A1		FOAM GLUE,BLACK	2		
B1	1916890	CHARGER FOR LITHIUM BATTERY(WITHOUT POWER C	1		
B2	1916891	CHARGER POWER CABLE	1		
	1916902	CHARGER COMPLETE			CONTENT IS B1 AND B2
C1		OWNER'S MANUAL-VERMEIREN	1		
C2		WARNING STICKER,PUT DOWN HANDLE	1		
C3		WARNING STICKER, BEFORE OPERATING THE SCOOTER	1		
C4		REAR CAR REFLECTIVE STICKER-R,RED	1		
C5		REAR CAR REFLECTIVE STICKER-L,RED	1		
C6		WARNING STICKER, EMI	1		
C7		STICKER, CE30x23mm	1		
C8		WARNING STICKER,ANTI-PINCH	6		
C9		STICKER, CONTROL PANEL	1		
C10		FRONT SHROUD ANTI-WEAR STICKER	1		
C11		SERIAL NUMBER STICKER	1		Provided by customers
C12		LOGO STICKER-VERMEIREN	1		Provided by customers
C13		CLUTCH STICKER	1		