





Detail A
Scale: 1:4

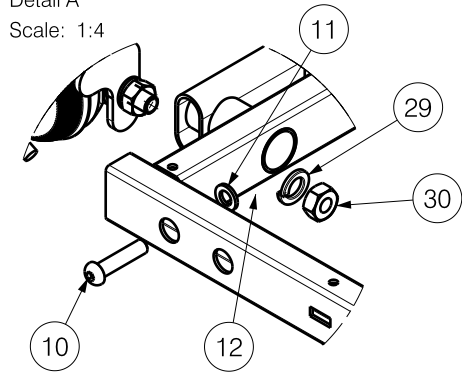

Isometric view
Scale: 1:2

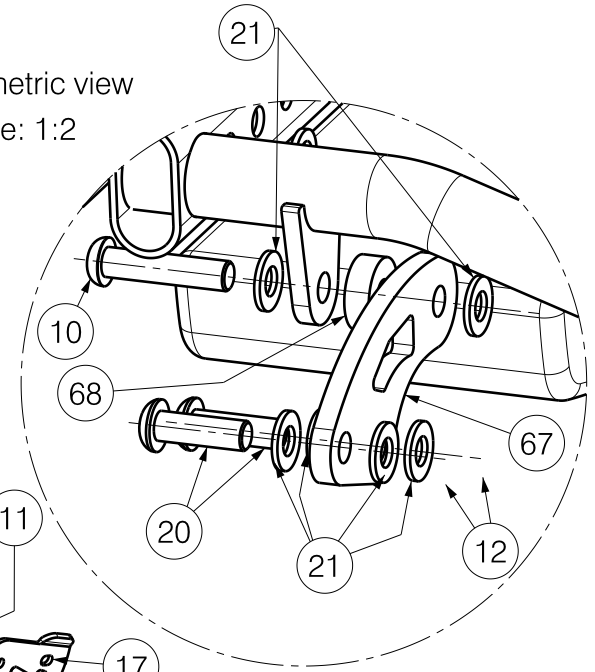

Isometric view
Scale: 1:3

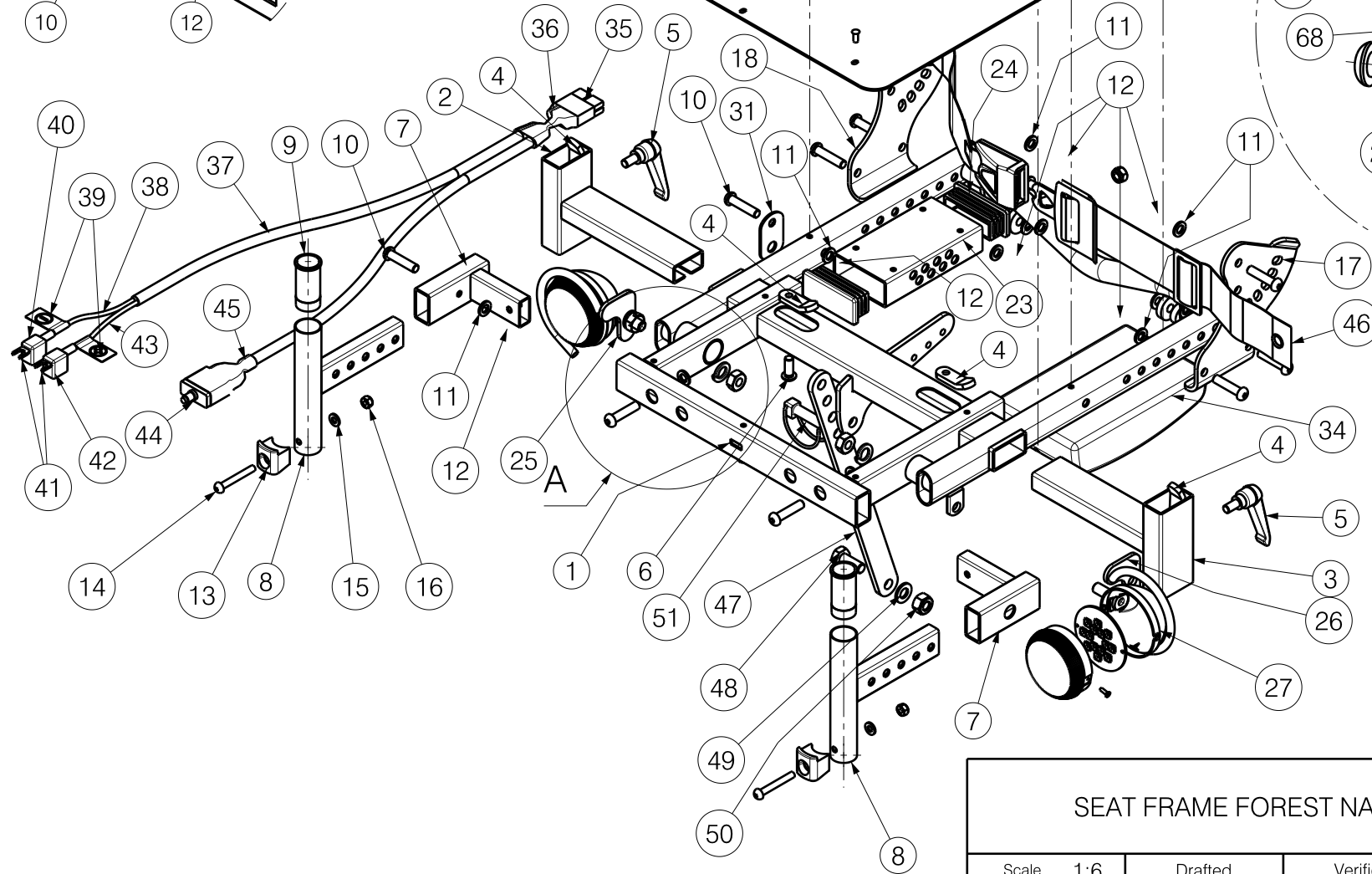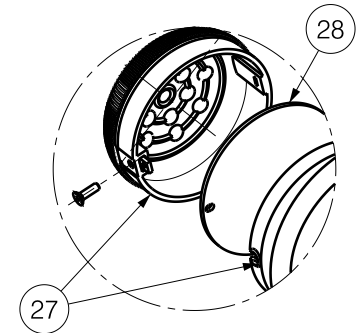

EP00042/C


SEAT FRAME FOREST NARROW KIDS 1/3				Plan No. EP00042 /C
Scale 1:6	Drafted	Verified	Confirmed	
Date	11/07/2011	28/09/2012	28/09/2012	
Init.	AZ	BS	TK	

Material			
No.	Plan	Article	Number
1	YP02039	18917	1
2	YP01678	18793	1
3	YP01679	18792	1
4	Y003955	14960	4
5	---	1006661	2
6	M8x20	1992349	2
7	YP03147	28249	2
8	YP02052	18913	2
9	X002559	1993949	2
10	M8x35	1993236	12
11	D8,2	1001556	10
12	M8	1001682	18
13	X002135	1992405	2
14	M6x45	1003109	2
15	D6,4	1001564	2
16	M6	1001680	2
17	YP01630	18758	1
18	YP01629	18757	1
19	YP02048	18912	1
20	M8X25	1990501	4
21	D8,5	1003199	12
22	---	2990455	2
23	YP01841	18812	1
24	---	1997465	2
25	YP01827	18808	1
26	YP01826	18809	1
27	---	1991682	2
28	XP00438	2990474	2
29	D10,5	1001566	2
30	M10	1001672	2
31	Y003562	1474454	1
32	YP02055	18922	1
33	D4,5x11	1990752	11
34	---	1996994	1
35	---	1994573	1
36	---	T001045	1
37	---	H005006	2
38	---	L000034	1

Material			
No.	Plan	Article	Number
39	---	1901755	2
40	---	1901731	1
41	---	L000001	2
42	---	1994180	1
43	---	L000035	1
44	---	1000746	1
45	---	T001009	0,002
46	---	1992267	1
47	YP02034	18925	1
48	M10x25	1001450	1
49	10,5	1001558	1
50	M10	1001683	1
51	D10x40	1991481	1
52	---	2990592	1
53	---	2990593	1
54	YP02108	18939	1
55	Y006560	15750	2
56	---	2990351	2
57	YP02222	18948	1
58	YP02160	18935	4
59	YP02159	18934	4
60	---	1001580	2
61	M6X20	1006999	4
62	---	1001564	4
63	M6X12	1007000	4
64	YP02231	18949	1
65	---	1002399	0,94 m
66	Y006559	1995662	2
67	YP03273	28627	2
68	X004200	1902768	2

EP00042/C

SEAT FRAME FOREST NARROW KIDS 3/3				Plan No.
				EP00042 /C
Scale:	Drafted	Verified	Confirmed	
Date	11/07/2011	28/09/2012	28/09/2012	
Init.	AZ	BS	TK	

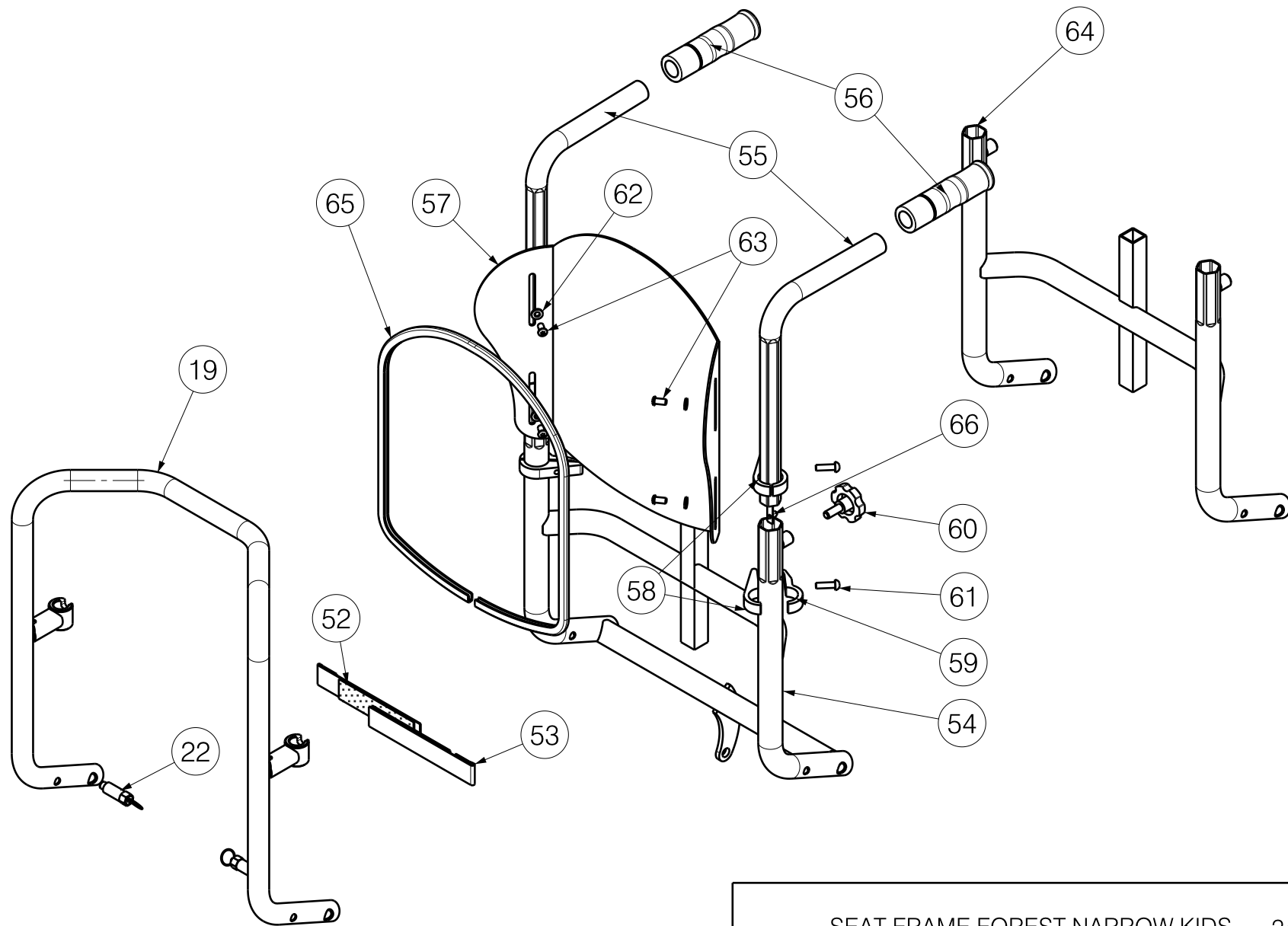

EP00042/C

SEAT FRAME FOREST NARROW KIDS 2/3				Plan No. EP00042 /C
Scale: 1:6	Drafted	Verified	Confirmed	
Date	11/07/2011	28/09/2012	28/09/2012	
Init.	AZ	BS	TK	