
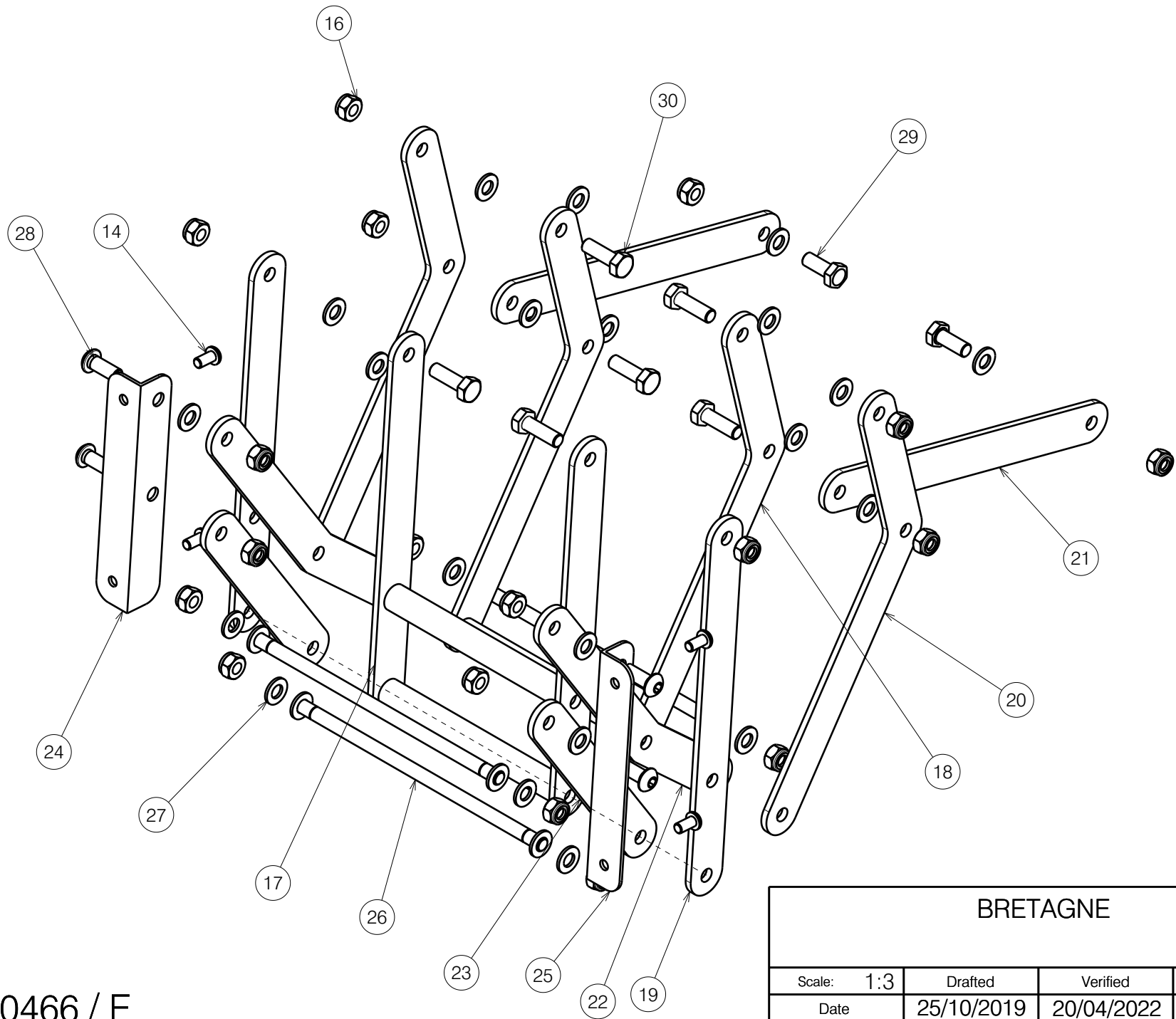



Add black plastic foot under wood (on both side)/
 Dodac czarna plastikowa stopke pod drewno (po dwoch stronach)

EP00466 / E

BRETAGNE				Plan No.
Sheet 1/3				EP00466/ E
Scale: 1:10	Drafted	Verified	Confirmed	
Date	25/10/2019	20/04/2022	20/04/2022	
Initials	PM	AZ	TK	



EP00466 / E

BRETAGNE				Plan No.
Sheet 2/3				EP00466/ E
Scale: 1:3	Drafted	Verified	Confirmed	
Date	25/10/2019	20/04/2022	20/04/2022	
Initials	PM	AZ	TK	

Material			
Nr	Plan	Article	Quantity
1	YP05829 DP	2726565	1
2a	YP05718	2722265	1
2b	YP08858 DP	2629865	1
3	XP02820 (beech)	HP00616	1
	XP02822 (oxide)	HP00618	1
4	XP02821 (beech)	HP00617	1
	XP02823 (oxide)	HP00619	1
5	X005930	1916350	4
6	Armrest left	EP00465	1
7	Armrest right	EP00465	1
8a	XP01828	2993544	1
8b	XP03185	2996809	1
9a'	XP01824	HP00263	1
10a	XP01827	2993543	1
10b	XP03183	2996807	1
11	M8X45 DIN933	1005901	1
12	M8X60 DIN912	1997781	2
13	Nut M6 DIN934	1001670	8
14	M6X12 ISO7380	1007000	16

Material			
Nr	Plan	Article	Quantity
15	Seat	EP00464	1
16	Nut M8 DIN985	1001682	19
17	YP00716	1825065	1
18	YP00715	1824965	1
19	YP00483	1822465	2
20	YP00568	1822265	2
21	YP05712	2722165	2
22	YP00491	1822365	2
23	YP00635	1822865	2
24	YP00412	1822565	1
25	YP00592	1822665	1
26	X001862	1447500	3
27	Washer M8	1003199	30
28	M8X20	1992349	4
29	M8X20 DIN933	1009400	2
30	M8x25 DIN933	1001442	4
31	YP06345	2735365	1
32	Cap 30x30	1003198	1
33	M6x16 DIN7991 Black	2994064	20
34*	Uph. footrest black	2994665	1
	Uph. footrest beige	2994667	1
	Uph. footrest brown	2994670	1
35*	Uph. backrest black	2994668	1
	Uph. backrest beige	2994669	1
	Uph. backrest brown	2994671	1

Material			
Nr	Plan	Article	Quantity
36	XP02829 (beech)	HP00624	1
	XP02830 (oxide)	HP00625	1
37	YP06347	2736065	1
38	YP06348	2736165	1
39	Washer M6	1001564	4
40	M6x20 DIN 7991	1903081	4
41b"	Y000462	F000050	2
42b"	X000273	G060600	2

a - method 1

b - method 2

choose only one method and don't mix method 1 and 2 parts

*not depicted on drawing

'only for method 1 - not needed in method 2

"only for method 2 - not needed in method 1

EP00466 / E

BRETAGNE				Plan No.
Sheet 3/3				EP00466/ E
Scale:	Drafted	Verified	Confirmed	
Date	25/10/2019	20/04/2022	20/04/2022	
Initials	PM	AZ	TK	