


		Material		
No.	Article	M	L	XL
1	2991014	0,12	0,18	0,25
2	1009517	4,6	4,6	4,6
3	1009512	12	12	12
4	1009508	6	6	6
5	2990337	0,001	0,001	0,001
6	2991191	2	2	2
7	1995604	0,18	0,23	0,23
8	1995605	1	1	1
9	2991886	3	3,5	4

	Sizes [cm]					Article	
	A	B	C	D	E	FR	BK
M	30,5	18	20,5	4	9	2991252	2991249
L	38	21,5	23	6	10	2991253	2991250
XL	46	25,5	27	6	10	2991254	2991251

Mounting			
No.	Plan	Article	Quantity
1	XP01976	2993641	4
2	XP01980	2993813	4
3	XP01979	2993812	4
4	M6X30 collar bolt	2994294	4
5	YP07178	2760099	4

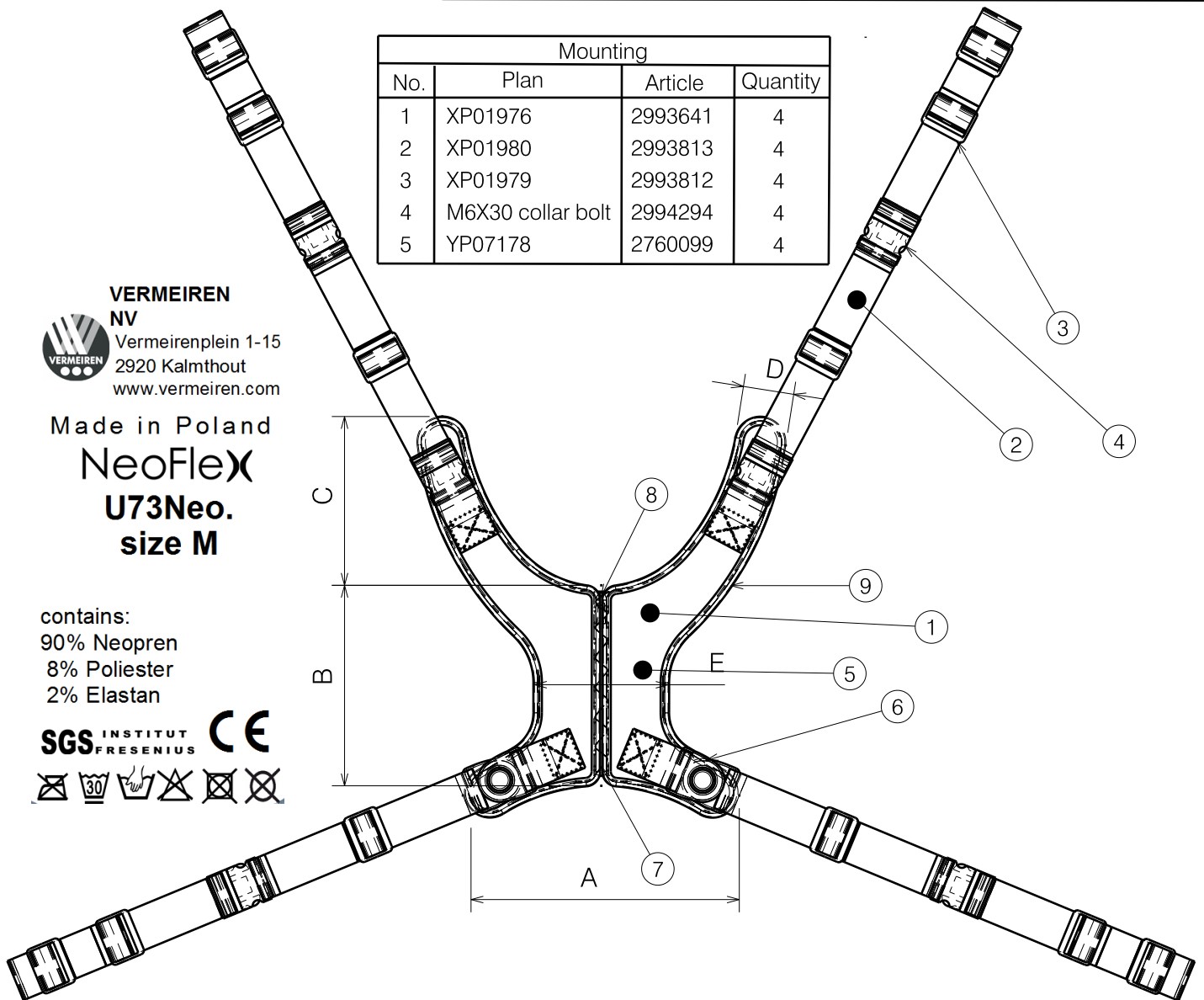

**VERMEIREN NV**  
 Vermeirensplein 1-15  
 2920 Kalmthout  
 www.vermeiren.com

Made in Poland  
**NeoFlex**  
**U73Neo.**  
 size M

contains:  
 90% Neopren  
 8% Poliester  
 2% Elastan



NEO U73N NEOFLEX STABILIZATION VEST				Plan No.
				EP00342/ B
Scale: N/A	Drafted	Verified	Confirmed	
Date	31/07/2018	04/03/2019	04/03/2019	
Initials	FD	AS	TK	

EP00342 / B