

E000733 / J

D100 B74/B80 DETACHABLE DRUM BRAKE WHEELS				Plan No. E000733/J
D100 B74/B80 AFNEEMBARE TROMMELREMWIELEN				
Scale: 1:4	Drafted	Verified	Confirmed	
Date	30/05/2011	10/12/2021	10/12/2021	
Initials	SD	AZ	JJQ	

Material				
No.	Plan	Article	Quantity	Description
1	---	3990041	2	HAND GRIP FOR ECLD CN /200F/300F/708D
2	X002356	1502800	2	DRUM BRAKE ATTACHMENT ECLIPS ZINK
3	Handlebar(22)	3990706	2	BRAKE SET WITH CABLE D22
	DRUMSET(L)	3990705	1	DRUM SET WITH AXLE LEFT SIDE
	DRUMSET(R)	3990708	1	DRUM SET WITH AXLE RIGHT SIDE
3A	X002437	1992699	2	NUT FOR ADAPTOR B80+B72 ECLIPS
3B	---	1906205	2	AXLE QUICK RELEASE ■12 B74
3C	---	1993218	(2X) 0.75m	CABLE HOSE GREY BLACK FOR ECLIPS
3D	---	1004094	(2x)0.95m	BRAKE CABLE 1.25M
3E	---	1004109	2	COVER FOR "STOP"CABLE ALU
3F	φ22	1994012	2	BRAKE LEVER WITH PAWL VERTICAL φ22
3G	---	1911653	2	LOCKWASHER DIN128 M16
3H	X005815	1914817	2	ADAPTOR FOR B80 B74 (62mm)
4	XC00017	1C11600	2	BUSHING φ12.3Xφ6,1X12 ZINC
5	X003986	1902199	2	ALU BUSHING φ25Xφ16,1X16
6	X004898	17551 (L)	1	WHEEL BLOCK LEFT V100G RAW
	X004897	17552 (R)	1	WHEEL BLOCK RIGHT V100G RAW
7	---	1001486	2	COUNTERSUNK SOCKET SCREWS DIN 7991 M6X50
8	Y006560	1575054	2	TUBE FOR GRIP D100 C13
9	---	1006999	2	HEXAGON SOCKET BUTTON HEAD SCREW M6X20
10	---	3990599	2	WHEEL 24"PU DRUMBRAKE STEEL HUB 36 SPOKE
3G	---	1993237	2	LOCKWASHER DIN128 M16
12	---	1008492	4	CLOSED NUT M6X12
13	---	1003909	2	HEX.SOCKET HEAD CAP SCREW DIN912 M6X40
14*				
15**	---	1001580	4	BOLT IN PVC HEAD ■ 40MM M8X20
16	Y006559	1995662	2	SPRING FOR HAND GRIP JAZZ 30
17	Y001772	17609	2	GRIP EXTENSION WITHOUT NUT
18	---	1001589	2	PIVOT 5X18 K11 POINT BOLT


3A - 3G part of 3

\*: not depicted

\*\* : two spare pieces

E000733 / J

D100 B74/B80 DETACHABLE DRUM BRAKE WHEELS				Plan No.
D100 B74/B80 AFNEEMBARE TROMMELREMWIELEN				E000733/J
Scale: 1:4	Drafted	Verified	Confirmed	
Date	30/05/2011	10/12/2021	10/12/2021	
Initials	SD	AZ	JJQ	

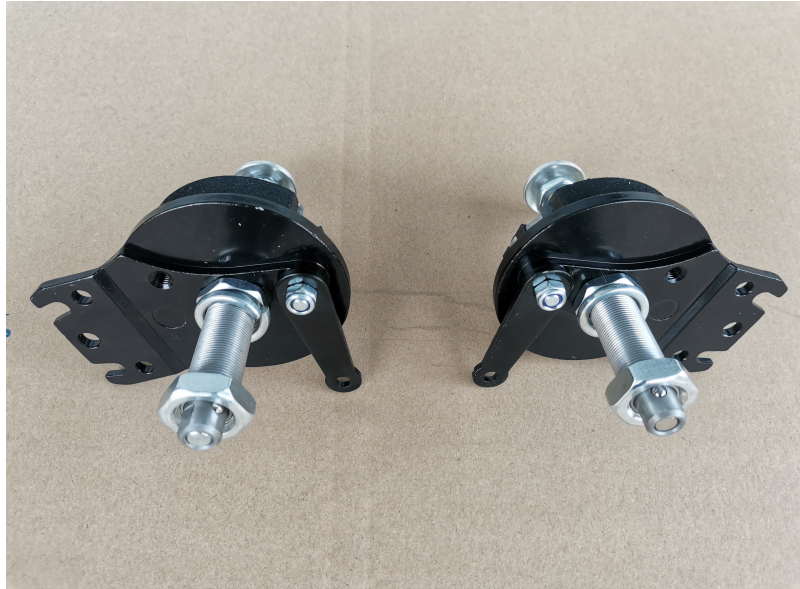

3990705

3990708



3990706

E000733/J

D100 B74/B80 DETACHABLE DRUM BRAKE WHEELS				Plan No. E000733/J
D100 B74/B80 AFNEEMBARE TROMMELREMWIELEN				
Scale:	1:4	Drafted	Verified	Confirmed
Date	20/08/2021	10/12/2021	10/12/2021	10/12/2021
Initials	AZ	AZ	JJQ	

